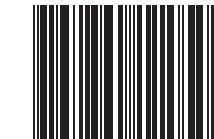
Manometer Accessory

for use with the **Aerobika*** Oscillating Positive Expiratory Pressure Therapy System

0543

Quantity: 1

REF 110502 0 762860 110502

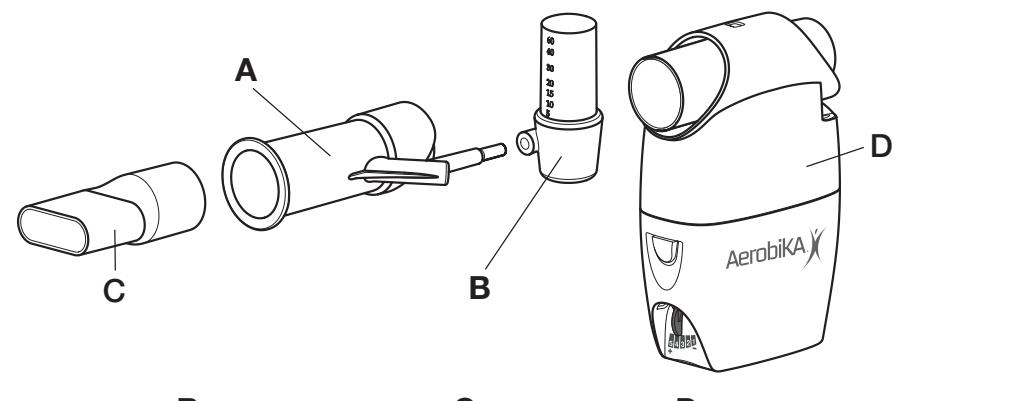
Trudell Medical International[®]

Trudell Medical International
725 Third Street, London, Ontario, Canada N5V 5G4
+1-519-455-7060 +1-519-455-6478
info@trudellmed.com www.trudellmed.com


110159-001 Rev A. * trade marks and registered trade marks of Trudell Medical International. Copyright © Trudell Medical International 2019.
All rights reserved.

EC REP Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands

Distributed in the United Kingdom by:
Trudell Medical UK Limited
info@trudellmedical.co.uk
01256 338400

A Manometer adapter	B Manometer gauge	C Mouthpiece	D OPEP device
Manometeradapter	Manometeranzeige	Mundstück	OPEP-Gerät
Adaptador del manómetro	Medidor del manómetro	Bocilla	Dispositivo OPEP
Adaptateur du manomètre	Jauge du manomètre	Embout buccal	Dispositif de PEP oscillante
Adattatore del manometro	Indicatore del manometro	Boccaglio	Dispositivo OPEP
Manometeradapter	Attezing manometer	Mondstuk	OPEP-apparaat
Adapter manometru	Wskaznik manometru	Ustnik	Aparat do leczenia
Manometre Adaptörü	Manometre Göstergesi	Ağızlık	oscylacyjnym dodatnim ciśnieniem wydechowym
مهانی جهاز قیاس الضغط	مقياس جهاز قیاس الضغط	الجزء الامامي	OPEP Cihazı
壓力計アダプター	圧力計ゲージ	マウスピース	OPEP装置
血压计转接器	血压计仪表	呼気嘴	OPEP 设备

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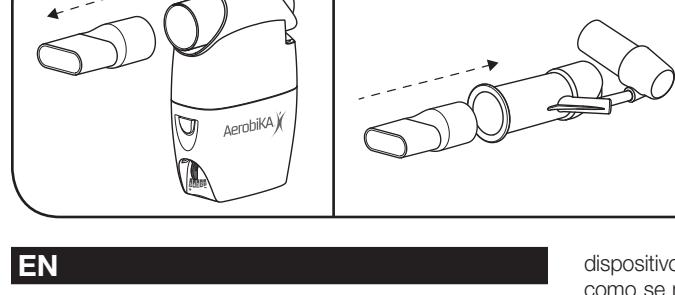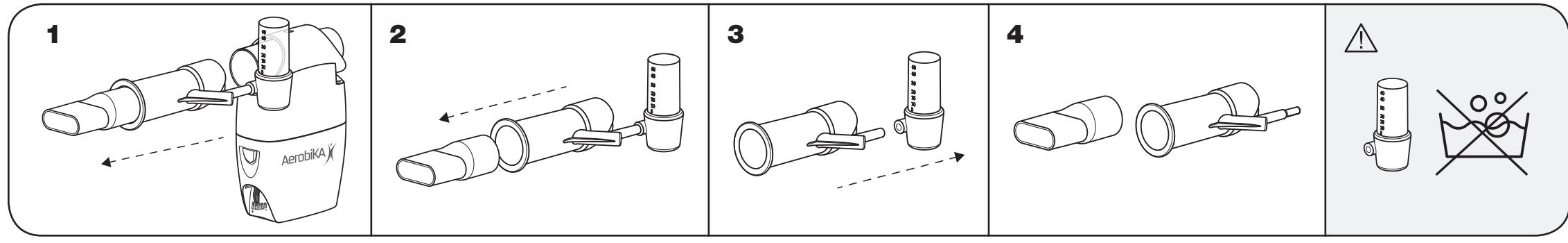
EN

CLEANING AND DISINFECTION YOUR MANOMETER ADAPTER

1 Remove the manometer adapter from the **Aerobika*** OPEP device. 2 Remove the mouthpiece, from the manometer adapter. 3 Remove the manometer gauge from manometer adapter. 4 The mouthpiece and manometer adapter may be cleaned in soap and water, allowing parts to soak for 15 minutes. Agitate gently. Rinse parts in clean water. **OR** Place parts in a dishwasher basket on the top rack. Wash in a normal cycle. Shake to remove excess water and allow to air dry thoroughly before reassembly.

DISINFECTION

The mouthpiece and manometer adapter may be disinfected weekly using any of the following methods:

- A Electronic Steam Sterilizer
 - B Boil for 5 minutes
 - C Microwave Steam Bag (according to manufacturer's instructions)
 - D Soak in 70% isopropyl alcohol for 5 minutes. Rinse well.
 - E Soak in 3% hydrogen peroxide for 30 minutes. Rinse well.
- Allow to air dry thoroughly before reassembly.

CAUTIONS

- The manometer gauge cannot be cleaned or disinfected. It is a sealed unit.
- Do not soak, rinse, or sterilize the manometer gauge as these procedures may leave harmful residues or cause malfunction or inaccuracy of the device. The design and material used are not compatible with conventional cleaning and sterilization procedures.

NOTES

- Product should be replaced after 6 months of use or immediately if damaged.
- This product has not been made (or manufactured) with BPA (Bisphenol A) or natural rubber latex.
- Store the manometer with your **Aerobika*** OPEP device in a dry, dust-free place avoiding continuous exposure to sunlight.

DISPOSAL

The manometer (adapter and gauge) can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

TECHNICAL INFORMATION

Storage Conditions	-40° to 140°F (-40° to 60°C), ambient to 75% Relative Humidity (RH)
Operating Conditions	+59° to 113°F (+15° to 45°C), ambient to 85% RH
Measurement Error	0 - 30 cm H ₂ O = ± 2 cm H ₂ O 30 - 40 cm H ₂ O = ± 3 cm H ₂ O 40 - 60 cm H ₂ O = ± 5 cm H ₂ O

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