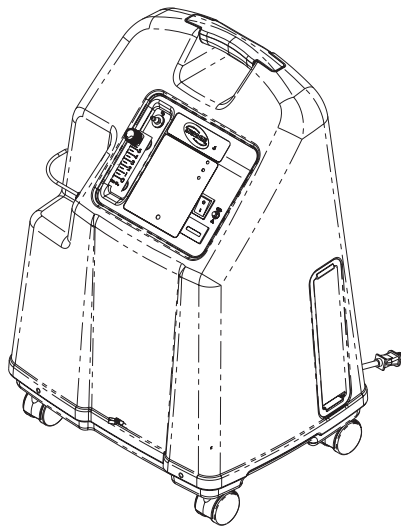


# **Platinum™ Oxygen Concentrator**

The logo for HomeFill II, featuring the letters 'HFII' in a bold, sans-serif font. The 'H' and 'F' are connected at the top, and the 'I' and 'I' are also connected. Above the 'F' and 'I' are several small dots arranged in a semi-circle.

***Model IRC5LXAW (HomeFill™ II Compatible)  
Model IRC5LXO2AW (With SensO2® and HomeFill II  
Compatible)  
Model IRC5LXO2AWQ (With SensO2 and HomeFill II  
Compatible)  
Model IRC9LXO2AWQ (With SensO2 and HomeFill II  
Compatible)***

Dealer: This manual **MUST** be given to the user of the product.  
User: **BEFORE** using this product, read this manual and save for  
future reference.

For more information regarding  
Invacare products, parts, and services,  
please visit [www.invacare.com](http://www.invacare.com)



**Yes, you can.®**

---

---

**⚠ WARNING**

**DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.**

---

**⚠ ACCESSORIES WARNING**

**INVACARE PRODUCTS ARE SPECIFICALLY DESIGNED AND MANUFACTURED FOR USE IN CONJUNCTION WITH INVACARE ACCESSORIES. ACCESSORIES DESIGNED BY OTHER MANUFACTURERS HAVE NOT BEEN TESTED BY INVACARE AND ARE NOT RECOMMENDED FOR USE WITH INVACARE PRODUCTS.**

---

*NOTE: Updated versions of this manual are available on [www.invacare.com](http://www.invacare.com).*

# TABLE OF CONTENTS

<b>SPECIAL NOTES .....</b>	<b>6</b>
<b>TYPICAL PRODUCT PARAMETERS .....</b>	<b>7</b>
<b>FEATURES .....</b>	<b>10</b>
<b>SHIPPING AND HANDLING .....</b>	<b>11</b>
Unpacking.....	11
Inspection .....	11
Storage .....	11
<b>SECTION 1 — IMPORTANT SAFEGUARDS .....</b>	<b>12</b>
Accessories Information .....	12
General Warnings .....	12
Maintenance .....	13
Radio Frequency Interference .....	13
To Reduce the Risk of Burns, Electrocutation, Fire or Injury to Persons.....	13
<b>SECTION 2 — INSTALLATION/SEQUENCE OF OPERATION .....</b>	<b>14</b>
Installation/Verification of Battery Free Power Loss Alarm.....	14
Sequence of Operation .....	14
SensO <sub>2</sub> Oxygen Sensor Technology - Ceramic Zirconia Sensor .....	15
Technical Description .....	15
Operating Sequence .....	15
<b>SECTION 3 — PNEUMATIC DIAGRAM .....</b>	<b>17</b>
<b>SECTION 4 — TROUBLESHOOTING .....</b>	<b>18</b>
<b>SECTION 5 — CABINET .....</b>	<b>26</b>
Removing Cabinet .....	26
<b>SECTION 6 — PREVENTIVE MAINTENANCE .....</b>	<b>27</b>
Cleaning the Cabinet Filter .....	27
Replacing the Outlet HEPA Filter.....	28
Check the Outlet HEPA Filter .....	28
Replace the Outlet HEPA Filter .....	28
Replace the Outlet HEPA Filter .....	29
Replacing the Compressor Inlet Filter.....	30
Replacing Exhaust Canister/Exhaust Muffler (Platinum 5 Manufactured before 7/24/03).....	31
Inspecting/Replacing Throttling Muffler (Platinum 5 AW Manufactured after 7/24/03) .....	32
Inspecting/Replacing the Throttle Muffler (Platinum 5 and 9 AWQ).....	33
Cleaning the Heat Exchanger.....	34

# TABLE OF CONTENTS

<b>SECTION 7 — COMPRESSOR .....</b>	<b>37</b>
Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003.....	37
Replacing Compressor Assembly - 5 Liter Models manufactured after July 24, 2003 .....	38
Replacing Compressor Assembly - 9 Liter Models .....	39
Replacing Capacitor .....	40
Rebuilding the Thomas Model 2650 and 2660 Compressor.....	41
Removing Compressor and Compressor Heads .....	42
Rebuilding the Compressor.....	46
<b>SECTION 8 — P.E. VALVE .....</b>	<b>50</b>
Replacing P.E. Valve.....	50
<b>SECTION 9 — SIEVE BEDS .....</b>	<b>52</b>
Replacing Sieve Beds.....	52
<b>SECTION 10 — CHECK VALVES .....</b>	<b>55</b>
Replacing Check Valves.....	55
<b>SECTION 11 — REGULATOR .....</b>	<b>57</b>
Replacing Regulator.....	57
Adjusting Regulator.....	58
<b>SECTION 12 — HEAT EXCHANGER .....</b>	<b>60</b>
Replacing Heat Exchanger .....	60
Platinum 5 Models Manufactured Before 9/9/2005.....	60
Platinum 5 Models Manufactured After 9/9/2005.....	61
Platinum 9 Concentrators.....	62
<b>SECTION 13 — CONTROL PANEL .....</b>	<b>64</b>
Removing Control Panel.....	64
<b>SECTION 14 — COOLING FAN .....</b>	<b>66</b>
Replacing Cooling Fan .....	66
<b>SECTION 15 — P.C. BOARD .....</b>	<b>68</b>
Replacing P.C. Board.....	68
<b>SECTION 16 — TRANSFORMER .....</b>	<b>70</b>
Replacing Transformer Assembly.....	70
<b>SECTION 17 — ON/OFF SWITCH .....</b>	<b>71</b>
Replacing On/Off Switch.....	71
<b>SECTION 18 — FLOWMETER .....</b>	<b>73</b>
Replacing Flowmeter .....	73

# TABLE OF CONTENTS

<b>SECTION 19 — HOUR METER .....</b>	<b>75</b>
Replacing Hour Meter .....	75
<b>SECTION 20 — 4-WAY VALVE .....</b>	<b>77</b>
Replacing 4-Way Valve.....	77
Replacing Manifold Assembly.....	79
5 Liter Models Manufactured before 7/24/03.....	79
5 Liter Models Manufactured after 7/24/03 and 9 Liter Models.....	82
Replacing Pilot Valve Poppets and O-Rings.....	84
<b>SECTION 21 — PURITY INDICATORS .....</b>	<b>87</b>
SensO <sub>2</sub> Alarm Indicators .....	87
Operation.....	87
Indicator Light Explanation (SensO <sub>2</sub> Models).....	87
Indicator Light Explanation (LX Models) .....	88
<b>SECTION 22 — TIMING .....</b>	<b>89</b>
Autotuning.....	89
Manual Tuning.....	91
<b>SECTION 23 — LEAK TEST .....</b>	<b>93</b>
Leak Test.....	93
<b>SECTION 24 — ALARM TEST .....</b>	<b>96</b>
Alarm and Sensor Tests.....	96
Power Loss Alarm .....	96
Low Pressure Test.....	96
Time-Out Test.....	97
High Pressure Test .....	97
Oxygen Sensor .....	98
P.E. Valve Coil Test .....	99
<b>SECTION 25 — SIEVE BED PRESSURE CHECK .....</b>	<b>101</b>
Models without HomeFill Connection On Back Of Unit .....	101
Installing the Sieve Bed Pressure Tap-In Kit and Checking Bed Pressure .....	101
Pressure Testing.....	102
After Testing .....	103
Models With HomeFill Connection On Back Of Unit. ....	104
Pressure Testing.....	104
<b>LIMITED WARRANTY .....</b>	<b>107</b>

# SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the table below for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage.

---

## NOTICE

The information contained in this document is subject to change without notice.

---

### **DANGER**

**DO NOT SMOKE** while using this device. Keep all matches, lit cigarettes or other sources of ignition out of the room in which this product is located and away from where oxygen is being delivered.

**NO SMOKING** signs should be prominently displayed. Textiles and other materials that normally would not burn are easily ignited and burn with great intensity in oxygen enriched air. Failure to observe this warning can result in severe fire, property damage and cause physical injury or death.

---

# TYPICAL PRODUCT PARAMETERS



Electrical Requirements:	230 VAC $\pm$ 10% (253 VAC/207 VAC), 50 Hz
Rated Current Input:	1.4 A (IRC5LXO2AWQ)
	2.0 A (IRC5LXAW, IRC5LXO2AW)
	2.5 A (IRC9LXO2AWQ)
Sound Level:	39.5 dB Average (IRC5LXO2AWQ)
	44 dB Average (IRC5LXAW, IRC5LXO2AW)
	50 dB Average (IRC9LXAWQ)
Altitude:	For IRC5LXAW, IRC5LXO2AW, IRC5LXO2AWQ - Up to 1828 meters (6,000 ft.) above sea level without degradation of concentration levels. NOT RECOMMENDED FOR USE ABOVE 1828 M (6,000 ft)
	For IRC9LXO2AWQ - Up to 4,000 ft. (1230 meters) above sea level without degradation of concentration levels. For 4,000 ft (1230 meters) to 13,129 ft (4000 meters) below 90% efficiency.
*Oxygen Output Concentration Levels: All 5LXO2AWQ/5LXAW/5LXO2AW models * (Stated concentration levels achieved after initial warm-up period (approximately 30 minutes).	For IRC5LXAW, IRC5LXO2AW, IRC5LXO2AWQ 93% minimum at 1 to 3 L/min. 91% minimum at 4 L/min. 87% minimum at 5 L/min (maximum recommended flow)
	For IRC9LXO2AWQ 93% minimum at 1 to 7 L/min; 91% minimum at 8 L/min; 87% minimum at 9 L/min;
Maximum Outlet Pressure:	5 Liter is 34.5 $\pm$ 3.45 kPa (5 $\pm$ 0.5 psi); 9 Liter is 62.05 kPa $\pm$ 3.45 kPa (9 $\pm$ 0.5 psi)
Flow Range:	For IRC5LXAW, IRC5LXO2AW, IRC5LXO2AWQ - 0.5 to 5 L/min. Flowrates less than 1 L/min. are not recommend
	For IRC9LXO2AWQ - 1 L/min. to 9 L/min. Flowrates less than 1 L/min. are not recommended.
Low Flow Alarm:	For IRC5LXAW, IRC5LXO2AW, IRC5LXO2AWQ - 0 L/min. to 0.5 L/min. Rapid Audible Alarm Beeping (No Accessories Connected).
	For IRC9LXO2AWQ - 0 L/min. to 1.0 L/min. Rapid Audible Alarm Beeping (No Accessories Connected).
Average Power Consumption:	320 W (IRC5LXO2AWQ)
	395 W (IRC5LXO2AW, IRC5LXAW)
	500 W (IRC9LXO2AWQ)
Pressure Relief Mechanism Operational at:	241 kPa $\pm$ 24.1 kPa (35 psi $\pm$ 3.5 psi)
Change in maximum recommended flow when back pressure of 7k Pa is applied:	0.7 L/min.
Filters:	Cabinet (2), Outlet HEPA and Compressor Inlet.

## TYPICAL PRODUCT PARAMETERS


Safety System:	Current overload or line surge shutdown High temperature compressor shutdown High Pressure Alarm w/compressor shutdown Low Pressure Alarm w/compressor shutdown Battery Free Power Loss Alarm SensO <sub>2</sub> Oxygen System (SensO <sub>2</sub> Model) Low Flow Alarm
Width:	46.7 cm ± 1 cm (18 3/8 in ± 3/8 in)
Height:	67.0 cm ± 1 cm (26 3/8 in. ± 3/8 in)
Depth:	36.5 cm ± 1 cm (14 3/8 in. ± 3/8 in)
Weight:	For IRC5LXAW, IRC5LXO2AW, IRC5LXO2AWQ - 23.6 kg ± 1 kg (52 lbs ± 2 lbs)
	For IRC9LXO2AWQ - 24.0 kg ± 1 kg (53 lbs ± 2 lbs)
Shipping Weight:	For IRC5LXAW, IRC5LXO2AW, IRC5LXO2AWQ - 26.8 kg ± 1 kg (59 lbs ± 2 lbs)
	For IRC9LXO2AWQ - 27.2 kg ± 1 kg (60 lbs ± 2 lbs)
Operating Ambient Temperature:	10°C - 35°C (50°F - 95°F) at 20-60% relative humidity
Exhaust:	Less than Ambient +19°C (+ 35°F)
Oxygen Output:	Less than Ambient (+ 4°F) +2°C.
Cabinet:	Impact Resistant flame-retardant plastic cabinet that conforms to UL 94-V0
Regulatory Listing:	5LXAW/5LXO2AW/5LXO2AWQ ETL certified complying with: EN550011 EN61000-3-2 EN61000-3-3 IEC 60601-1, A1, A2 IEC 60601-1-2  5LXO2AW/5LXO2AWQ/IRC9LXO2AWQ Models Only ISO8359 MDD
CE marked models:	9LXO2AWQ, 5LXO2AW, 5LXO2AWQ
Electrical:	No extension cords.
Placement:	No closer than 7.5 cm (3 inches) from any wall, furniture, draperies, or similar surfaces
Tubing:	For IRC5LXAW, IRC5LXO2AW, IRC5LXO2AWQ - 7 ft cannula with a maximum 50 ft of Crush-Proof Tubing (DO NOT pinch)
	For IRC9LXO2AWQ - Recommended use up to 50 ft high flow tubing with high flow cannula at all flow rates
Relative Humidity:	20 to 60%
Time of Operation:	Up to 24 hours per day
Recommended Storage and Shipping Temperature:	-29°C to 65°C (-20°F to 150°F) at 15-95% relative humidity

### SENSO<sub>2</sub> INDICATORS




LABEL SYMBOL	STATUS	INDICATOR LIGHTS (LED)
O <sub>2</sub>	SYSTEM OKAY O <sub>2</sub> over 85% (± 2%)	GREEN Indicator Light
	O <sub>2</sub> Between 73% (± 3%) to 85% (±2%)	YELLOW Indicator light A. YELLOW Solid B. YELLOW Flashing Sensor Failure Call a qualified technician.
	SYSTEM FAILURE O <sub>2</sub> Below 73% (±3%)	RED Indicator Light Continuous Audible Sieve-Gard™ Compressor shutdown Call a qualified technician.

**LX INDICATORS**

LABEL SYMBOL	STATUS	INDICATOR LIGHTS (LED)
I/O	SYSTEM OKAY	GREEN Indicator Light
	SYSTEM FAILURE	RED Indicator light Continuous Audible Alarm Sieve-GARD™ Compressor Shutdown Call a qualified technician

# FEATURES

## FRONT VIEW

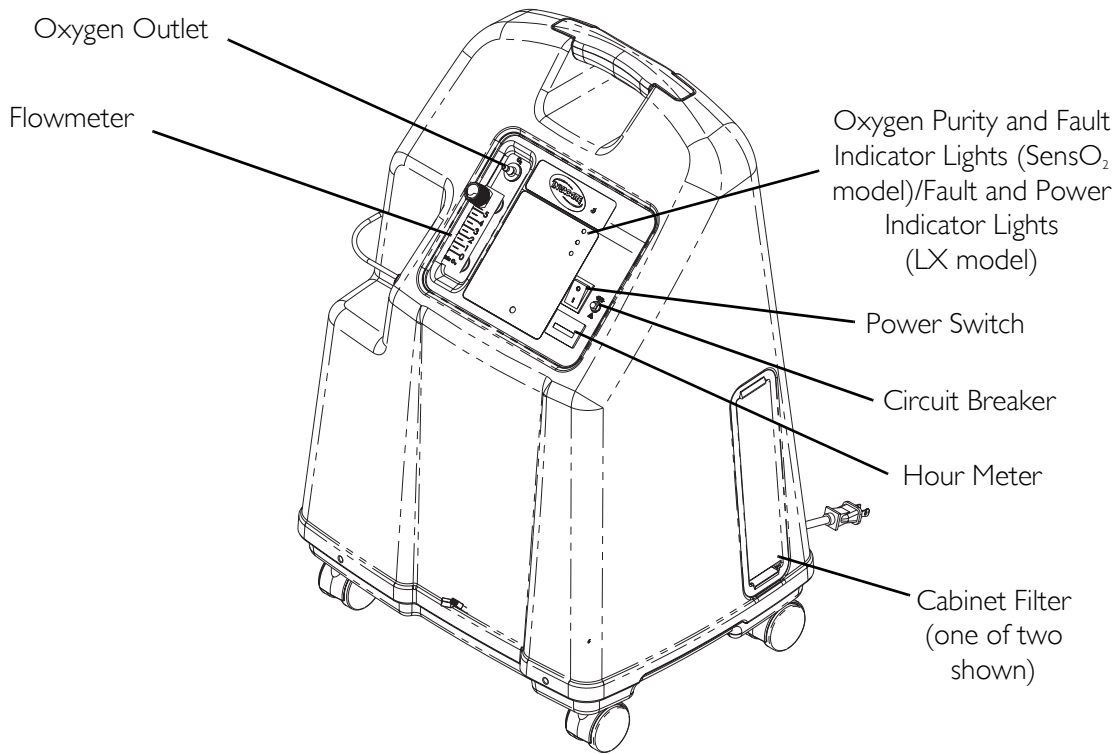


**FIGURE 4.1** IRC5LXAW, IRC5LXO2AW, IRC5LXO2AWQ, IRC9LXO2AWQ

### ACCESSORIES (NOT SHOWN):

Homefill II home oxygen compressor - IOH200AW

Contact your local Invacare representative for available accessory information.

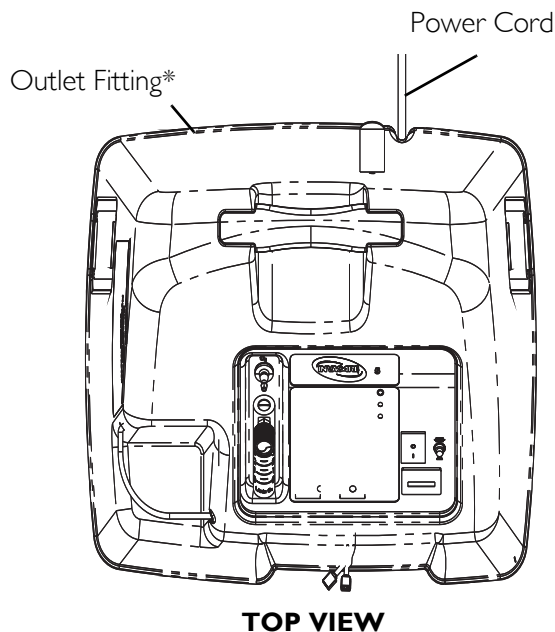

*\*NOTE: This outlet fitting is to be used only for filling oxygen cylinders with the HomeFill II home oxygen compressor. The outlet fitting does not affect concentrator performance. Refer to the HomeFill II owner's manual, part number 1116444, for connection and operating instructions. When not in use, the plug provided with the concentrator should be inserted into the outlet fitting. For more information about the HomeFill II, contact Invacare.*

# SHIPPING AND HANDLING

Before you install and operate the concentrator, please refer to the Platinum Owner's Manual (1118389).

The concentrator should ALWAYS be kept in the upright position to prevent cabinet damage while being transported. The shipping container has been designed to assure maximum protection of the concentrator.

If the concentrator is to be reshipped by common carrier, additional cartons are available from Invacare.

The air compressor suspension system has been engineered to withstand severe motion and orientation.

## Unpacking

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the carrier, or your local dealer.
2. Remove all loose packing from the carton.
3. Carefully remove all the components from the carton.


*NOTE: Unless the oxygen concentrator is to be used IMMEDIATELY, retain containers and packing materials for storage until use of the concentrator is required.*

## Inspection

1. Inspect/examine exterior of the oxygen concentrator for nicks, dents, scratches or other damages. Inspect all components.

## Storage

1. Store the repackaged oxygen concentrator in a dry area.
2. DO NOT place other objects on top of the repackaged concentrator.

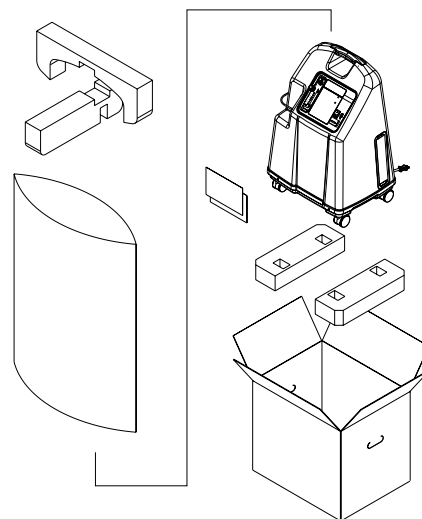

**FIGURE 5.1** Unpacking

# SECTION I—IMPORTANT SAFEGUARDS

---

## **⚠ WARNING**

**IMPORTANT SAFEGUARDS** contains important information for the safe operation and use of this product.

---

## **Accessories Information**

There are many different types of humidifiers, oxygen tubing, cannulas and masks that can be used with this device. You should contact your local home care provider for recommendations on which of these devices will be best for you. They should also give you advice on the proper usage, maintenance, and cleaning.

Use of some administration accessories or certain humidifiers, not specified for use with oxygen concentrator, may impact the performance.

## **General Warnings**

The use of oxygen therapy requires that special care be taken to reduce the risk of fire. Any materials that will burn in air, and some that will not, are easily ignited and burn rapidly in high concentrations of oxygen. For safety concerns, keep all sources of ignition out of the room in which this product is located and away from areas where oxygen is being delivered. Textiles, oil and other combustibles are easily ignited and will burn with great intensity in oxygen-enriched air. NO SMOKING signs should be prominently displayed.

A spontaneous and violent ignition may occur if oil, grease or greasy substances come in contact with oxygen under pressure. These substances **MUST** be kept away from the oxygen concentrator, tubing and connections, and all other oxygen equipment. **DO NOT** use any lubricants unless recommended by Invacare.

For optimum performance, Invacare recommends that each concentrator be on and running for a minimum of 30 minutes at a time. Shorter periods of operation may be harmful for maximum product life.

If the concentrator has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water, call Qualified Technician for examination and repair.

Keep the oxygen tubing, cord, and unit away from heated or hot surfaces, including space heaters, blankets, stoves and similar electrical appliances.

**DO NOT** move or relocate concentrator by pulling on the power cord.

**NEVER** drop or insert any object into any opening.

---

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air opening may be blocked. Keep the openings free from lint, hair and the like.

Fill humidifier with water to the level shown by the manufacturer. DO NOT overfill.

Invacare recommends that Crush-Proof oxygen tubing be used with this product and NOT exceed 15.2 m (50 ft) in length.

## **Maintenance**

The Platinum Oxygen Concentrator was specifically designed to minimize routine preventive maintenance. Only professionals of the healthcare field or persons fully conversant with this process, such as authorized or factory trained personnel, should perform preventive maintenance or performance adjustments on the oxygen concentrator.

## **Radio Frequency Interference**

Most electronic equipment is influenced by Radio Frequency Interference (RFI). CAUTION should be exercised with regard to the use of portable communications equipment in the area around such equipment.

## **To Reduce the Risk of Burns, Electrocution, Fire or Injury to Persons**

DO NOT come in contact with the concentrator while you are wet.

DO NOT place or store product where it can drop into water or other liquids.

DO NOT reach for product that has fallen into water. Unplug IMMEDIATELY.

This device is to be used only in accordance with the prescription of a physician and the Platinum 5 Owner's Manual. If at any time the patient or attendant concludes that the patient is receiving an insufficient amount of oxygen, the supplier and/or physician should be contacted IMMEDIATELY. No adjustments should be made to the flowrate unless prescribed by a physician therapist.

The product should NEVER be left unattended when it is plugged in.

Close supervision is necessary when this product is used by, on or near children or physically challenged individuals.

The Platinum concentrator is intended to be used only as an oxygen supplement as described in the Owner's Manual.

DO NOT connect the concentrator in parallel or series with other oxygen concentrators or oxygen therapy devices.

Avoid creation of any spark near medical oxygen equipment. This includes sparks from static electricity created by any type of friction.

# SECTION 2—INSTALLATION/ SEQUENCE OF OPERATION

## Installation/Verification of Battery Free Power Loss Alarm

When your new Platinum concentrator arrives, it should be checked for proper operating conditions.

1. If the unit has been in below-freezing temperatures, allow it to warm up to room temperature before operating.
2. The concentrator may need to be turned on for 4 to 5 seconds to charge the Battery Free Power Loss Alarm. Connect power cord to outlet and turn the concentrator on. Turn flow control knob counterclockwise and flow will begin IMMEDIATELY. Set flow rate to five L/min. Let the unit run for 30 minutes, then turn unit off.
3. Unplug the power cord and press on/off (I/O) switch to the On (I) position. An intermittent audible alarm will sound. This confirms proper operation of the Battery Free Power Loss Alarm. Turn On/Off (I/O) switch Off (O).
4. Connect power cord to outlet and turn on concentrator. Unit will beep on start-up.
5. Check the oxygen concentration per specifications after 30-40 minutes running time.

## Sequence of Operation

Turning on (I) the power switch applies 230 VAC to the compressor motor, hour meter, transformer, cooling fan and the printed circuit (P.C.) board.

Room air enters the compressor via the cabinet filter and the compressor inlet filter. The air is compressed by the wobble pistons in the compressor to a pressure of 144.79 kPa (21 psi) for 5 Liter model and 172.36 kPa (25 psi) for 9 Liter model.

As increased pressure creates increased temperature, a heat exchanger is used to lower the temperature before the air enters the 4-way valve. It is then channeled to a sieve bed containing the adsorption material. Restriction downstream of the sieve bed causes pressure to build up inside the sieve bed which is necessary for the adsorption process. A small amount of relatively pure oxygen enters the top of the second bed through a restrictive leak in the pressure equalization (P.E.) valve with the balance entering a storage tank. The nitrogen removed is exhausted back from the bed through the 4-way valve into room air. A muffler is located at the exhaust end of the valve to muffle the sound of the exhaust as it exits the concentrator.

The oxygen not being used to exhaust is channeled into the storage tank. The pressurized oxygen is regulated down to  $34.5 \pm 3.45$  kPa ( $5 \pm 0.5$  psi) for 5 Liter;  $62.05 \pm 3.45$  kPa ( $9 \pm 0.5$  psi) for 9 Liter, enters an accurate flow-measuring device, flows through Outlet HEPA filter and check valve, then out to the patient.

The electrical activation of the 4-way Valve is accomplished every 8 to 15 seconds by the pressure sensor and P.C. Board electronics when the pressure reaches a set point of 144.79 kPa (21 psi) for 5 Liter and 172.36 kPa (25 psi) for 9 Liter. The time between cycles is dependent on altitude, flow rate and internal environmental factors.

A P.E. valve opens just prior to the shift of the 4-way valve. This allows highly concentrated oxygen to enter the just exhausted bed from the top. This additional pressure allows the bed to start its cycle at a higher pressure. The P.E. valve will close just after the shift of the 4-way valve.

If main power is lost, the Battery Free Power Loss Alarm will sound a short BEEP, with a long pause after. All units are equipped with a diagnostic alarm system that signals if the pneumatic pressure or electrical systems malfunction. The Troubleshooting Guide in this manual explains the alarm system signals and reasons, in detail, for your convenience.

## **SensO<sub>2</sub> Oxygen Sensor Technology - Ceramic Zirconia Sensor**

### **Technical Description**

The oxygen being produced by the concentrator flows out of the product tank and into the flowmeter. A small flow of oxygen produced by the unit is sent through a precision orifice to the oxygen sensor mounted on the printed circuit board.

As the oxygen enters the sensor, it passes through a screen and contacts the sensing disk.

Electric current flowing through a metal film resistor heats the disk in excess of 300°C. Oxygen molecules contact the electrode of the disk and pick-up extra electrons to become oxygen ions. These oxygen ions are attracted to the electrode on the bottom of the zirconia sensing disk. Because of the crystal structure of the zirconia, only oxygen ions can pass through. When the oxygen ions reach the bottom electrode, the extra electrons are released from the oxygen ions and oxygen molecules return to the air. The number of electrons is directly related to the oxygen concentration. The electrons travel to the P.C. board where they are counted and the oxygen concentration reading is calculated.

A microprocessor on the P.C. board contains software that interprets the signal being received from the sensor. It compares the signal to clinically acceptable limits. Signals outside of the clinically acceptable limits generate responses in the form of lights, audible indicators, and/or system shut-down.

### **Operating Sequence**

Once the power switch has been turned on (I), the SensO<sub>2</sub> circuit will wait five minutes for the concentrator to begin producing clinically acceptable oxygen and the oxygen sensor to stabilize. The GREEN light will illuminate (indicating normal system operation) while the oxygen sensor is warming up.

After 5 minutes, if the oxygen purity exceeds 85% ± 2%, the GREEN light will continue to illuminate.

If the oxygen level is not above  $85\% \pm 2\%$  after the first five minutes, the system will continue to monitor the  $O_2$  and wait for a maximum of 30 minutes from start-up to reach  $85\% \pm 2\%$  before activating an alarm. Environmental factors such as low voltage, high altitude, or age of the machine will affect the time required to reach  $85\% \pm 2\%$ .


If the oxygen level is not above  $85\% \pm 2\%$  within the first 30 minutes, the oxygen concentration alarm sequence will activate and the unit will shut down.

When oxygen concentration is above  $85\% \pm 2\%$ , the sensor measures oxygen purity every 10 minutes. If a reading falls below  $85\% \pm 2\%$ , a YELLOW light will illuminate. If the oxygen purity falls below  $73\% \pm 3\%$ , the RED light/Alarm/Shut-Down mode will activate.



# SECTION 3—PNEUMATIC DIAGRAM

**Platinum Oxygen Concentrator**

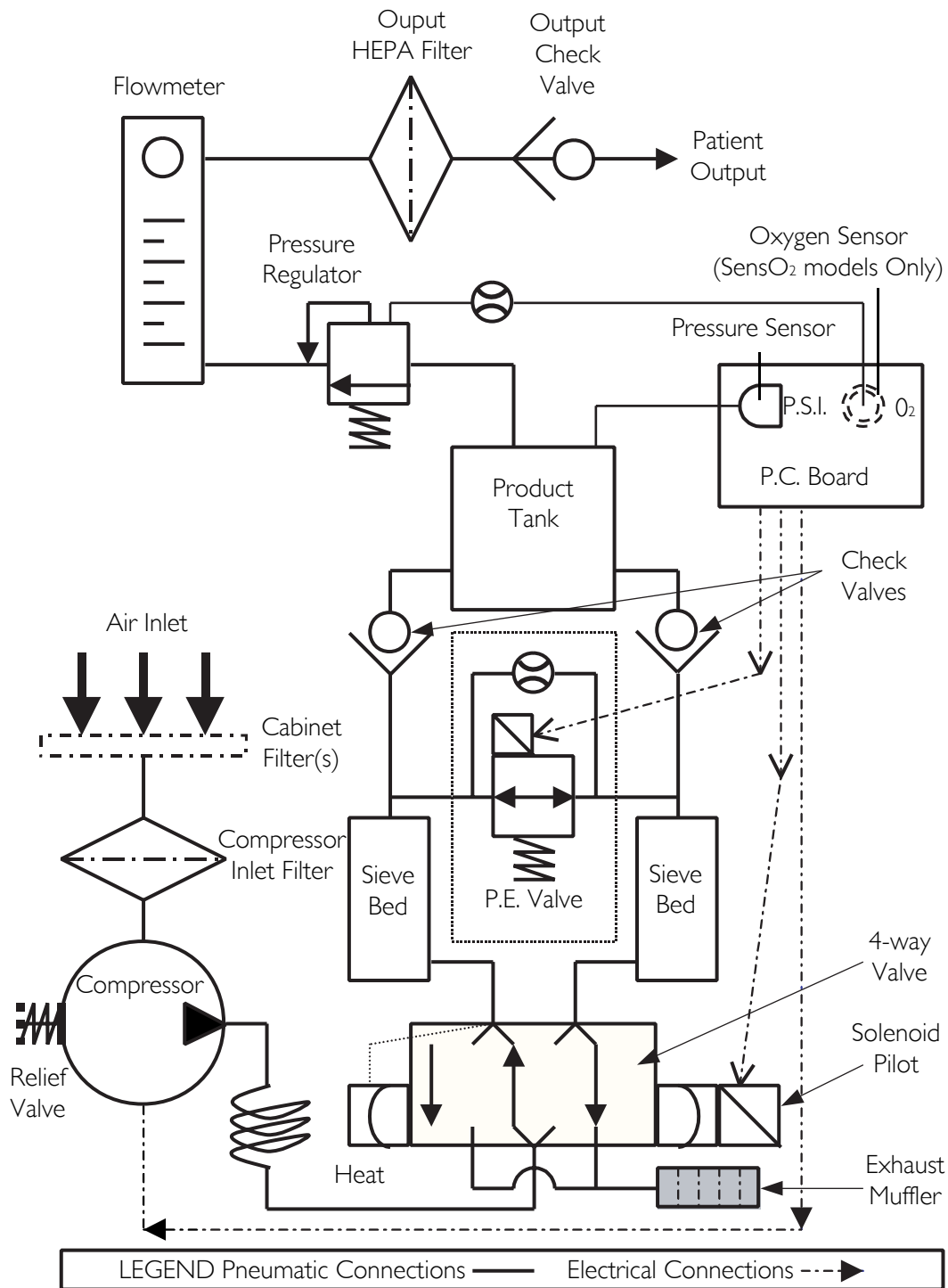

# SECTION 4—TROUBLESHOOTING

SYMPTOM	PROBABLE CAUSE	SOLUTION
Normal Operation Internal Status Indicators: RED: Off GREEN: Off Unit plugged in, power switch on. Single beep on start up.	No Problems	System Okay.
Power Loss Internal Status Indicators: RED: Off GREEN: Off Unit unplugged, power switch on, alarm off.	No Problems.	Battery Free Circuit drained. Plug in cord and turn power switch ON (I) to recharge.
Power Loss Internal Status Indicators: RED: Off GREEN: Off Unit plugged in, power switch, alarm off, battery Free circuit drained.	No power at outlet.	Check electrical outlet with a table lamp or voltmeter set on the appropriate VAC scale. If outlet isn't working, check protective device in home's electrical panel or consult an electrician. Also ensure that unit is properly plugged in. DO NOT use extension cords. Move to another outlet or circuit.
	Power cord: a. Frayed. b. Broken or damaged spade. c. Spade connector from power cord loose or disconnected (inside back of unit).	a. Reattach cord. b. Replace power cord connectors on plug. c. Reattach connector.
	On/Off (VO) switch. a. Disconnected wire.  b. Faulty switch.	Check all electrical connections to the On/Off (VO) switch for any disconnected wires.  If the concentrator does not come on at all and wiring is intact, color code and remove wires one at a time. Remove and replace with new one. Transfer wires from old switch to new switch one at a time to the matching contact.
	Circuit breaker tripped.	Reset breaker NOTE: Breaker may trip to safeguard concentrator during a power surge. If breaker trips IMMEDIATELY, there is a probable short in the unit. Check for pinched or charred wires. If the breaker does not trip, run unit for approximately two hours. The circuit breaker should be less than ten ohms. If breaker trips again, there is an internal problem. Contact Invacare Technical Service at 1-800-832-4707.
	P.C. board. a. P.C. board damaged.  b. Loose or damaged connector.	a. Replace P.C. Board. Refer to <u>Replacing P.C. Board</u> on page 68. b. Repair or replace connector. Refer to <u>Replacing P.C. Board</u> on page 68

SYMPTOM	PROBABLE CAUSE	SOLUTION
<p>Internal Power Loss SensO<sub>2</sub> Internal Status Indicators: RED: Off GREEN: Off Alarm may or may not be on. Control Panel Indicators: RED: Off YELLOW: Off GREEN: Off Fan operates, Compressor not operating.</p>	<p>Transformer assembly. a. Faulty b. Connector loose or disconnected. c. Faulty wiring.</p>	<p>a. Replace. Refer to <u>Replacing Transformer Assembly</u> on page 70. b. Reattach connector. c. Replace transformer assembly. Refer to <u>Replacing Transformer Assembly</u> on page 70.</p>
<p>High Pressure Internal Status Indicators:  RED: One Flash GREEN: Two Flashes  Unit plugged in, power switch on, continuous alarm. Compressor shut down.</p>	<p>P.C. board: a. Malfunction.  b. Disconnected wire.  c. Shifting valve at pressures greater than 25 p.s.i.</p>	<p>Replace P.C. board. Unit requires retiming after P.C. board replacement. Refer to <u>Replacing P.C. Board</u> on page 68.  a. Set flow to max L/min. for concentrator. Check voltage across Pilot Valve 2 on 200 volt scale. If meter reads 0 volts when unit is turned on, replace P.C. board b. Check spade connectors on pilot valve 2 and connectors on P.C. board. c. Replace P.C. board. Unit requires retiming after P.C. board replacement. Refer to <u>Replacing P.C. Board</u> on page 68.</p>
	<p>4-way Valve: a. Not shifting.  b. Bad coil.</p>	<p>a. Check voltage. If voltage rises to approximately 24 volts D.C. when unit shuts down, P.C. board is functioning properly. Replace the valve if shorted or open. Refer to <u>Replacing 4-Way Valve</u> on page 77. b. Coil resistance should read 80 ohm ± 10. Replace 4-way valve. Refer to <u>Replacing 4-Way Valve</u> on page 77.</p>
	<p>Sieve Beds contaminated.</p>	<p>a. Replace Sieve Beds.</p>

## SECTION 4—TROUBLESHOOTING

SYMPTOM	PROBABLE CAUSE	SOLUTION
<p>Low Pressure Internal Status Indicators: RED: One Flash GREEN: One Flash Or RED: One Flash GREEN: Three Flashes</p> <p>Control Panel Indicators: RED: On YELLOW: Off GREEN: Off</p> <p>Unit plugged in, power switch on, continuous audible alarm. Compressor shut down (Failure to cycle due to low pressure).</p>	Compressor inlet filter dirty or plugged.	Replace compressor inlet filter. Refer to <a href="#">Replacing the Compressor Inlet Filter</a> on page 30.
	Compressor: a. Leaks at fittings or tubing. b. Leaking or defective relief valve. c. Insufficient voltage at outlet. d. Worn cup seals or gaskets.	<p>a. Tighten fittings. Replace or repair.</p> <p>b. Repair leak or replace.</p> <p>c. DO NOT use extension cords. Use another outlet.</p> <p>d. Rebuild top end of compressor. Refer to <a href="#">Rebuilding the Thomas Model 2650 and 2660 Compressor</a> on page 41.</p>
	Heat exchanger: a. Leak at tubing or body chamber. b. Inspect tubing and heat exchanger.	<p>a. Replace or retighten.</p> <p>b. Replace or retighten tubing. Replace heat exchanger. Refer to <a href="#">Replacing Heat Exchanger</a> on page 60.</p>
	Regulator cracked or leaking.	Replace regulator. Refer to <a href="#">Replacing Regulator</a> on page 57.
	P.E. valve leaking.	<p>Replace P.E. Valve. Check voltage at P.E. valve connector on 24 volt D.C. scale. The P.E. Valve activates or energizes approximately one second prior to the activation of the 4 way valve with approximately 24 volts. If voltage is in excess of 24 volts consistently, replace the P.C. board. Refer to <a href="#">Replacing P.C. Board</a> on page 68. If the P.C. board voltage acts normally, replace the P.E. valve. Refer to <a href="#">Replacing P.E. Valve</a> on page 50.</p> <p>NOTE: Check for leaks starting at the compressor output through all the pneumatic connections. Major leaks will cause system pressures to remain below adequate shift (exhaust) pressures and will cause compressor shutdown.</p>
<p>Defective Parts Internal Status Indicators: RED: One Flash GREEN: Five Flashes Or RED: Two Flashes GREEN: Three Flashes Control Panel Indicators: RED: On YELLOW: Off GREEN: Off</p> <p>Unit plugged in, power switch on, continuous audible alarm. Compressor shut down.</p>	Defective main valve coil.	<p>a. Replace main valve. Refer to <a href="#">Replacing 4-Way Valve</a> on page 77.</p> <p>b. Replace main valve coil. Refer to <a href="#">Replacing 4-Way Valve</a> on page 77</p>
	Connector loose.	Reattach connector.
	Defective P.E. valve coil (Resistance 80 ohm's $\pm$ 10).	Replace P.E. coil if shorted or open on resistance check. Refer to <a href="#">Replacing P.E. Valve</a> on page 50.
	Defective PC Board.	Replace PC Board
	Defective P.E. valve coil. (Resistance 80 ohm's $\pm$ 10.)	Replace P.E. coil if shorted or open on resistance check. Refer to <a href="#">Replacing P.E. Valve</a> on page 50.
	Connector loose.	Reattach connector.
Defective P.C. board.	Replace P.C. board. Refer to <a href="#">Replacing P.C. Board</a> on page 68.	
<p>Unit Not Operating Alarm: On or Off Internal Status Indicators: RED: Two Flashes GREEN: Four Flashes Or RED: Two Flashes GREEN: Five Flashes</p>	P.C. board Failure.	Replace P.C. board. Refer to <a href="#">Replacing P.C. Board</a> on page 68.

SYMPTOM	PROBABLE CAUSE	SOLUTION
<p><u>Low Concentration</u> NOTE: Check for O<sub>2</sub> purity using a calibrated Oxygen Analyzer at Test Point 1 (Oxygen Outlet) of the concentrator.</p> <p>Internal Status Indicators: RED: Two Flashes GREEN: One Flash 73% Shutdown Control Panel Indicators: SensO<sub>2</sub> ONLY: RED: On YELLOW: Off GREEN: Off</p> <p>For SensO<sub>2</sub> units, the RED indicator will signal extremely low purity and will be accompanied by a continuous audible alarm and a system shutdown. Repairs are required.</p>	Cabinet filters dirty.	Clean or replace. Refer to <u>Cleaning the Cabinet Filter</u> on page 27, <u>Replacing the Outlet HEPA Filter</u> on page 28, and <u>Replacing the Compressor Inlet Filter</u> on page 30.
	Compressor inlet filter dirty.	Replace inlet filter. Refer to <u>Replacing the Compressor Inlet Filter</u> on page 30.
	Compressor: a. Defective.	a. Replace compressor. Refer to <u>Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003</u> on page 37.
	b. Faulty capacitor.	b. Replace capacitor. Refer to <u>Replacing Capacitor</u> on page 40.
	c. Bad motor windings.	c. Replace compressor. Refer to <u>Compressor</u> on page 37.
	d. Worn seals.	d. Rebuild top end of compressor. Refer to <u>Rebuilding the Thomas Model 2650 and 2660 Compressor</u> on page 41.
	e. Bad bearings.	e. Replace compressor. Refer to <u>Compressor</u> on page 37.
	f. Leak at fittings or tubing.	f. Replace fittings or tubing.
	g. Leaky or defective relief valve.	g. Replace compressor. Refer to <u>Compressor</u> on page 37.
h. Insufficient voltage (outlet).	h. DO NOT use extension cords.	
Heat exchanger: a. Leak at tubing or body chamber. b. Inspect tubing and heat exchanger.	a. Replace or retighten. b. Replace or retighten tubing. Replace heat exchanger. Refer to <u>Replacing Heat Exchanger</u> on page 60.	
Regulator cracked or leaking.	Replace. Refer to <u>Replacing Regulator</u> on page 57.	
Exhaust/Throttle muffler dirty or plugged.	Replace. Refer to <u>Replacing Exhaust Canister/Exhaust Muffler (Platinum 5 Manufactured before 7/24/03) or Inspecting/Replacing Throttling Muffler (Platinum 5 AW Manufactured after 7/24/03)</u> .	
Fan: a. Not operating. Unit overheating. b. Faulty fan.	a. Leads to fan disconnected. Reconnect. b. Replace. Refer to <u>Replacing Cooling Fan</u> on page 66.	
Sieve beds defective.	Replace. Refer to <u>Models without HomeFill Connection On Back Of Unit</u> on page 101.	
Tubing kinked.	Replace.	

## SECTION 4—TROUBLESHOOTING

SYMPTOM	PROBABLE CAUSE	SOLUTION
<p><u>Low Concentration</u> (Continued)</p>	<p>P.C. board: a. Shifts at wrong pressures.</p>	<p>Check pressure at product tank. Pressure should rise to 144.79 kPa (21 psi) at shift point. If not, replace P.C. Board. Refer to <u>Replacing P.C. Board</u> on page 68.</p>
	<p>Flowmeter: a. Flowmeter opened beyond maximum flow rate. b. Cracked or broken fitting. c. Input tubing leaking or loose.</p>	<p>a. Return flow to maximum setting. b. Replace fitting. c. Repair or replace. Refer to <u>Replacing Flowmeter</u> on page 73.</p>
	<p>Timing.</p>	<p>To accommodate for varying tolerances when replacing components, an adjustable timer is used to control the shifting of the Pressure Equalization (P.E.) valve. Refer to <u>Timing</u> on page 89.</p>
	<p>P.E. valve: a. Bad coil. b. Restrictor blockage.</p>	<p>a. Replace P.E. valve. b. Replace P.E. valve. Refer to <u>Replacing P.E. Valve</u> on page 50.</p>
	<p>Inspect P.C. board restrictor tubing for kinks or tears.</p>	<p>Replace P.C. board. Unit may need retiming after board replacement. Refer to <u>Replacing P.C. Board</u> on page 68.</p>
<p>Fluctuating Flow</p>	<p>Regulator/Flowmeter: a. Incorrectly set regulator. b. Flowmeter malfunction.</p>	<p>a. Check pressure at oxygen outlet. Adjust regulator. b. If flow is still unstable, check for leaks starting at the compressor outlet fitting through all pneumatic connections. If no leaks are found and flow is still fluctuating, replace the regulator. If pressure at test point is within spec <math>34.5 \pm 3.45</math> kPa (<math>5 \pm 0.5</math> psi) for 5 Liter; <math>62.05 \pm 3.45</math> kPa (<math>9 \pm 0.5</math> psi) for 9 Liter, replace flowmeter. Refer to <u>Replacing Flowmeter</u> on page 73.</p>
	<p>Outlet HEPA filter: a. Dirty or plugged</p>	<p>a. If low flow conditions persist, replace outlet HEPA filter. Refer to <u>Replacing the Outlet HEPA Filter</u> on page 28.</p>

SYMPTOM	PROBABLE CAUSE	SOLUTION
Unit Excessively Loud	Pneumatic exhaust: a. Muffler cracked, damaged or missing. b. Muffler tubing disconnected or damaged.	a. Replace. Refer to <u>Replacing Exhaust Canister/Exhaust Muffler (Platinum 5 Manufactured before 7/24/03)</u> on page 31. b. Reconnect or replace tubing. Refer to <u>Inspecting/Replacing Throttling Muffler (Platinum 5 AW Manufactured after 7/24/03)</u> on page 32.
	Compressor inlet filter missing and/or orange sticker removed.	a. Replace compressor inlet filter. Refer to <u>Replacing the Compressor Inlet Filter</u> on page 30.
	Compressor removed.	a. Replace Compressor. Refer to <u>Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003</u> on page 37.
	Incorrect style of inlet filter (aftermarket).	Replace with factory OEM sound reduced style inlet HEPA filter. Refer to <u>Replacing the Outlet HEPA Filter</u> on page 28.

## SECTION 4—TROUBLESHOOTING

SYMPTOM	PROBABLE CAUSE	SOLUTION
Unit Overheats	Base exhaust vent plugged or restricted.	Place unit at least 7.5 cm (3-inches) from any wall. DO NOT place unit on pile or shag carpeting that may restrict air flow.
	Cabinet filters dirty or blocked.	Clean or replace. Refer to <u>Cleaning the Cabinet Filter</u> on page 27.
	Fan: a. Leads to fan disconnected. b. Defective fan.  c. Fan installed upside down.	a. Reconnect leads. b. Replace fan. Refer to <u>Rebuilding the Thomas Model 2650 and 2660 Compressor</u> on page 41. c. Install fan with air flow arrow pointing down.
	Heat exchanger: a. Dirty or plugged. b. Damaged.	a. Clean heat exchanger. b. Replace heat exchanger. Refer to <u>Replacing Heat Exchanger</u> on page 60.
	Compressor: a. Defective.  b. Faulty capacitor.  c. Bad motor windings. d. Worn seals. e. Bad bearings.	a. Replace compressor. Refer to <u>Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003</u> on page 37. b. Replace capacitor. Refer to <u>Replacing Capacitor</u> on page 40. c. Replace compressor. Refer to <u>Compressor</u> on page 37 d. Replace compressor. Refer to <u>Compressor</u> on page 37 e. Replace compressor. Refer to <u>Compressor</u> on page 37
	Line voltage excessive (surge).	Have line voltage inspected by certified electrician. A voltage regulator may be required and is obtainable from your local electric company.
Oxygen Purity Good Internal Status Indicators: Control Panel Indicators: RED: Off YELLOW: On GREEN: On After 30 minutes of run time, unit operates normally, oxygen purity within normal range. GREEN or YELLOW panel indicator should illuminate.	P.C. board defective.	Replace P.C. board. Refer to <u>Replacing P.C. Board</u> on page 68. Unit may need retiming after P.C. board replacement. Refer to <u>Timing</u> on page 89.



SYMPTOM	PROBABLE CAUSE	SOLUTION
Unit Not Operating Internal Status Indicators: RED: Off GREEN: Off Control Panel Indicators: RED: Off YELLOW: Off GREEN: Off Power Switch ON. Continuous audible alarm.	Transformer assembly: a. Assembly connector disconnected. b. Faulty transformer assembly.	a. Reattach connector. b. Replace transformer assembly. Refer to <u>Replacing Transformer Assembly</u> on page 70.
Unit Operating Internal Status Indicators: RED: Three Flashes GREEN: One Flash CONTROL PANEL INDICATORS: RED: Off YELLOW: Flashing GREEN: On	Internal repairs required.	Replace SensO <sub>2</sub> circuit board. Refer to <u>Replacing P.C. Board</u> on page 68.
Low-Flow Alarm does not activate on flows less than 0.5 L/min. NOTE: This feature was added on units manufactured after November 2002. These units can be identified by a single beep on start up.	System leak.	Repair leak in product tank, regulator, tubing, fittings, or flow meter.
	Defective check valves.	Replace check valves. Refer to <u>Replacing Check Valves</u> on page 55.

*NOTE: On models manufactured after 09/03, the error code displayed on the internal status indicators is maintained in memory to aid in repairs. The error code must be cleared from the unit's memory after fixing the fault. The error code memory will be cleared after turning power on and off five times after the fault was detected.*

# SECTION 5—CABINET

## Removing Cabinet

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.


*NOTE: For this procedure, refer to FIGURE 5.1.*

*NOTE: This procedure applies to both Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove the four mounting screws that secure cabinet assembly to the base assembly.
3. Lift the cabinet straight up.

*NOTE: When required, vacuum inside of the cabinet and exposed foam insulation.*

4. To re-install cabinet, reverse STEPS 2-3.

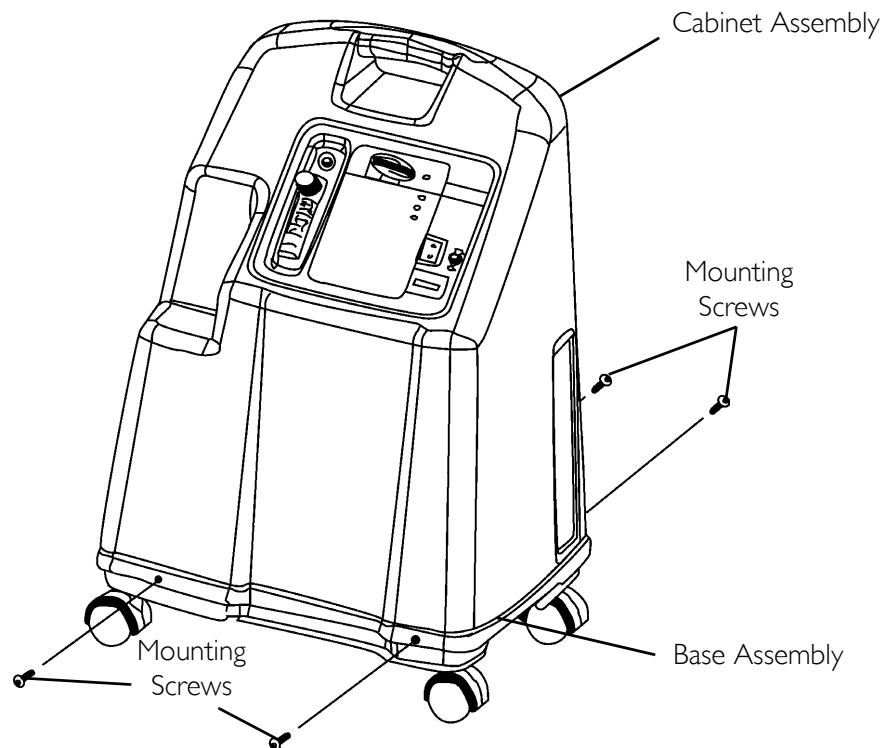

**FIGURE 5.1** Removing Cabinet

# SECTION 6—PREVENTIVE MAINTENANCE

## ⚠ DANGER

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

*NOTE: The Platinum Concentrators are specifically designed to minimize routine preventive maintenance at intervals of once per year. In places with high dust or soot levels, maintenance may need to be performed more often. The following **MUST** be performed at a minimum of one year in service to assure years of additional reliability. Refer to Troubleshooting on page 18 for plugged filter symptoms. Only qualified technicians should perform preventive maintenance on the concentrator. Power **MUST** be disconnected before beginning preventive maintenance on the Invacare concentrator.*

## Cleaning the Cabinet Filter

*NOTE: For this procedure, refer to FIGURE 6.1.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*


*NOTE: Perform this procedure as needed depending upon the environment the concentrator is used in.*

*NOTE: There are two cabinet filters, one located on each side of the cabinet.*

1. Remove each filter and clean at least once a week depending on environmental conditions.

*NOTE: Environmental conditions that may require more frequent cleaning of the filters include but are not limited to: high dust, smoking, air pollutants, etc.*

2. Clean the cabinet filters with a vacuum cleaner or wash in warm soapy water and rinse thoroughly.
3. Dry the filters thoroughly before reinstallation.

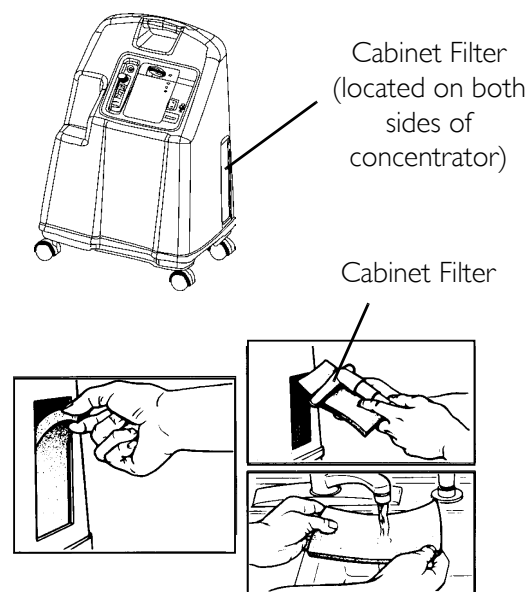

**FIGURE 6.1** Cleaning the Cabinet Filter

## CAUTION

**DO NOT** operate the concentrator without the filters installed.

## Replacing the Outlet HEPA Filter

### Check the Outlet HEPA Filter

*NOTE: For this procedure, refer to FIGURE 6.2 on page 29*

*NOTE: The outlet HEPA filter can be checked by performing the following procedure.*

1. Turn the concentrator on and adjust the flowmeter to the maximum flow of the unit.
2. Observe the flowmeter's flow indicator while connecting a 50 foot (15.2 meters) cannula tube to the outlet barb of the concentrator (not shown).
3. If the flow indicator fluctuates, the outlet HEPA filter may need replacement. Refer to [Troubleshooting](#) on page 18.


*NOTE: To replace the Outlet HEPA filter, perform this procedure as needed depending on the Outlet HEPA filter check results.*

### Replace the Outlet HEPA Filter

*NOTE: For this procedure, refer to FIGURE 6.2 on page 29.*

*NOTE: Perform this procedure as needed depending on the outlet HEPA filter check results.*

1. Unplug the unit.
2. Locate the filter access and humidifier adapter storage plate located on the side of the concentrator.
3. Insert a flathead screwdriver in the plate groove on the top edge of the plate and gently pry the humidifier adapter storage plate open (Detail "A").
4. Remove plate and set aside.
5. Grasp the existing outlet HEPA filter and disconnect from the short silicone tube (Detail "B").
6. Disconnect existing outlet HEPA filter from the long silicone tube (Detail "B").
7. Discard existing outlet HEPA filter.
8. Attach new outlet HEPA filter to short and long silicone tubes.
9. Reinstall filter access and humidifier adapter storage plate.

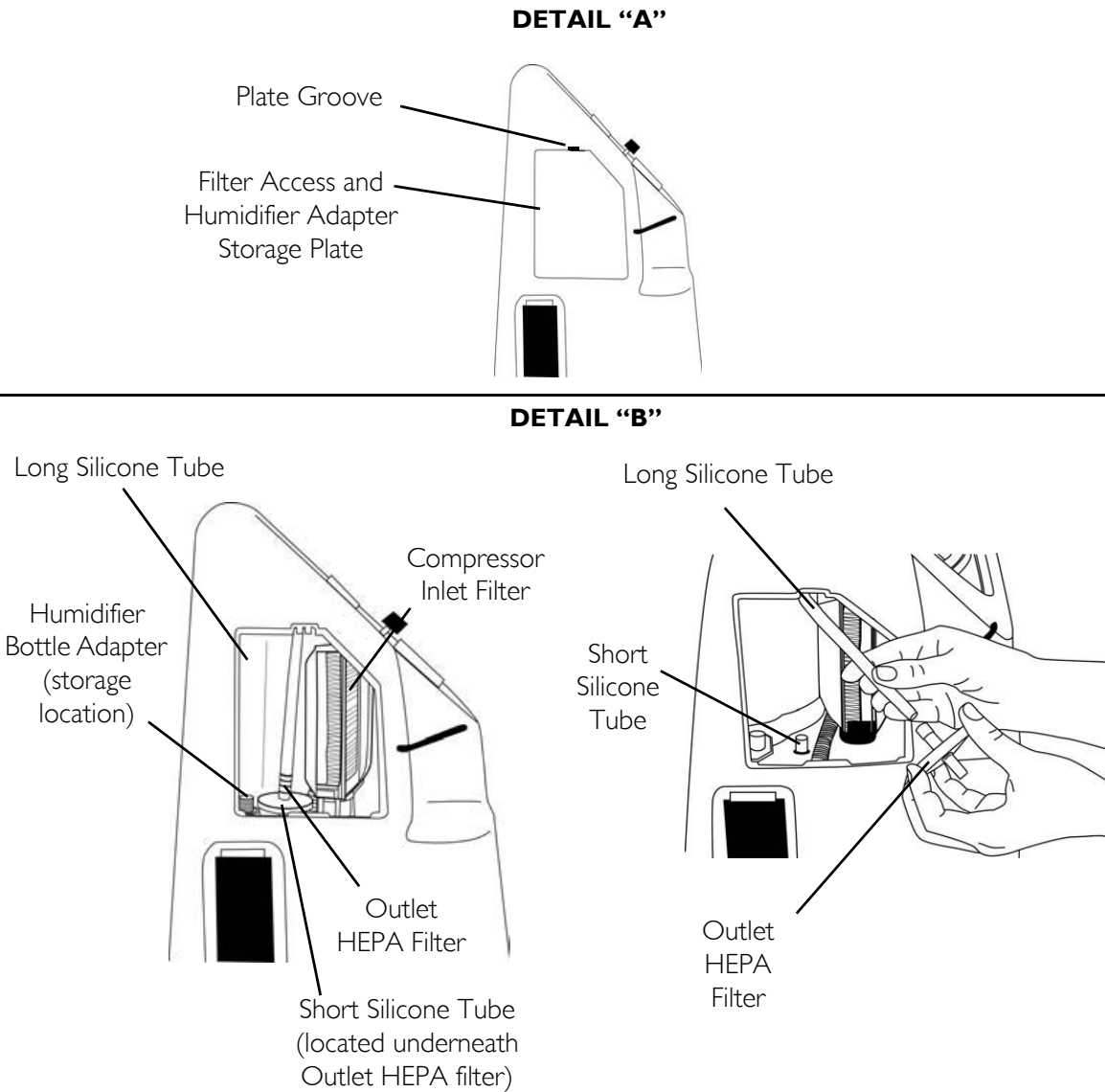


**FIGURE 6.2** Replace the Outlet HEPA Filter

### Replace the Outlet HEPA Filter

*NOTE: For this procedure, refer to FIGURE 6.3 on page 30.*

*NOTE: Perform this procedure as needed depending on the outlet HEPA filter check results.*

1. Unplug the unit.
2. Remove the cabinet. Refer to [Removing Cabinet](#) on page 26.
3. Remove the tubing from both sides of the existing HEPA filter and discard old filter.
4. Connect tubing to both sides of the new HEPA filter as shown below.
5. Install cabinet. Refer to [Removing Cabinet](#) on page 26.

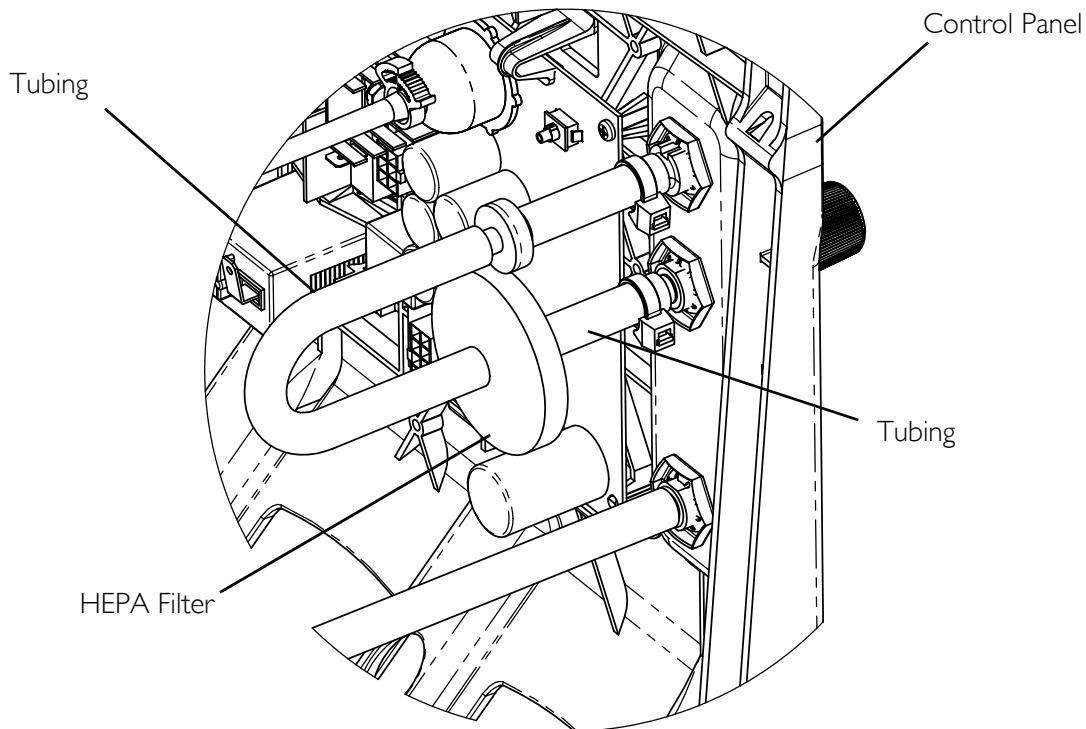

**FIGURE 6.3** Replace the Outlet HEPA Filter

## Replacing the Compressor Inlet Filter


*NOTE: For this procedure, refer to FIGURE 6.2 on page 29 and FIGURE 6.4 on page 31.*

*NOTE: Perform this procedure as needed depending upon the environment the concentrator is used in.*

1. Unplug the unit.
2. Locate the filter access and humidifier adapter storage plate located on the side of the concentrator.
3. Insert a flathead screwdriver in the plate groove on the top edge of the plate and gently pry the humidifier adapter storage plate open to remove the plate. Refer to Detail "A" in FIGURE 6.2.
4. Grasp the compressor inlet filter and pull downward.
5. Discard the existing compressor inlet filter.

*NOTE: Filter, p/n 1131249, is fully compatible with all models. This filter replaces 1108078.*

6. Install new compressor inlet filter by inserting the filter opening into the rubber base (Detail "B" of FIGURE 6.4).
7. Push the filter down until the rubber base touches the edge of the filter.
8. Reinstall the filter access and humidifier adapter storage plate.

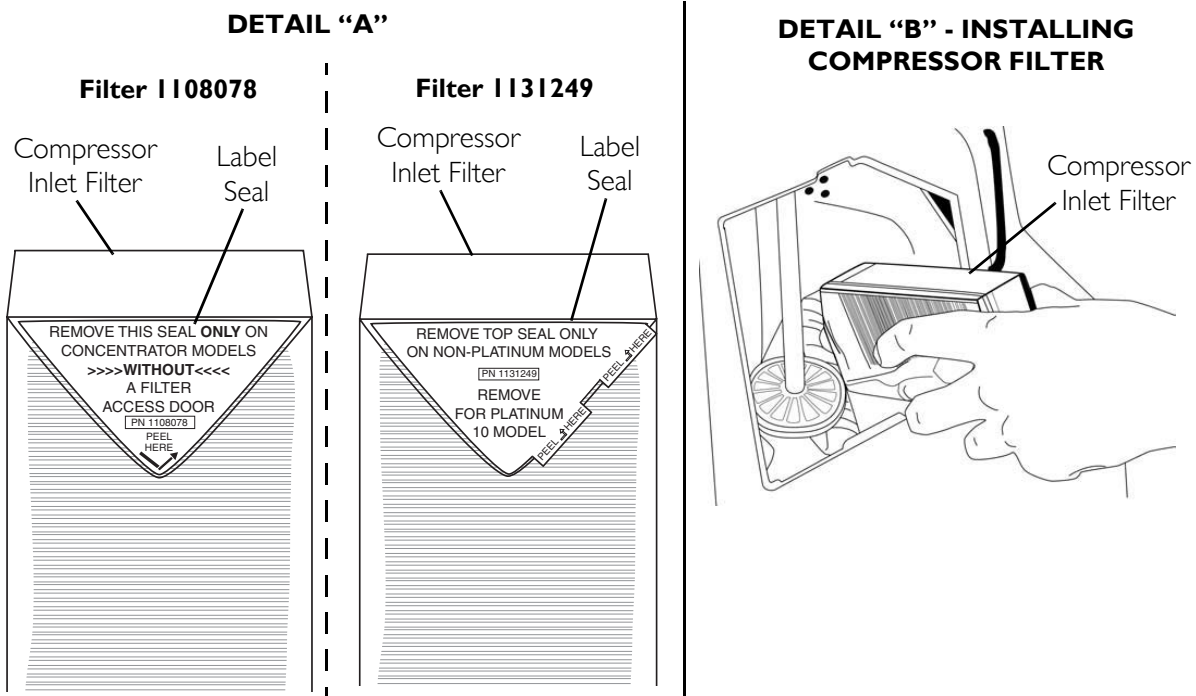


**FIGURE 6.4** Replacing the Compressor Inlet Filter

## Replacing Exhaust Canister/Exhaust Muffler (Platinum 5 Manufactured before 7/24/03)

*NOTE: For this procedure, refer to FIGURE 6.5 on page 32.*

*NOTE: The following should be performed at 20,000 hour intervals although these procedures can be performed as needed depending upon the environment the concentrator is exposed to.*

1. Unplug unit.
2. Remove cabinet. Refer to [Removing Cabinet](#) on page 26.
3. Remove the two mounting screws from rear of sound box.
4. Disconnect and remove the manifold exhaust tubing from the barbed fittings on top of the exhaust canister.
5. Unscrew the exhaust muffler located on the exhaust canister by turning counterclockwise.
6. Inspect exhaust canister to ensure the barbed fittings are clear and do one of the following:
  - Replace exhaust canister and muffler if plugged or restricted.
  - Install only new exhaust muffler if exhaust canister is in good condition.
7. Install new/re-install existing exhaust canister by reversing STEPS 3-5.
8. Reinstall cabinet. Refer to [Removing Cabinet](#) on page 26.

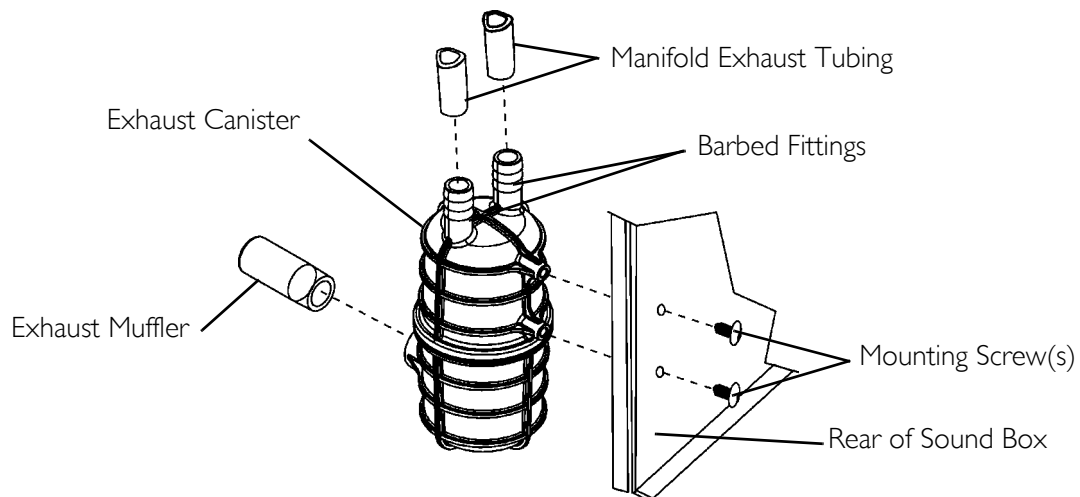

**FIGURE 6.5** Replacing Exhaust Canister/Exhaust Muffler (Platinum 5 Manufactured before 7/24/03)

## Inspecting/Replacing Throttling Muffler (Platinum 5 AW Manufactured after 7/24/03)


*NOTE: For this procedure, refer to FIGURE 6.7 on page 34.*

*NOTE: The following should be performed at 20,000 hour intervals although these procedures can be performed as needed depending upon the environment the concentrator is exposed to.*

1. Unplug unit.
2. Remove cabinet. Refer to [Removing Cabinet](#) on page 26.
3. Remove the Y-tube from the manifold assembly.
4. Very carefully remove the blue tie wrap from the Y-tube. DO NOT sever the Y-tube.
5. Separate the throttle muffler from the Y-tube and inspect the muffler for residue build-up.
6. If excessive buildup, remove the throttle muffler and discard. Replace with a new muffler. If no buildup present, reattach the throttle muffler.
7. Place the new blue tie wrap onto the Y-tube.
8. Insert the new muffler approximately 0.5 in. (1.27 cm) into the Y-tube opening.
9. Adjust the tie wrap so it is resting just above the lip on the Y-tube and tighten.
10. Reattach Y-tube to the manifold assembly.
11. Reinstall cabinet. Refer to [Removing Cabinet](#) on page 26.



## 5 Liter AW Models

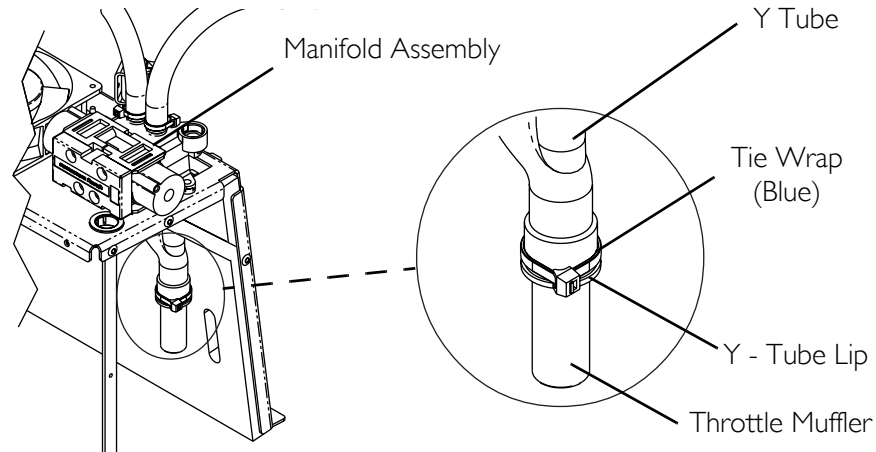


**FIGURE 6.6** Inspecting/Replacing Throttling Muffler (Platinum 5 AW Manufactured after 7/24/03)

## Inspecting/Replacing the Throttle Muffler (Platinum 5 and 9 AWQ)

*NOTE: For this procedure, refer to FIGURE 6.7 on page 34.*

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Remove the throttle muffler from exhaust canister.
4. Cut the two tie wraps that secure the exhaust canister together.
5. Remove the exhaust canister from the Y-tube.
6. Separate the two halves of the exhaust canister (Detail "A").
7. Inspect the throttle muffler located inside the exhaust canister (Detail "A").
8. Perform one of the following:
  - If excessive buildup, remove the throttle muffler and discard. Replace with a new muffler. Proceed to STEP 9.
  - If no buildup present use existing throttle muffler and proceed to STEP 9.
9. Install throttle muffler into Y-tube (Detail "A").
10. Reconnect both halves of the exhaust canister (Detail "A").
11. Install exhaust canister over throttle muffler that is inserted into the Y-tube (Detail "A").
12. Inspect remaining throttle muffler.
13. Perform one of the following:
  - If excessive buildup, remove the throttle muffler and discard. Replace with a new muffler. Proceed to STEP 14.

- If no buildup present, use existing throttle muffler and proceed to STEP 14.
14. Install throttle muffler into the exhaust canister approximately 0.5 in. (1.27 cm). Refer to Detail “A”.
  15. Secure both halves of the exhaust canister together with two tie wraps.
  16. Reinstall cabinet. Refer to Removing Cabinet on page 26.

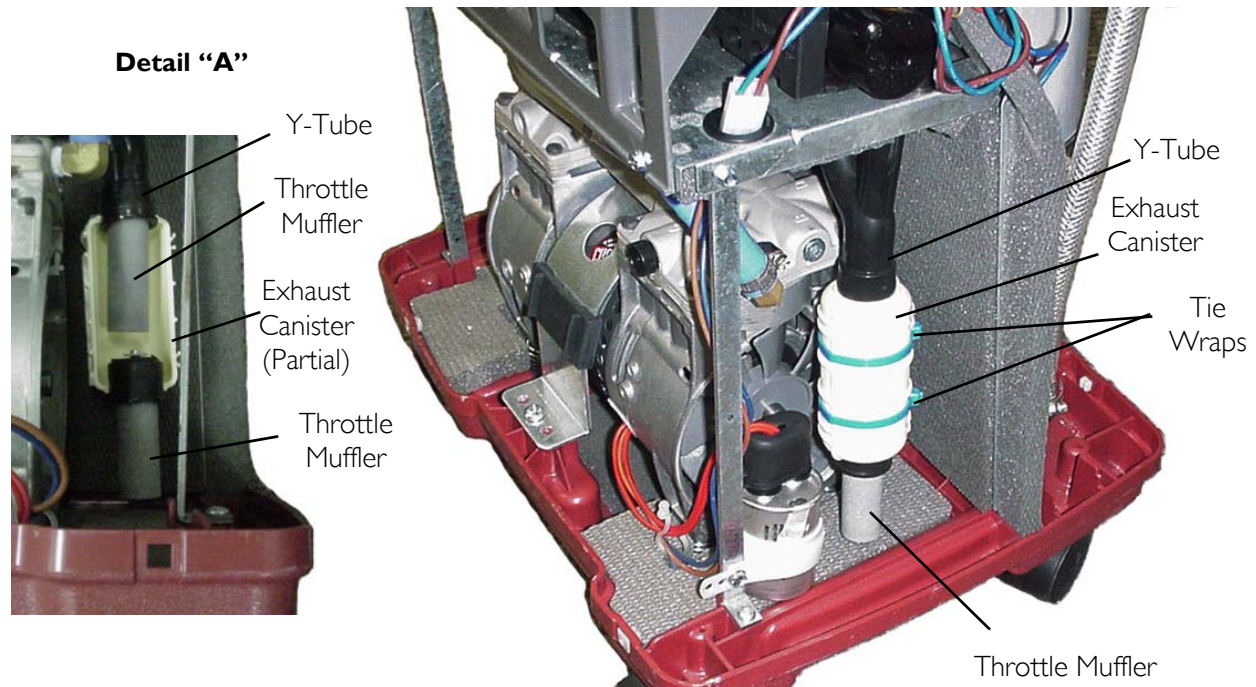

**FIGURE 6.7** Inspecting/Replacing the Throttle Muffler (Platinum 5 and 9 AWQ)

## Cleaning the Heat Exchanger

---

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 6.8 on page 35.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
  2. Remove cabinet. Refer to Removing Cabinet on page 26.
- 

### **CAUTION**

Use care not to deform heat exchanger when installing, removing or cleaning.

---

3. Remove excess dirt using compressed air or vacuum.
  4. Reinstall cabinet. Refer to Removing Cabinet on page 26.
-



**FIGURE 6.8** Cleaning the Heat Exchanger

**PREVENTIVE MAINTENANCE RECORD INVACARE PLATINUM OXYGEN  
CONCENTRATOR**

Model No. IRC \_\_\_\_\_

Serial No. \_\_\_\_\_

On Each Inspection

Record Date Of Service										
Record Elapsed Hours on Hour Meter										
Clean Cabinet Filters										
Check Prescribed L/min. Flow Rate										
Annually, During Preventive Maintenance Schedule, Or Between Patients										
Clean/Replace Cabinet Filters										
Check outlet HEPA Filter*										
Check Compressor Inlet Filter*										
Check Oxygen Concentration (SensO <sub>2</sub> models)										
Check Power Loss Alarm										
As Required										
Clean Heat Exchanger										
Replace Exhaust Muffler										
Rebuild Top End Compressor										
Every 180 Days (LX model)										
Check oxygen Concentration										

\*NOTE: Refer to Preventive Maintenance section of Service Manual.

---

# SECTION 7—COMPRESSOR

---

## **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

## **Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003**

*NOTE: For this procedure, refer to FIGURE 7.1 on page 38 and FIGURE 7.2 on page 39.*


1. Unplug unit.
2. Remove the cabinet. Refer to Removing Cabinet on page 26.
3. Loosen clamp and disconnect intake hose from compressor assembly.
4. Disconnect the compression fitting nut that connects the heat exchanger inlet to compressor fitting elbow (not shown).
5. Remove compressor wires from wire clamps or tie-wraps and disconnect motor connector from the main harness connector (not shown).

*NOTE: Motor connector goes through grommet before reaching the main harness connector.*

6. Remove protective boot on capacitor.

*NOTE: Reuse the protective boot if the replacement parts do not provide one.*

7. Disconnect capacitor wires (not shown) from the top of the capacitor.
8. Remove the two mounting screws that secure the compressor restraint bracket and remove the bracket.
9. Tilt compressor assembly toward front of unit and lift out.
10. Reverse STEPS 3-9 to install new compressor assembly.
11. Reinstall the cabinet. Refer to Removing Cabinet on page 26.
12. Run unit and inspect for leaks. Refer to Leak Test on page 93.

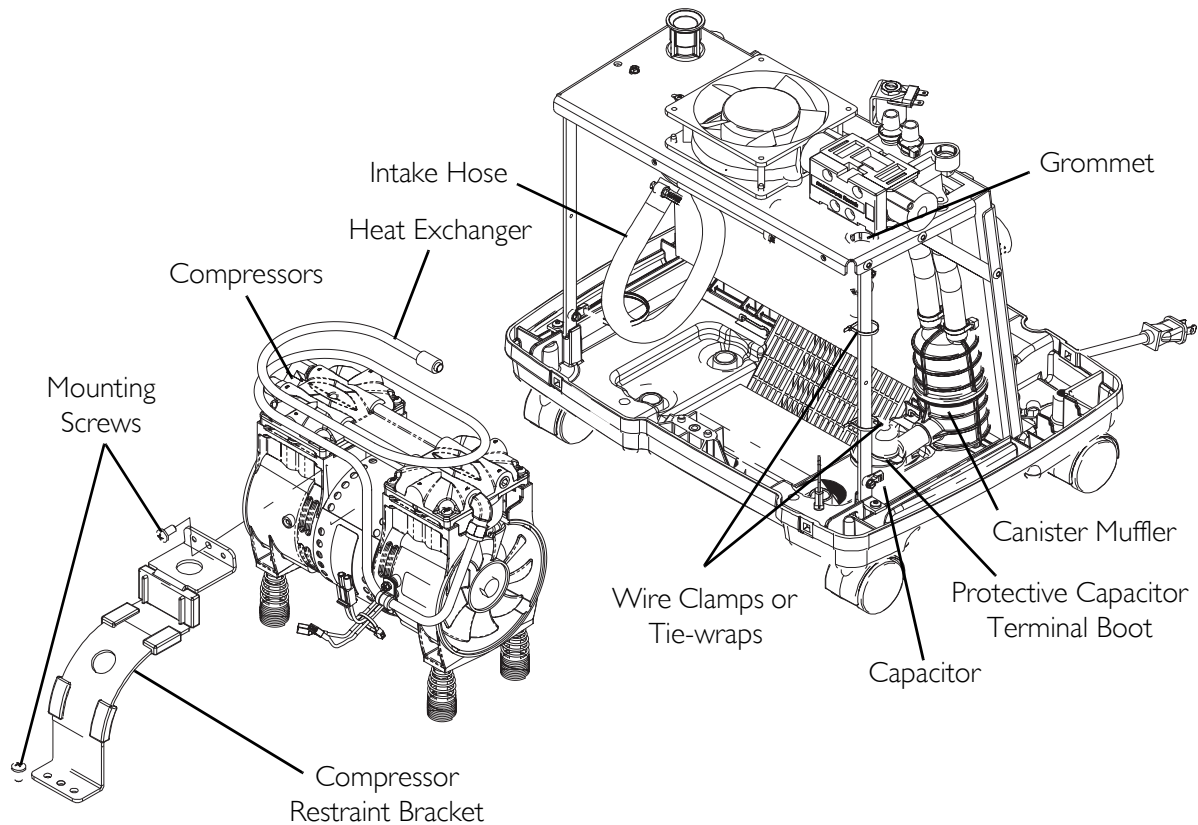

**FIGURE 7.1** Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003

## Replacing Compressor Assembly - 5 Liter Models manufactured after July 24, 2003

*NOTE: For this procedure, refer to FIGURE 7.2 on page 39.*


1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Loosen clamp and disconnect intake hose and heat exchanger input fitting from compressor assembly.
4. Remove heat exchanger cushioned clamp screw from the compressor assembly.
5. Remove compressor wires from wire clamps or tie-wraps and disconnect motor connector from the main harness connector.

*NOTE: Motor connector goes through grommet before reaching the main harness connector.*

6. Remove protective boot on capacitor.
7. Disconnect capacitor wires from the top of the capacitor.

*NOTE: Re-use the protective capacitor terminal boot if the replacement parts DO NOT provide one.*

8. Remove mounting screws that secure the compressor restraint bracket and remove the bracket.
9. Lift heat exchanger, tilt compressor assembly toward front of unit and lift out.
10. Reverse STEPS 3-7 to install new compressor assembly.
11. Run unit and inspect for leaks. Refer to Leak Test on page 93.
12. Reinstall cabinet. Refer to Removing Cabinet on page 26

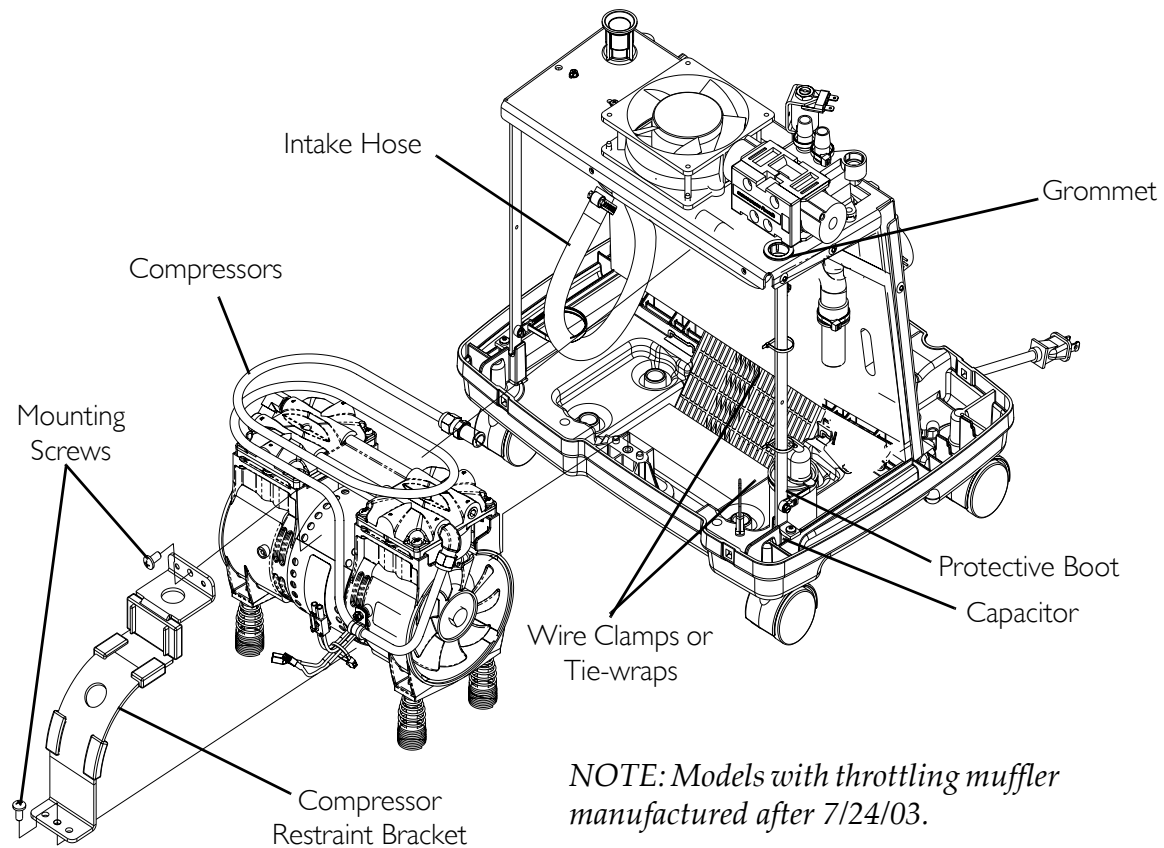


**FIGURE 7.2** Replacing Compressor Assembly - 5 Liter Models manufactured after July 24, 2003

## Replacing Compressor Assembly - 9 Liter Models

*NOTE: For this procedure, refer to FIGURE 7.3 on page 40.*

1. Unplug unit.
2. Remove the cabinet. Refer to Removing Cabinet on page 26.
3. Loosen clamps and disconnect intake hose and compressor output hose (blue) from compressor assembly.
4. Remove compressor wires from wire clamps or tie-wraps and disconnect motor connector from the main harness connector.

5. Remove protective boot on capacitor.
  6. Disconnect capacitor wires from the top of the capacitor.
- NOTE: Re-use the protective capacitor terminal boot if the replacement parts DO NOT provide one.*
7. Remove the two mounting screws that secure the compressor restraint bracket and remove the bracket.
  8. Tilt compressor assembly toward front of unit and lift out.
  9. Reverse STEPS 3-8 to install new compressor assembly.
  10. Reinstall the cabinet. Refer to [Removing Cabinet](#) on page 26.
  11. Run unit and inspect for leaks. Refer to [Leak Test](#) on page 93.

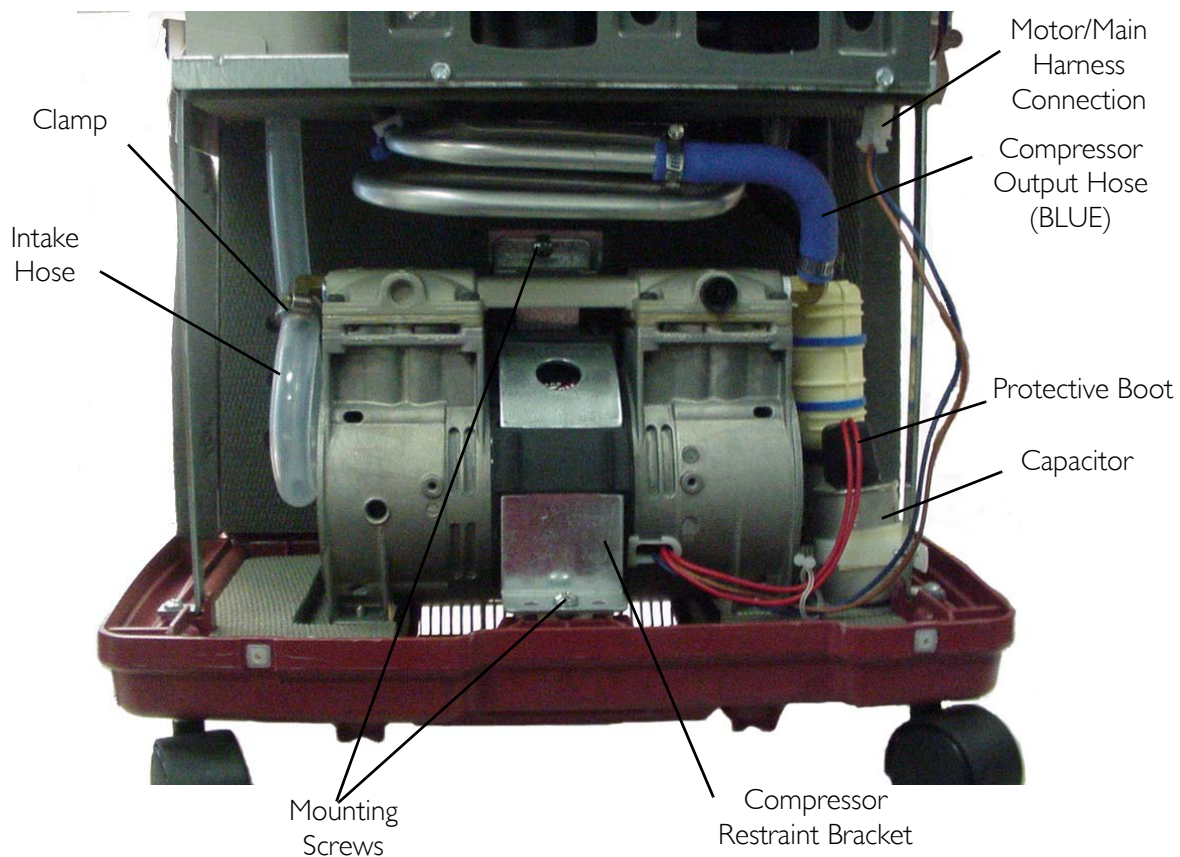

**FIGURE 7.3** Replacing Compressor Assembly - 9 Liter Models

## Replacing Capacitor


*NOTE: For this procedure, refer to Detail "A" in FIGURE 7.4 on page 41.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug the unit.
2. Remove the cabinet. Refer to [Removing Cabinet](#) on page 26.



3. Remove protective boot to reveal capacitor wires.
4. Disconnect the spade connectors from the capacitor terminals (not shown).
5. Remove the screw that secures the capacitor clamp to the soundbox brace.
6. Remove the capacitor from the concentrator base assembly.
7. Install new capacitor by reversing STEPS 3-6.
8. Re-install the cabinet. Refer to [Removing Cabinet](#) on page 26.

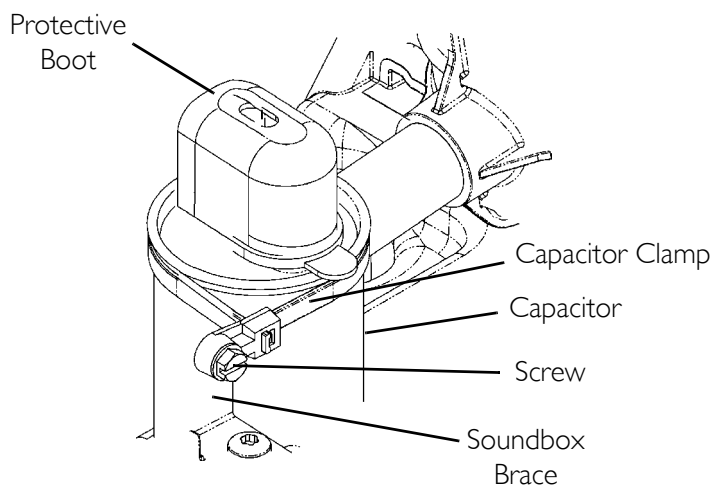

**FIGURE 7.4** Replacing Capacitor

## Rebuilding the Thomas Model 2650 and 2660 Compressor

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

This kit is designed for rebuilding the Thomas Model 2660 Series compressor. The serial number is located on the front of the compressor. If the serial number does not begin in this way, this kit will not work.

**The 5-Liter Thomas 2660 Series Compressor Rebuild Kit contains the following:**

DESCRIPTION	QUANTITY
Teflon® Piston Cups	2
Aluminum Piston Sleeves	2
Flapper Valves - Intake and Exhaust	4
Valve Keepers	4
O-Ring - Sleeve	2

DESCRIPTION	QUANTITY
Valve Plate Gaskets	2
Head Mounting Screws	8
Valve Screws	4
Valve Restraints	2
Piston Cup Retainer Screws	2

**Tools Required:**

- Torque wrench that has an inch-pound scale (for head screws, set screws, flapper valve screw, and pipe plugs)
- Torx® T-25 drive for torque wrench (for head screws)
- Torx T-27 drive for torque wrench (for retainer screws)
- 1/8-inch Allen wrench attachment for torque wrench (for eccentric set screw)
- 5/32-inch Allen wrench attachment for torque wrench
- Flat blade screwdriver attachment or 1/4-inch hex socket for torque wrench (for flapper valve screw)
- Phillips screwdriver attachment for torque wrench (for retainer screws)

**CAUTION**

**DO NOT lubricate or use oil on any moving parts. The compressor eccentric uses a precision sealed bearing. Therefore, additional lubrication is not necessary.**

**Removing Compressor and Compressor Heads****Compressor**

1. Unplug concentrator and carefully remove compressor from concentrator. Refer to Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003 on page 37.

**Compressor Head Plate**

*NOTE: For this procedure, refer to FIGURE 7.5 on page 43.*

1. Clean loose dirt from the outside of the compressor.
2. Use the Torx T-25 torque wrench and loosen the eight head screws.
3. Note head placement before removing compressor heads.
4. Remove the compressor head plate.


## Valve Plates

*NOTE: For this procedure, refer to FIGURE 7.5.*

1. Note the position of the plates before removing.
2. Lift off the valve plate on one side.
3. Repeat for the other side.

*NOTE: Note the position of the tab on valve plate for reassembly.*

*NOTE: To avoid confusion, only service one end of the compressor at a time.*

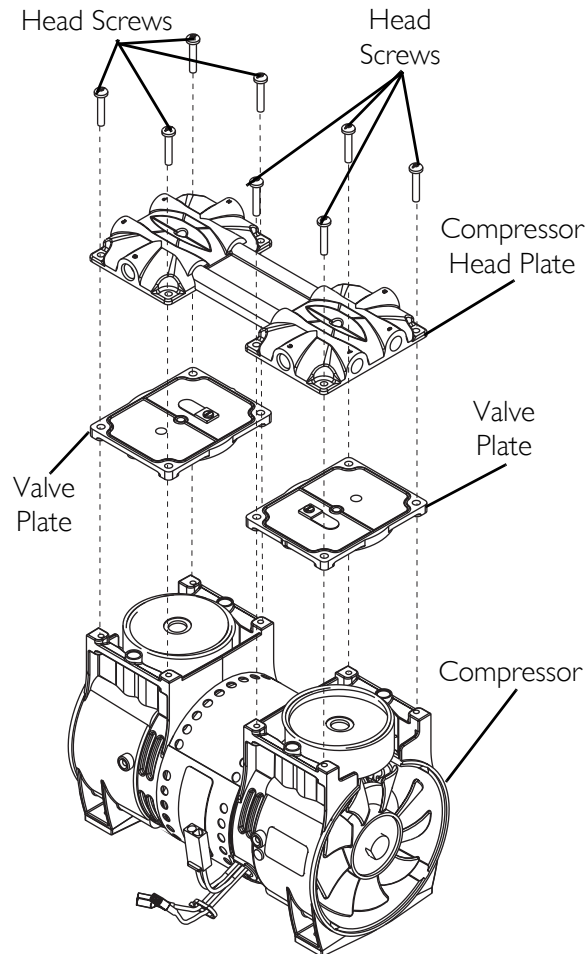

**FIGURE 7.5** Removing Compressor and Compressor Heads - Rebuilding the Compressor - Compressor - Compressor Head Plate - Valve Plates

## Fan

*NOTE: For this procedure, refer to FIGURE 7.6 on page 44.*

1. Remove the fan by pulling it straight off the motor shaft grabbing the center housing. DO NOT pull the fan blades.

*NOTE: Note position/orientation of fan for reassembly.*

### Connecting Rod and Eccentric Assembly

NOTE: For this procedure, refer to FIGURE 7.6.


1. Rotate shaft to align the eccentric's set screw with the access hole in the bottom of the compressor housing.
2. Insert the 1/8-inch Allen wrench into the access hole in the bottom of the compressor housing (Detail "A" of FIGURE 7.6).
3. Loosen the set screw ¼ turn.
4. Rotate connecting rod to top dead center (180°) and slide the connecting rod/eccentric assembly off the shaft through the opening in the housing.
5. Secure the rod assembly in a fixture.
6. Remove the sleeve from the connecting rod and discard.

### Piston Cup

NOTE: For this procedure, refer to FIGURE 7.6.

1. Remove the torque screw (discard) from the piston cup retainer (retain for reassembly).
2. Remove the piston cup (discard) and wipe debris from the top of the connecting rod with a clean damp cloth.

NOTE: As the piston travels up and down it will also rock from side-to-side. This is a feature of the WOB-L Piston.

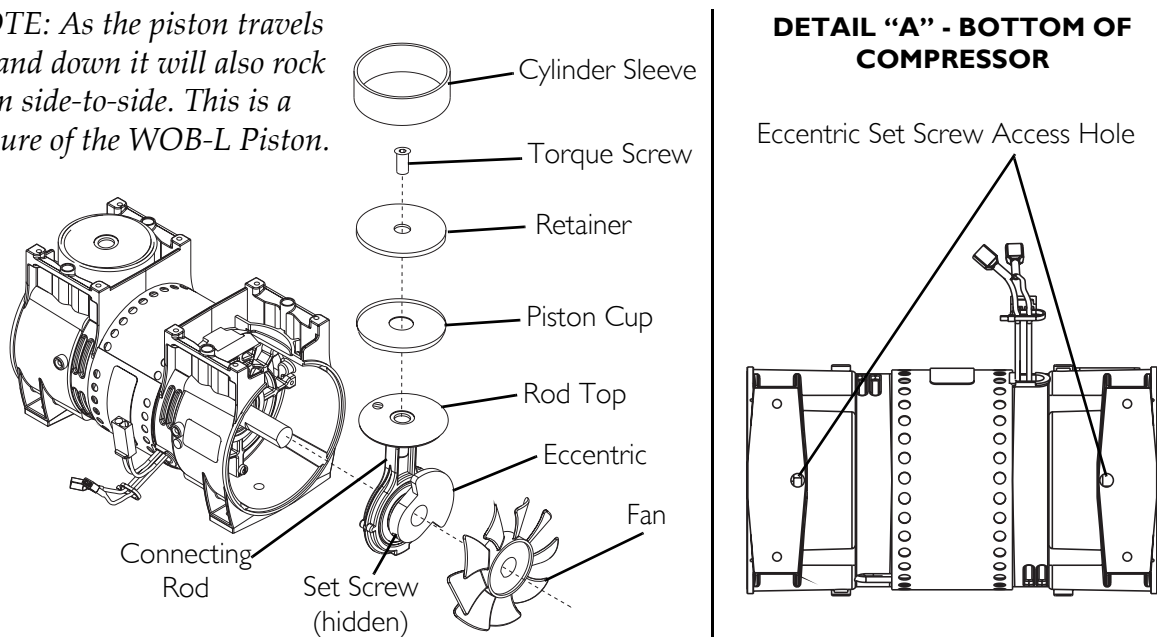


**FIGURE 7.6** Removing Compressor and Compressor Heads - Rebuilding the Compressor - Fan - Connecting Rod and Eccentric Assembly - Piston Cup

### O-ring, Intake Valve Keeper and Valve Keeper Strip

NOTE: For this procedure, refer to FIGURE 7.7 on page 45.

1. Remove the O-ring, intake valve flapper and valve keeper strip from the bottom of the valve plate. Discard all.
2. Clean the bottom of the valve plate with a clean, soft cloth.

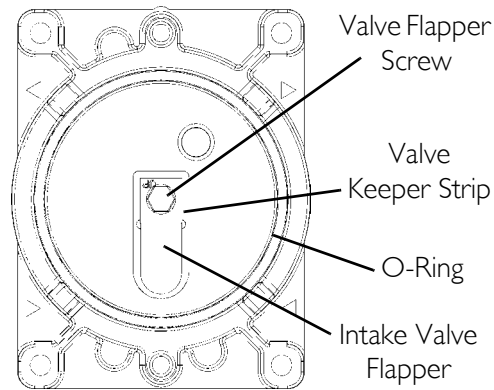

**FIGURE 7.7** Removing Compressor and Compressor Heads - Rebuilding the Compressor - O-ring, Intake Valve Keeper and Valve Keeper Strip

### Exhaust Valve Flapper, Valve Restraint and Valve Keeper Strip


*NOTE: For this procedure, refer to FIGURE 7.8.*

1. Remove the exhaust valve flapper, valve restraint and valve keeper strip from the top of the valve plate. Discard all.
2. Clean the top of the plate with a clean, soft cloth.

### Gasket

*NOTE: For this procedure, refer to FIGURE 7.8.*

1. Remove the old gasket from the top of the valve plate and discard.
2. Clean the valve plate with a clean, soft cloth.

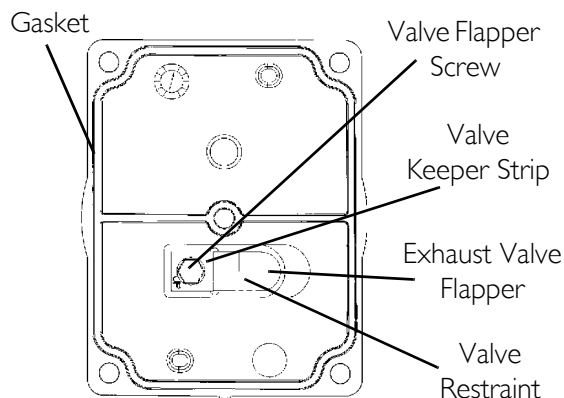

**FIGURE 7.8** Removing Compressor and Compressor Heads - Rebuilding the Compressor - Exhaust Valve Flapper, Valve Restraint and Valve Keeper Strip

## Rebuilding the Compressor

*NOTE: Before reassembly, wipe any residue from all components with a clean soft cloth.*

### Gasket

*NOTE: For this procedure, refer to FIGURE 7.8 on page 45.*

1. Install new gasket; seating the gasket firmly in the groove at the top of the valve plate with your finger or blunt object.

---

### CAUTION

**Make sure that the gasket is not twisted when seated in the groove of the top of the valve plate.**

---

### Exhaust Valve Flapper, Valve Restraint and Valve Keeper Strip

*NOTE: For this procedure, refer to FIGURE 7.8 on page 45.*

1. Install the new exhaust valve flapper, valve restraint and valve keeper strip to the top of the valve plate with the valve flapper screw.

*NOTE: Torque flapper screws to 1.36 N-M (12 inch-lbs).*

### O-ring, Intake Valve Flapper and Valve Keeper Strip

*NOTE: For this procedure, refer to FIGURE 7.7 on page 45.*

1. Install the new valve keeper strip on top of the new intake valve flapper so that the word up is visible. Fasten with the valve flapper screw.
2. Install the new O-ring, seating it firmly into the groove with your finger or blunt object.

*NOTE: Torque flapper screws to 1.36 N-M (12 inch-lbs).*

### Piston Cup

*NOTE: For this procedure, refer to FIGURE 7.6 on page 44.*

1. Place cylinder sleeve over rod top with smooth edge facing up.
2. Place piston cup on rod top making sure it is centered.
3. Place retainer on top of piston cup and seat retainer in rod top pocket, making sure the retainer and cup align properly along with screw holes.
4. Insert new screw into rod top and torque to 3.39 N-M (30 inch-lbs).

### Connecting Rod/Eccentric Assembly

*NOTE: For this procedure, refer to FIGURE 7.6 on page 44.*

1. With gloved-hand or clean cloth, pre-form piston cup if necessary.
2. Gently slide cylinder sleeve up and over cup to fully form it.

---

**CAUTION**

**DO NOT lubricate or use oil on any moving parts. The compressor eccentric uses a precision sealed bearing.**

**DO NOT crimp the piston cup when you replace the sleeve. If the cup is crimped, it MUST be replaced.**

---

3. With bearing side of connecting rod/eccentric assembly facing the center (motor) of the compressor, slide the assembly onto the shaft bringing it flush to the motor bearing.

*NOTE: Make sure the eccentric set screw is positioned over the flat of the shaft.*

4. Using the access hole, tighten set screw to 4.2 N-M (40 inch-lbs) (Detail "A" of FIGURE 7.6).
5. Hold the sleeve down to the housing with one hand and slowly rotate the eccentric with the other hand to ensure all the components are aligned properly.

**Fan**

*NOTE: For this procedure, refer to FIGURE 7.6 on page 44.*

1. Reinstall the fan onto the shaft in original orientation.

**Valve Plate**

*NOTE: For this procedure, refer to FIGURE 7.5 on page 43.*

1. With the sleeve located and firmly seated on the housing, replace the valve plate in the same manner as it was.
2. Repeat STEP 1 for other valve plate.

*NOTE: Make sure the top edge of the sleeve locates in the O-ring groove in the bottom of the valve plate.*

---

**CAUTION**

**Make sure that the O-ring is not twisted when seated in the groove of the bottom of the valve plate.**

---

**Compressor Heads**

*NOTE: For this procedure, refer to FIGURE 7.5 on page 43.*

1. Place the heads on top of the valve plates.
2. Tighten the head screws in a criss-cross pattern to 4.2 N-M (40 inch-lbs).

## Compressor

**⚠ WARNING**

To avoid personal injury or property damage, rotate the fan by hand prior to connecting the unit to a power source. Check for suction at the air inlet by placing your finger over the port as you rotate the fan. You should feel a slight suction with each rotation of the fan. If you **DO NOT** feel a suction but you feel or hear a thump as you turn the fan, **DO NOT** connect the unit to a power source. Review the assembly procedure for possible error.

*NOTE: For this procedure, refer to FIGURE 7.9 on page 49.*

1. Reinstall compressor. Refer to Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003 on page 37.
2. Run unit and check for leaks around compressor hoses and compressor tubes. Refer to Leak Test on page 93.
3. Take pressure readings to ensure proper system pressures. Refer to Models without HomeFill Connection On Back Of Unit on page 101.


**FIELD SERVICE PARTS LIST FOR THOMAS 2660 SERIES COMPRESSOR**

ITEM NO.	PART NO.	QTY. PER ASM	QTY. PER UNIT	DESCRIPTION
1	See Chart	—	2	Connecting Rod, Eccentric & Bearing Assembly
2	624277	1	2	Piston Cup
3	625776	1	2	Screw - Piston Cup Retainer
4	626175	1	2	Piston Cup Retainer
5	618114	1	2	Cylinder Sleeve
6	See Chart	—	1	Head
7	623143	—	2	O-Ring - Head Gasket
8	625175	—	8	Screw - Head
9	621591	—	2	Valve Plate Assembly
	638574	—	1	Valve - Pressure Relief-Not Shown (115 V)
10	617177	1	2	Valve Restraint
11	617562	2	4	Valve Keeper Strip
12	621485	2	4	Valve Flapper - Intake & Exhaust
13	621641	1	2	Valve Plate
14	625094	2	4	Screw - Valve Flapper
15	623137	—	2	O-Ring Valve Plate
16	638281	—	1	Fan - Gray
17	638282	—	1	Fan - Black

**2660 CHI44-989 COMPRESSOR (AW); 2660 CHI39-989 COMPRESSOR (AWQ)**

ITEM	PART NO.	QTY.	DESCRIPTION
1	607731	2	Connecting Rod, Eccentric & Bearing Assembly
6	665167	1	Head



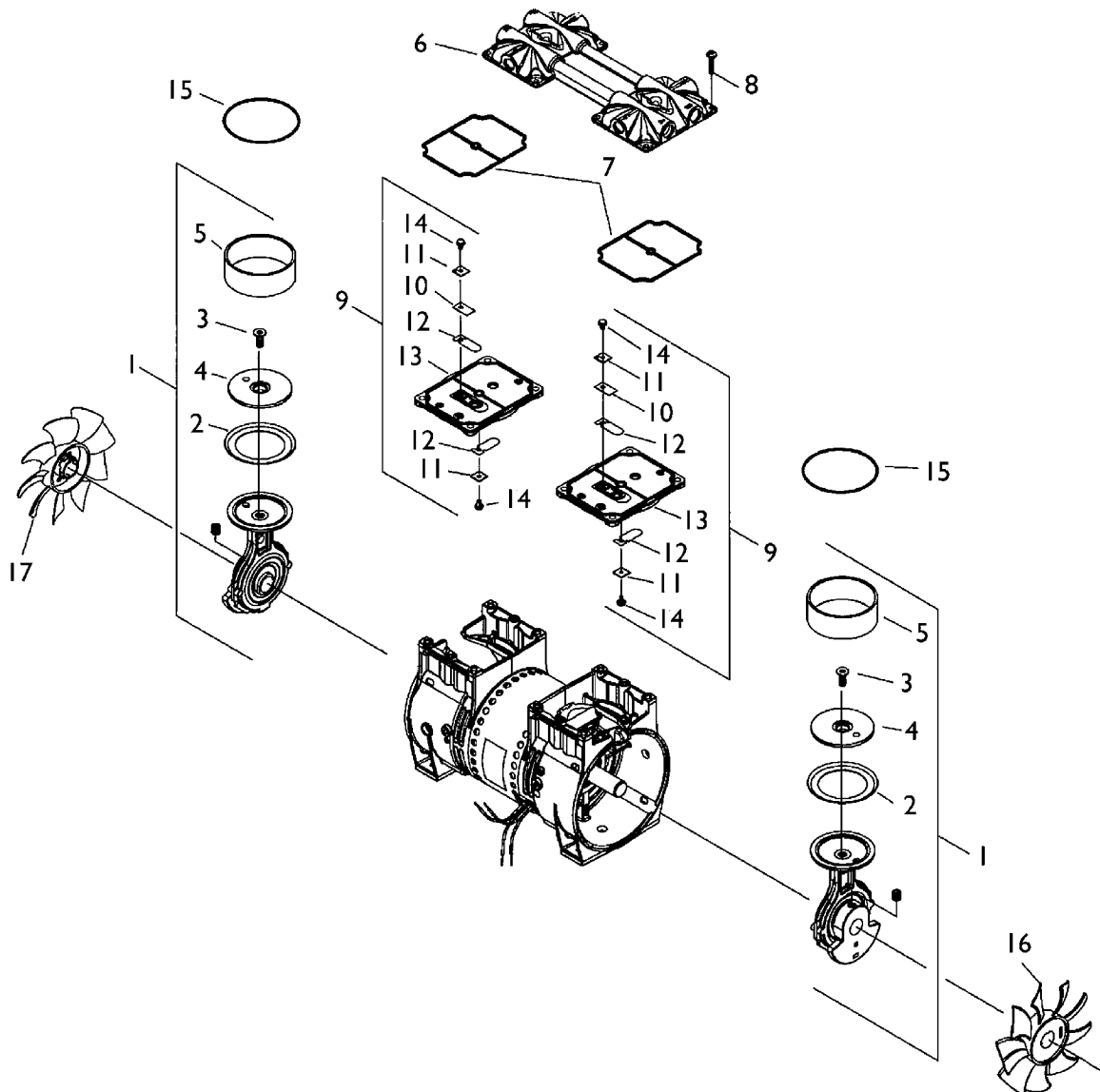

### COMPRESSOR PARTS LEGEND

1. Connecting Rod, Eccentric & Bearing Assembly	10. Valve Restraint
2. Piston Cup	11. Valve Keeper Strip
3. Screw - Piston Cup Retainer	12. Valve Flapper - Intake & Exhaust
4. Piston Cup Retainer	13. Valve Plate
5. Cylinder Sleeve	14. Screw - Valve Flapper
6. Head	15. O-Ring Valve Plate
7. O-Ring - Head Gasket	16. Fan - Grey
8. Screw - Head	17. Fan - Black
9. Valve Plate Assembly	

**FIGURE 7.9** Compressor Exploded View

## SECTION 8—P.E. VALVE

### Replacing P.E. Valve

---

#### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 8.1 on page 51.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
  2. Remove cabinet. Refer to Removing Cabinet on page 26.
  3. Remove spade connectors from P.E. valve assembly.
  4. Remove two tie-wraps that secure the sieve bed tubing to each side of the P.E. valve assembly.
  5. Remove P.E. valve assembly from tubing.
- 

#### **CAUTION**

**If P.E. valve is not to be installed IMMEDIATELY, plug the P.E. valve tubing to prevent sieve bed contamination.**


---

*NOTE: To ensure proper orientation, the “Out” port barb should face to the right side of the sieve bed when viewing the back of the unit.*

6. Install new P.E. Valve assembly IMMEDIATELY, by reversing STEPS 3-5.
7. Connect sieve bd tubing to each side of P.E. valve.
8. Secure sieve bed tubing with new tie wraps.
9. Connect spade connectors to P.E. valve assembly.

*NOTE: After replacing P.E. valve, retiming of the concentrator may be necessary. Refer to Timing on page 89.*

10. Run unit and check for leaks. Refer to Leak Test on page 93.
11. Reinstall cabinet. Refer to Removing Cabinet on page 26.

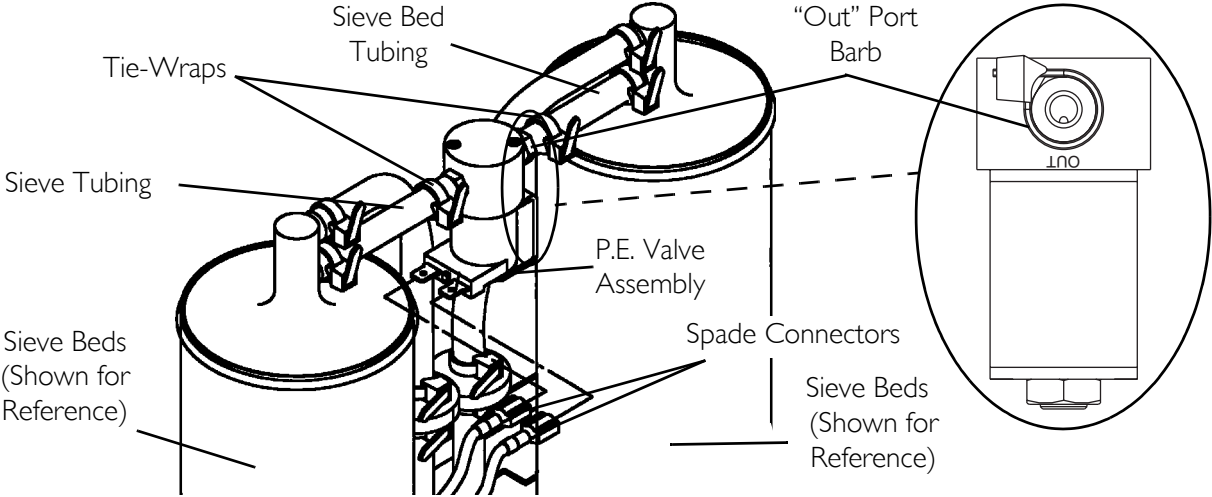

**FIGURE 8.1** Replacing P.E. Valve

# SECTION 9—SIEVE BEDS

## Replacing Sieve Beds

---

### **DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

*NOTE: For this procedure, refer to FIGURE 9.1 on page 53 and FIGURE 9.2 on page 54.*

*NOTE: ALWAYS replace sieve beds in pairs to ensure that both beds are in optimum condition.*

1. Unplug unit.
  2. Remove cabinet. Refer to Removing Cabinet on page 26.
  3. Remove tie-wraps and disconnect P.E valve asm and sieve bed tubing from top of sieve beds.
  4. For 5 Liter models manufactured before 1/27/05 only - Remove large tie-wraps.
  5. Loosen adjustable clamps on sieve beds. Slide clamps down below the adjustable clamp hangers.
  6. Lift sieve beds up and rest sieve beds on top rear of base.
  7. Remove tie-wraps/clamps and reinforced tubing from bottom of both sieve beds and remove sieve beds.
- 

### **CAUTION**


**DO NOT** remove plastic caps (not shown) from new sieve bed fittings until ready to install new sieve beds. Severe sieve contamination can occur if uncapped beds are exposed to air.

---


8. Remove plastic caps from the top and bottom fittings of the new sieve beds.
9. Install new sieve beds reversing STEPS 3-7.

*NOTE: After replacing sieve beds, retiming is necessary. Refer to Timing on page 89.*

10. Run unit and check for leaks. Refer to Leak Test on page 93.
11. Reinstall cabinet. Refer to Removing Cabinet on page 26.

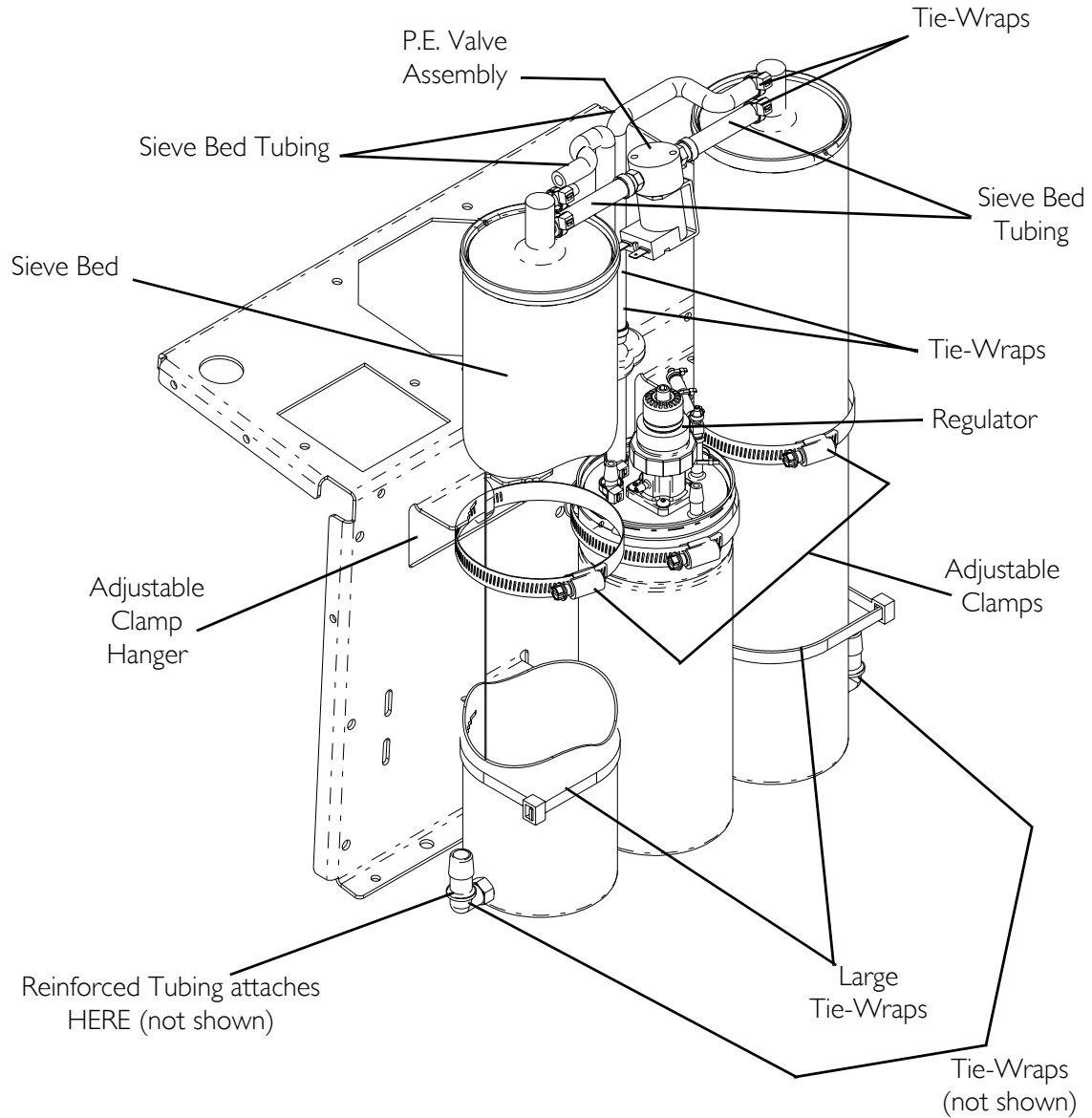

**FIGURE 9.1** Replacing Sieve Beds - 5 Liter models manufactured before 1/27/05

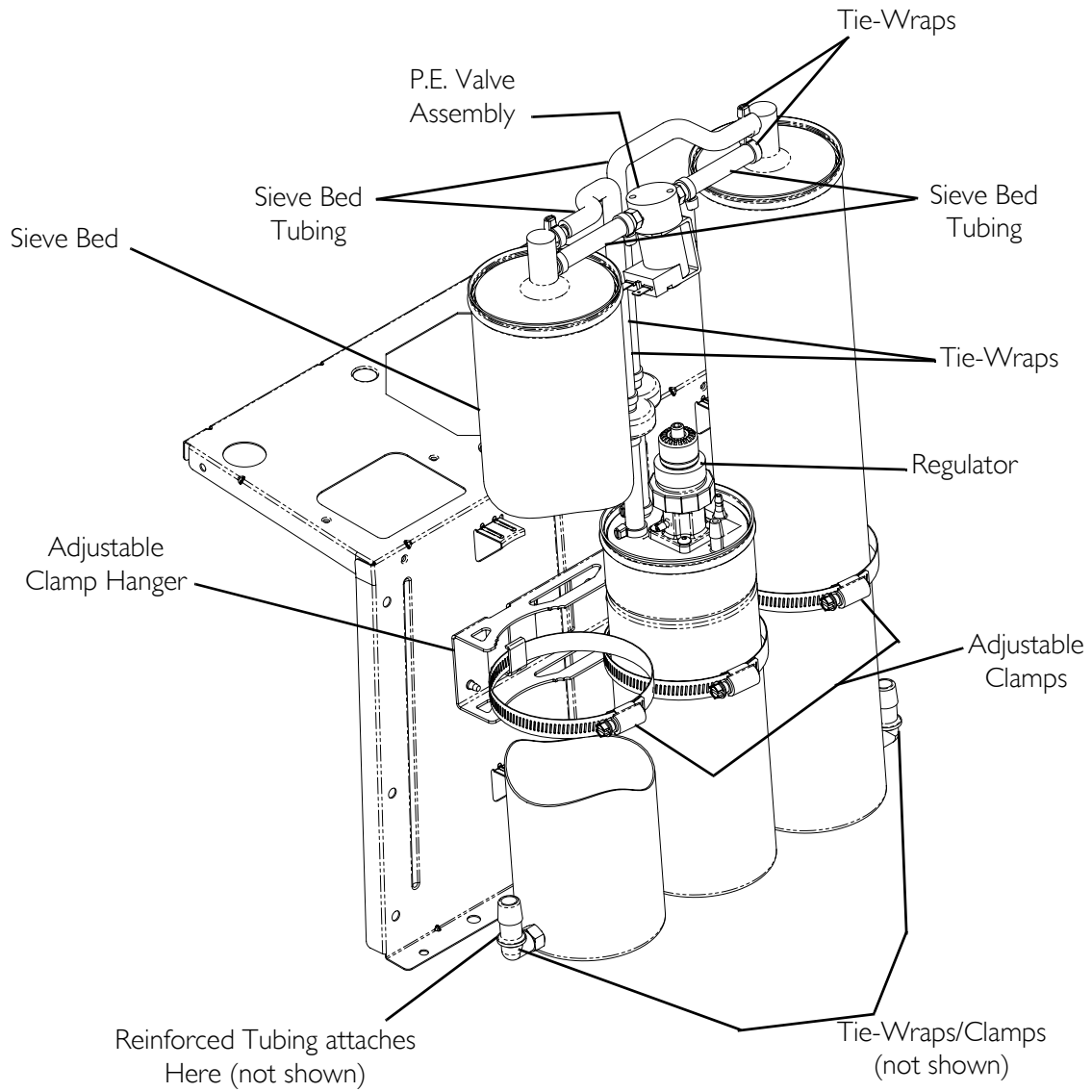

**FIGURE 9.2** Replacing Sieve Beds - 5 Liter models manufactured after 1/27/05 and 9 Liter models

# SECTION 10—CHECK VALVES

## Replacing Check Valves

---

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 10.1 on page 56.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove cabinet. Refer to [Removing Cabinet](#) on page 26.

*NOTE: Replace one or both check valves when performing this procedure. The check valves are one-way directional and can be checked by passing air through them. Air should flow in one direction only.*

3. Loosen adjustable clamp on product tank, slide clamp down below the adjustable clamp hanger, and pull product tank forward to access check valve(s).
4. Remove tie-wraps from top and bottom of the check valve(s) and disconnect check valve(s) from ¼-inch PVC tubing.

---

### **⚠ WARNING**

The check valves are one-way directional and **MUST** be installed correctly. The letters “VAC” are printed on the check valves. The check valves **MUST** be installed with the “VAC” on the underside of the check valve. Low system pressures and eventual bed contamination will result if check valves are not properly installed.

---


*NOTE: To ensure proper orientation, the “Out” port barb should face to the right side of the sieve bed when viewing the back of the unit.*

5. Install the new check valve(s) into the ¼-inch PVC tubing.
6. Make sure “VAC” is on the underside of the check valve to ensure proper orientation of check valves when installing.
7. Secure the check valve(s) by installing new tie-wraps to the ¼-inch PVC tubing where previously removed.

*NOTE: If ¼-inch PVC tubing is damaged in any way, replace the section of ¼-inch PVC tubing before installing check valve(s).*

*NOTE: Ensure check valves are not crossed. Left barb product tank check valve goes to left barb on sieve bed. Right barb product tank check valve to right barb on sieve bed.*

8. Run units and check for leaks. Refer to [Leak Test](#) on page 93.
9. Reinstall cabinet. Refer to [Removing Cabinet](#) on page 26.

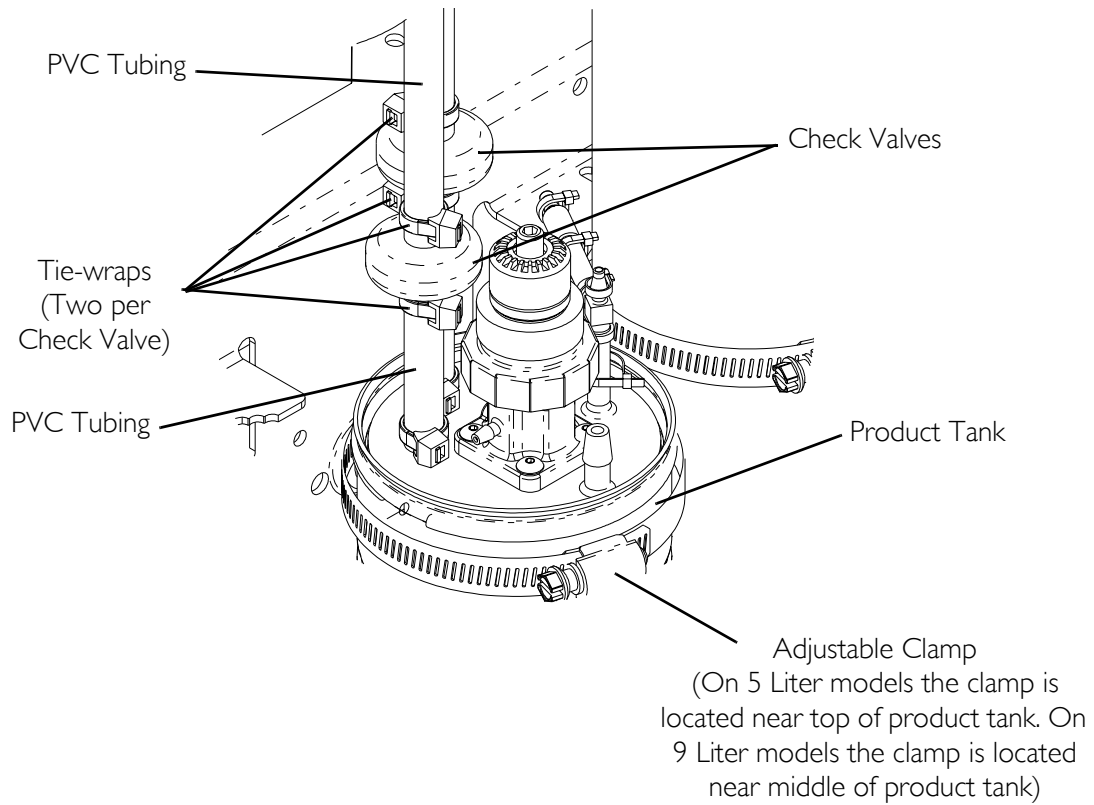

**FIGURE 10.1** Replacing Check Valves



---

# SECTION 11—REGULATOR

## Replacing Regulator

---

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 11.1 on page 58.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Loosen adjustable clamp on product tank and slide clamp down below the adjustable clamp hanger (Detail "A").
4. Pull product tank back and up to access regulator.

*NOTE: On SensO<sub>2</sub> concentrator models only, remove the 1/8-inch tubing to oxygen sensor housing.*

5. Remove the four mounting screws that secure regulator to product tank cap (Detail "B" or Detail "C").
6. Remove regulator from product tank lid ensuring that the O-rings are removed.


*NOTE: Clean mounting surface of product tank cap. Ensure that the two O rings for the new regulator are installed onto the ports (underside) of the new regulator before installation.*

---

### **⚠ WARNING**

**Ensure proper installation of regulator. Note flow arrow on regulator. This MUST face toward the front of unit or flow will be interrupted causing system shutdown.**

---

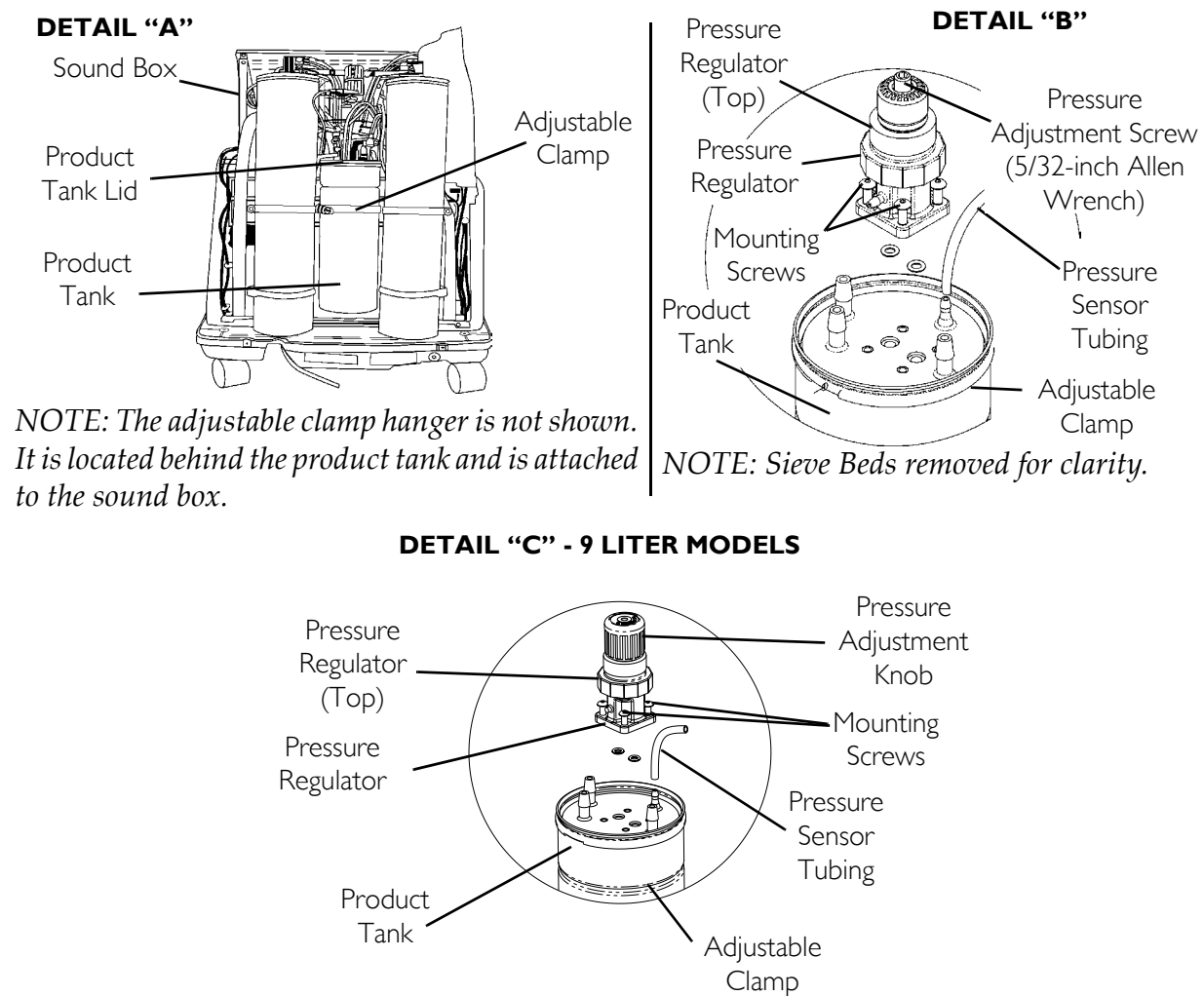**FIGURE 11.1** Replacing/Adjusting Regulator

7. Install new regulator onto product tank by reversing STEPS 5 and 6.
  8. Torque mounting screws to .56 N-M  $\pm$ .11 N-M (5  $\pm$  1 in-lbs).
  9. Re-install product tank and secure with the adjustable clamp.
- NOTE: On units with SensO<sub>2</sub>, reinstall 1/8-inch tubing to oxygen sensor housing.*
10. Adjust regulator pressure if necessary. Refer to [Adjusting Regulator](#) on page 58.
  11. Run unit and check for leaks. [Refer to Leak Test](#) on page 93.
  12. Reinstall cabinet. [Refer to Removing Cabinet](#) on page 26.

## Adjusting Regulator

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

*NOTE: For this procedure, refer to FIGURE 11.1.*

1. Turn unit on (I).

2. Install pressure gauge onto oxygen outlet.

*NOTE: Check O<sub>2</sub> pressure at oxygen outlet. It should read a steady 34.5 ± 3.45 kPa (5 ± 0.5 psi) for 5 Liter; 62.05 kPa ± 3.45 kPa (9 ± 0.5 psi) for 9 Liter. If pressure is not in specification, proceed to STEP 3. If pressure falls within specification, no adjustment is needed.*

3. Unplug unit.

4. Remove cabinet. Refer to Removing Cabinet on page 26.

5. Move product tank back to access regulator. Refer to Replacing Regulator on page 57.

6. Plug unit in and turn power switch on (I).

7. Locate the pressure adjustment screw for 5 Liter models (pressure adjustment knob for 9 Liter models) in center of pressure regulator top.

8. For 5 Liter models only, insert a 5/32-inch Allen wrench into the pressure adjustment screw.

9. While reading pressure gauge, do one of the following:

- Turn pressure adjustment screw or adjustment knob, clockwise to increase output pressure or,
- Turn pressure adjustment screw or adjustment knob, counterclockwise to decrease output pressure.

10. Adjust until pressure reads:

- For 5 Liter models: a steady 34.4 kPa ± 3.45 kPa (5 psi ± 0.5 psi).
- For 9 Liter models: a steady 62.05 kPa ± 3.45 kPa (9 psi ± 0.5 psi)

11. Once required pressure is achieved, reinstall cabinet. Refer to Removing Cabinet on page 26.

*NOTE: After ten minutes of run time, retest output pressure of concentrator to ensure proper operation and/or spec levels.*

12. Refer to Troubleshooting on page 18 if you are unable to adjust or maintain 34.4 kPa ± 3.5 kPa (5 psi ± 0.5 psi) for 5 Liter model and 62.05 kPa ± 3.45 kPa (9 psi ± 0.5 psi) for 9 Liter model.

# SECTION 12—HEAT EXCHANGER

## Replacing Heat Exchanger

### Platinum 5 Models Manufactured Before 9/9/2005

#### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.


*NOTE: For this procedure, refer to FIGURE 12.1 on page 60.*

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.

#### **CAUTION**

Use care not to deform heat exchanger when installing, removing or cleaning.

3. Remove compressor assembly. Refer to Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003 on page 37 or Replacing Compressor Assembly - 5 Liter Models manufactured after July 24, 2003 on page 38.
4. Remove screw securing cushioned clamp to the compressor.
5. Disconnect the compression fitting nut from it's body.
6. Loosen hose clamp and slide out heat exchanger.
7. Remove heat exchanger from the compressor assembly.
8. Reverse STEPS 3-6 to install new heat exchanger assembly.
9. Run unit and check for leaks. Refer to Leak Test on page 93.
10. Reinstall cabinet. Refer to Removing Cabinet on page 26.

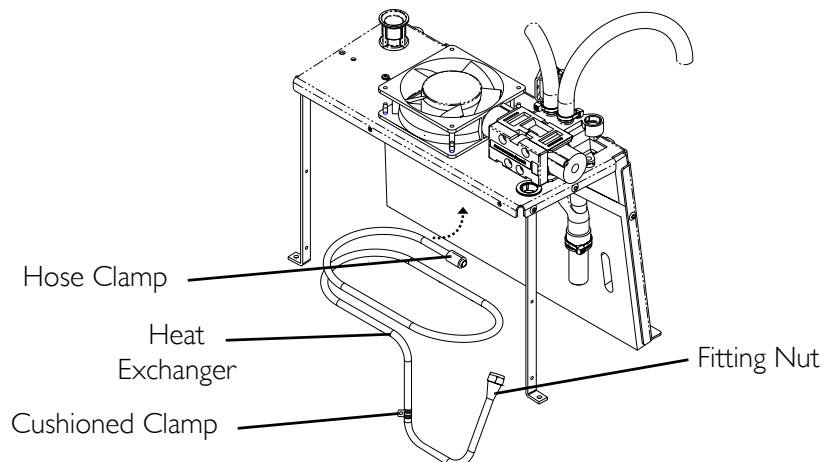

**FIGURE 12.1** Replacing Heat Exchanger - Platinum 5 Models Manufactured Before 9/9/2005

---

**Platinum 5 Models Manufactured After 9/9/2005**

---

**⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 12.2 on page 62.*


1. Unplug unit.
  2. Remove cabinet. Refer to Removing Cabinet on page 26.
- 

**CAUTION**

**Use care not to deform heat exchanger when installing, removing or cleaning.**

---

3. Remove compressor assembly. Refer to Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003 on page 37 or Replacing Compressor Assembly - 5 Liter Models manufactured after July 24, 2003 on page 38.
4. Loosen hose clamp and separate heat exchanger output fitting from hose elbow.
5. Remove heat exchanger from the soundbox assembly.
6. Reverse STEPS 3-6 to install new heat exchanger assembly.
7. Run unit and check for leaks. Refer to Leak Test on page 93.
8. Reinstall cabinet. Refer to Removing Cabinet on page 26.

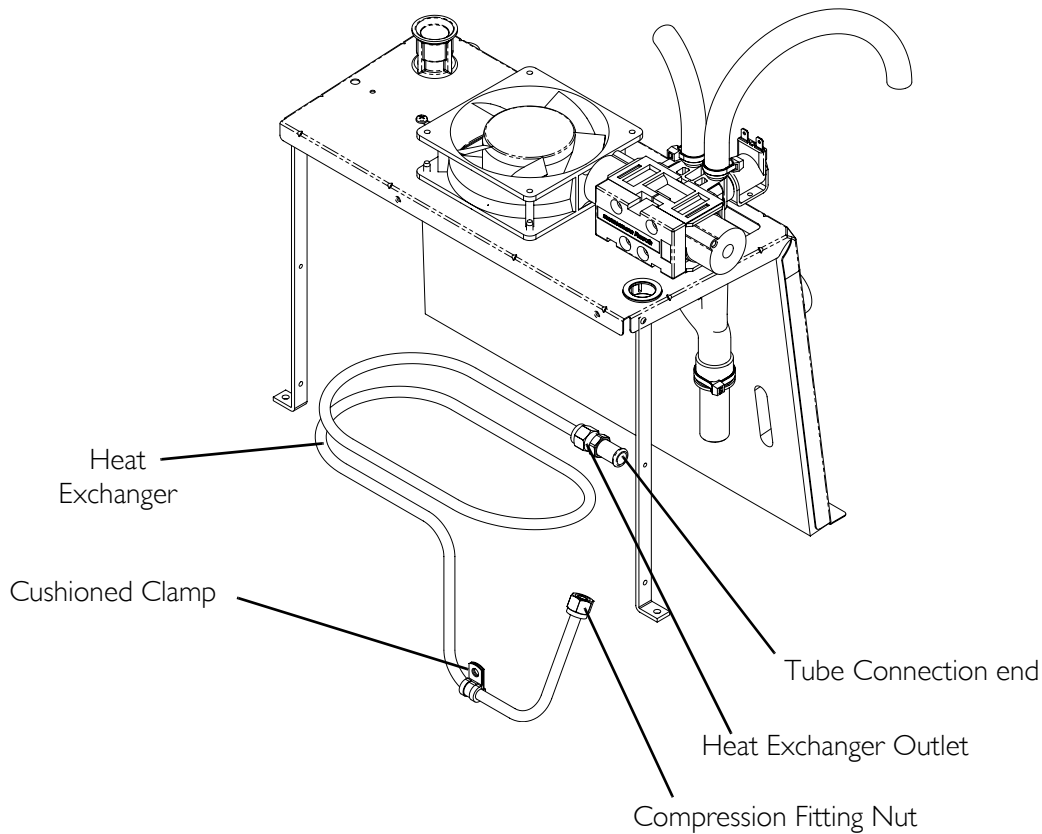

**FIGURE 12.2** Replacing Heat Exchanger - Platinum 5 Models Manufactured After 9/9/2005

**FIGURE 12.3**

## Platinum 9 Concentrators

---

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 12.4 on page 63.*

1. Unplug unit.
  2. Remove cabinet. Refer to Removing Cabinet on page 26.
- 


### **CAUTION**

**Use care not to deform heat exchanger when installing, removing or cleaning.**

---

3. Loosen the adjustable clamps on the compressor output hose (BLUE) and valve manifold input hoses at the front and rear of the heat exchanger.

4. Remove screw securing heat exchanger standoff to the soundbox.
5. Move the heat exchanger out to access the hoses and remove them from the heat exchanger.
6. Remove heat exchanger.
7. Reverse STEPS 3-6 to install new heat exchanger.
8. Run unit and check for leaks. Refer to Leak Test on page 93.
9. Reinstall cabinet. Refer to Removing Cabinet on page 26.

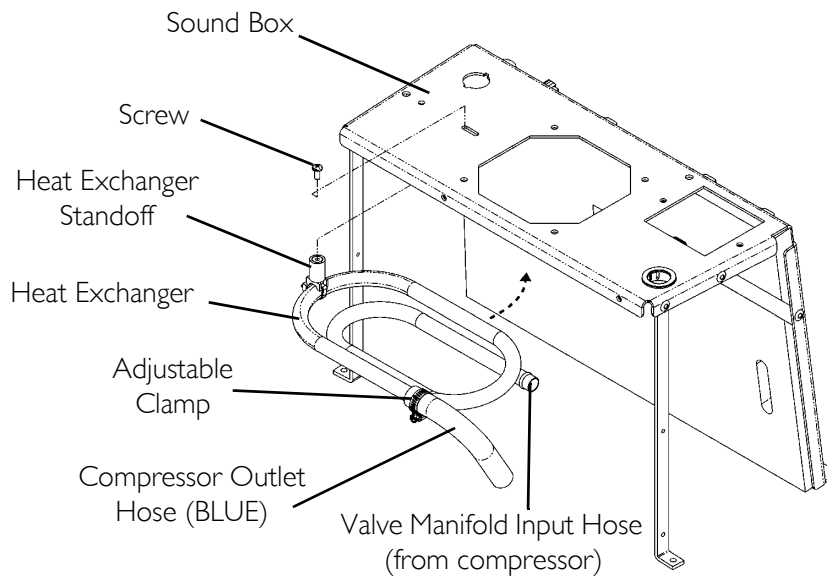

**FIGURE 12.4** Replacing Heat Exchanger

# SECTION 13—CONTROL PANEL

## Removing Control Panel

---

### **DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 13.1 on page 65.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

Tools Required:

- Diagonal Cutters
  - Flathead Screwdriver
  - ¼-inch ratchet, ¼-inch socket, and ¼-inch extension
  - Pliers
  - Tie-wraps
1. Unplug unit.
  2. Remove cabinet. Refer to [Cabinet](#) on page 26.
  3. Remove the two upper mounting screws that secure the control panel.
  4. Loosen the two lower mounting screws that secure the control panel.
- 

### **WARNING**

Use caution when removing the pressure sensor tubing from the product tank. Injury could result when tubing disconnects from product tank.

---

5. Cut the tie-wraps that secure the 1/8-inch I.D. pressure sensor tubing to the product tank connection and remove tubing.
6. Remove the ¼-inch I.D. tubing from the top and bottom barbed fittings of the flowmeter.
7. Cut tie wrap and remove ¼-inch I.D. tubing from the patient outlet barbed fitting behind the control panel.

*NOTE: On SensO<sub>2</sub> models the oxygen sensor tube attached to the side of the regulator MUST also be released.*


8. Disconnect the nine or ten pin connector from P.C. board.
9. Disconnect the eight pin connector from P.C. board.
10. Disconnect the transformer harness connector.

*NOTE: Before performing STEP 10, code all wires to ensure correct reinstallation.*

---



11. Remove spade connectors from circuit breaker, hour meter and On/Off (I/O) switch. (Jumper wire harness from On/Off switch to circuit breaker does not have to be removed).
12. Remove control panel and reverse STEPS 3-10 to re-install control panel.
13. Run unit and check for leaks at pressure sensor tubing on product tank cap. Refer to Leak Test on page 93
14. Reinstall cabinet. Refer to Removing Cabinet on page 26.

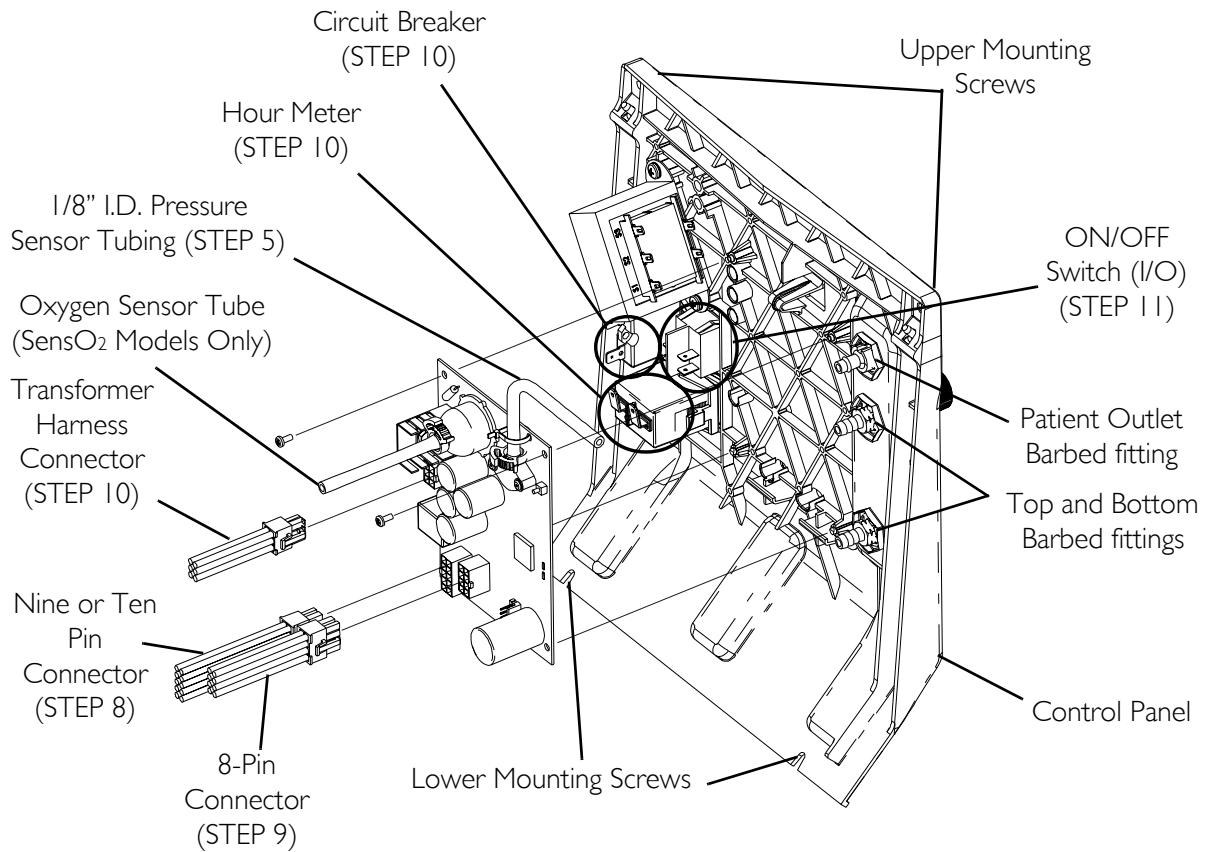

**FIGURE 13.1** Removing Control Panel

# SECTION 14—COOLING FAN

## Replacing Cooling Fan

---

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 14.1 on page 67.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
  2. Remove cabinet. Refer to Cabinet on page 26.
  3. Remove control panel. Refer to Removing Control Panel on page 64.
  4. Remove compressor assembly. Refer to Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003 on page 37 or Replacing Capacitor on page 40.
  5. Disconnect and remove spade connectors from fan terminals on back side of cooling fan (not shown).
  6. Remove four mounting screws securing the cooling fan to the sound box.
  7. Remove existing cooling fan.
- 

### **CAUTION**


**Fan MUST be positioned properly so air from the fan blows DOWN onto the compressor (see air flow arrow on back of fan) or damage to the unit will occur.**

---

8. Install new cooling fan.

*NOTE: Run the self-threading mounting screws through the fan housing holes to cut the threads and remove them before mounting the fan for ease of installation.*

9. Reinstall compressor. Refer to Replacing Compressor Assembly - 5 Liter Models manufactured before July 24, 2003 on page 37 or Replacing Capacitor on page 40.
10. Reinstall control panel. Refer to Removing Control Panel on page 64.
11. Plug unit in and check for leaks. Refer to Leak Test on page 93.
12. Run unit for at least 30 minutes. If unit runs properly, replace cover. If not, call Invacare Technical Service.
13. Reinstall cabinet. Refer to Cabinet on page 26.

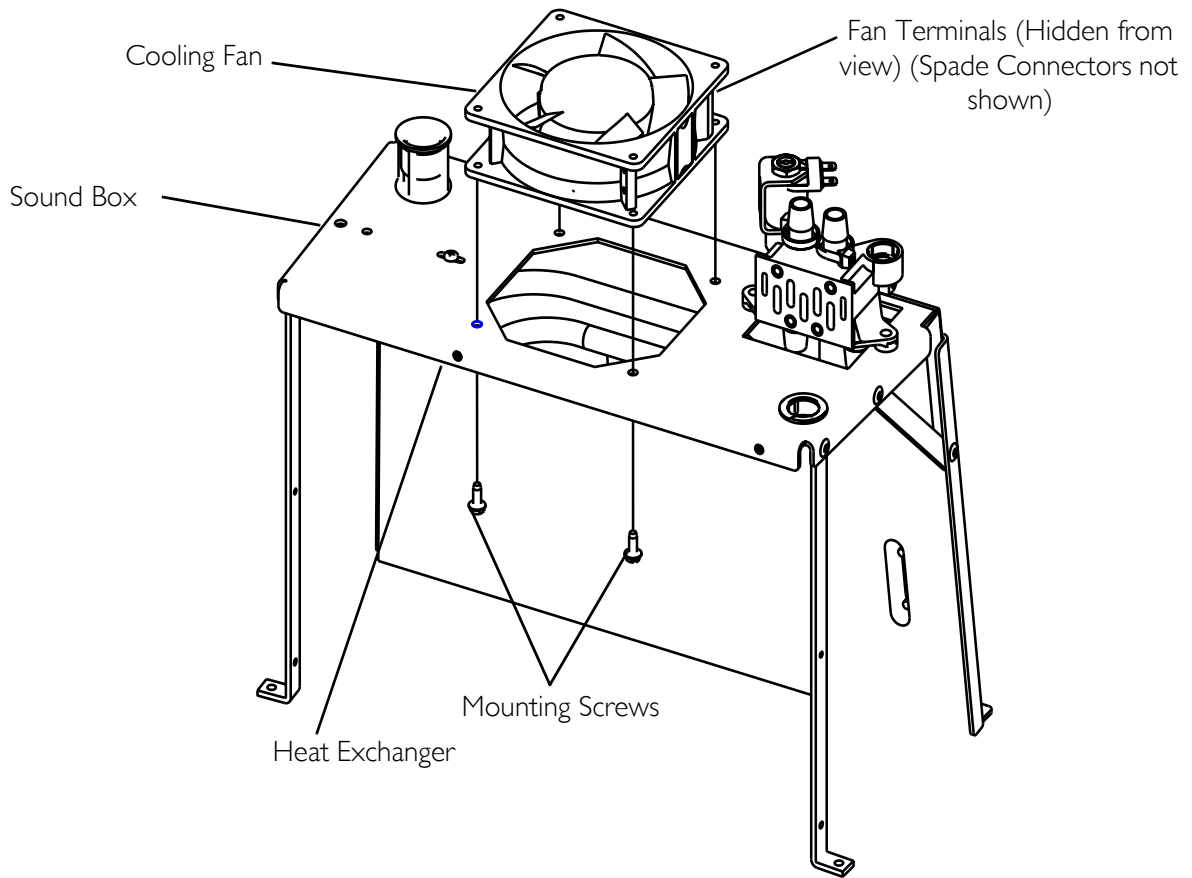

**FIGURE 14.1** Replacing Cooling Fan

# SECTION 15—P.C. BOARD

## Replacing P.C. Board

---

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

### **CAUTION**

Only qualified service technicians should make any changes to this equipment.

Follow these pre-cautions to prevent damage to the P.C. boards:

Before handling any P.C. boards, you need to be properly grounded to prevent static damage to the components of the board. A Static Cuff **MUST** be worn and properly grounded using an alligator clip. Electrical conduit or a water pipe is normally sufficient when a known good ground is not available. Care should be taken to ensure that the alligator clip contacts with bare metal surface.

When removing quick disconnects terminals, **DO NOT** pull on wire itself as damage to the connection may occur. Hold down the P.C. board with one hand and use an upward force with a slight rocking motion to remove the terminals.

Before installing any P.C. boards, ensure that all insulators are in place.

---

*NOTE: For this procedure, refer to FIGURE 15.1 on page 69.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
  2. Remove cabinet. Refer to [Removing Cabinet](#) on page 26.
  3. Remove control panel. Refer to [Removing Control Panel](#) on page 64.
  4. Disconnect the following:
    - A. Wiring Harness from nine or ten pin wiring harness connector
    - B. Wiring Harness from eight pin wiring harness connector
    - C. Transformer harness connector
- 


### **⚠ WARNING**

**DO NOT** remove sensor tubing from P.C. Board. Remove from tee fitting or regulator **ONLY**. Damage to sensor assembly could occur. A faulty sensor may result in improper oxygen delivery and harm to the user.

---

- D. Pressure sensor tubing from product tank, not P.C. Board.
  - E. If applicable, Oxygen Sensor tubing from Regulator Fitting. Refer to [Replacing Regulator](#) on page 57.
-

5. Remove the two mounting screws on top corners of P.C. board. Bend latching tab up to release P.C. board from control panel.
6. Position new P.C. board.
7. Reverse STEPS 4 and 5. Be sure all tie-wraps removed from tubing connections are replaced.
8. Reinstall control panel. Refer to Removing Control Panel on page 64.
9. Run concentrator to ensure unit operates to specifications.
10. After replacing P.C. board, retiming may be necessary. Refer to Timing on page 89.
11. Check for leaks at pressure sensor tubing on product tank cap. Refer to Leak Test on page 93.
12. Reinstall cabinet. Refer to Removing Cabinet on page 26.

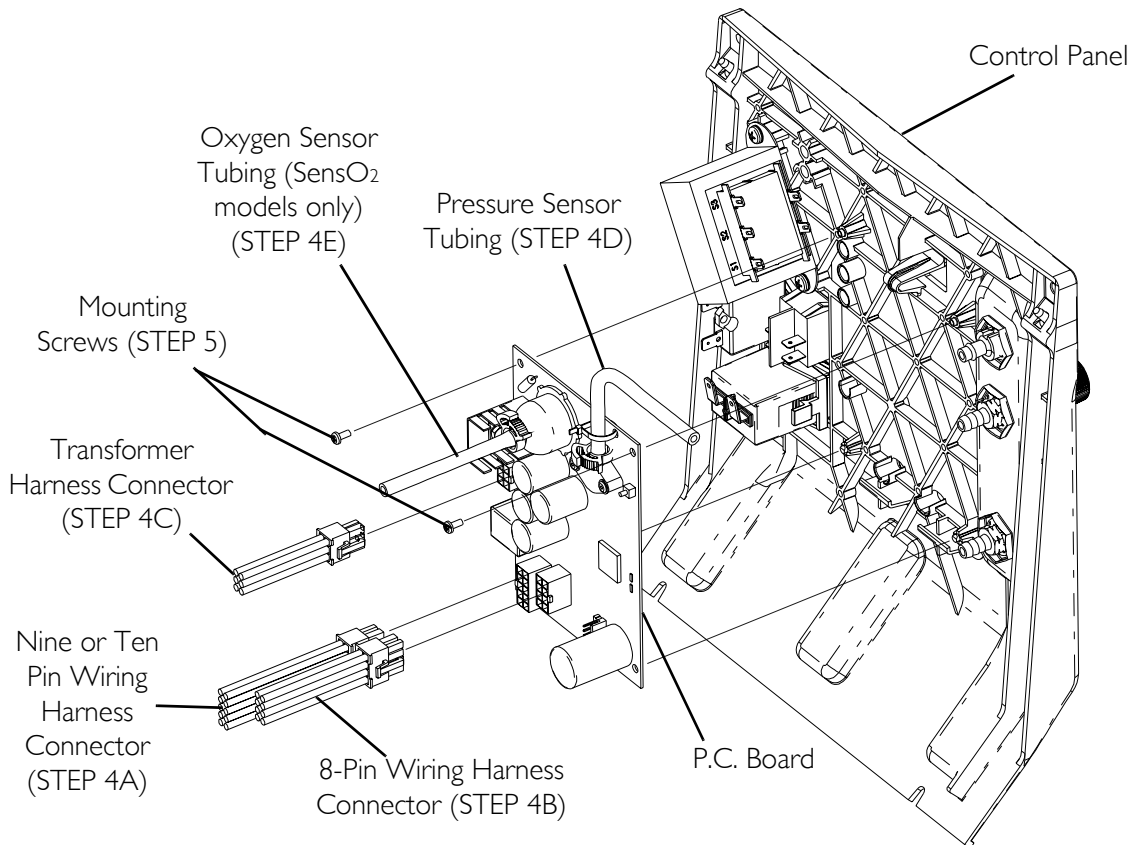

**FIGURE 15.1** Replacing P.C. Board

# SECTION 16—TRANSFORMER

## Replacing Transformer Assembly


### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

*NOTE: For this procedure, refer to FIGURE 16.1.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Remove control panel. Refer to Removing Control Panel on page 64.
4. Locate the transformer assembly in the upper left corner on the back of control panel.
5. Unplug the transformer connector from the P.C. board.
6. Remove the two keps screws that secure transformer to the back of the control panel.
7. Reverse STEPS 5-6 to install new transformer. Torque keps screw to  $1.53 \text{ N-M} \pm 0.11 \text{ N-M}$  ( $13.5 \pm 1 \text{ inch-lbs}$ ).
8. Reinstall control panel. Refer to Control Panel on page 64.
9. Reinstall cabinet. Refer to Cabinet on page 26.

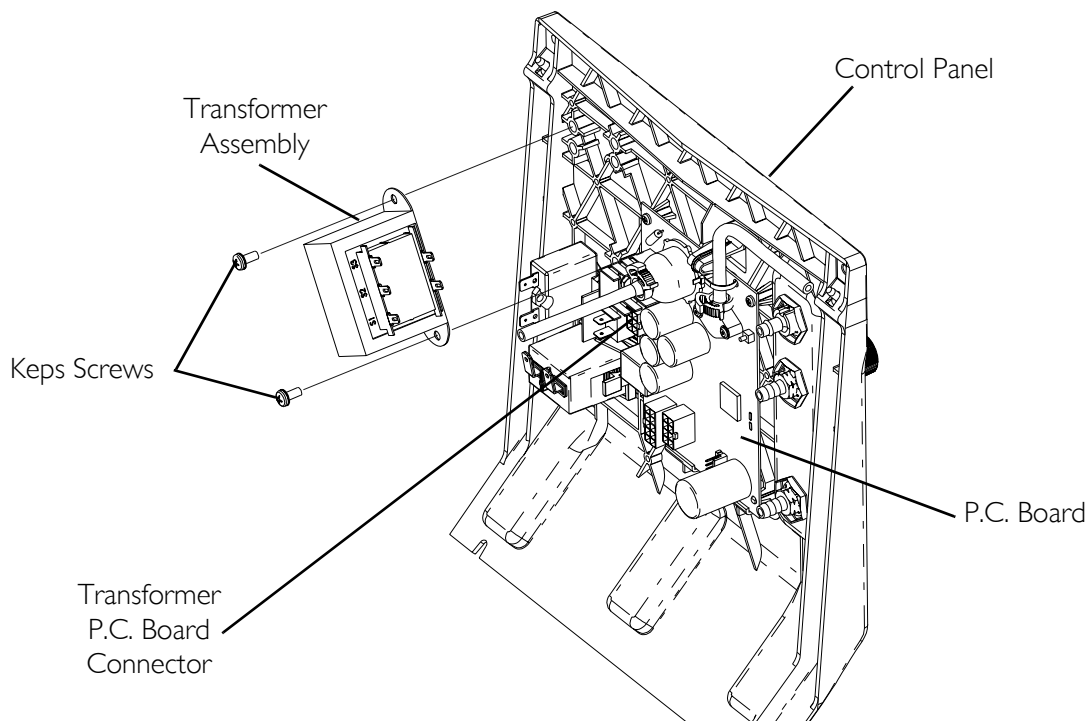

**FIGURE 16.1** Replacing Transformer Assembly

# SECTION 17—ON/OFF SWITCH

## Replacing On/Off Switch

---

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 17.1 on page 72.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Remove control panel. Refer to Removing Control Panel on page 64.
4. Remove and code the four spade connectors from back of existing on/off switch.
5. Compress retaining grips on back of existing on/off switch and push switch out through front of control panel.

*NOTE: Ensure proper orientation of the On/Off (I/O) switch and wiring connections before installation. Refer to FIGURE 17.1, Detail "A" and Detail "B" below.*


---

### **CAUTION**

**DO NOT** install the On/Off (I/O) switch upside down. Universal Off (O) symbol should be at bottom and Universal On (I) symbol should be at the top. Possible damage to the concentrator may result if not properly installed.

---

6. Remove switch cover from old part and place it on new switch prior to installation.
7. Reverse STEP 4 to secure new On/Off (I/O) switch.
8. Reinstall control panel. Refer to Removing Control Panel on page 64.
9. Reinstall cabinet. Refer to Removing Cabinet on page 26.

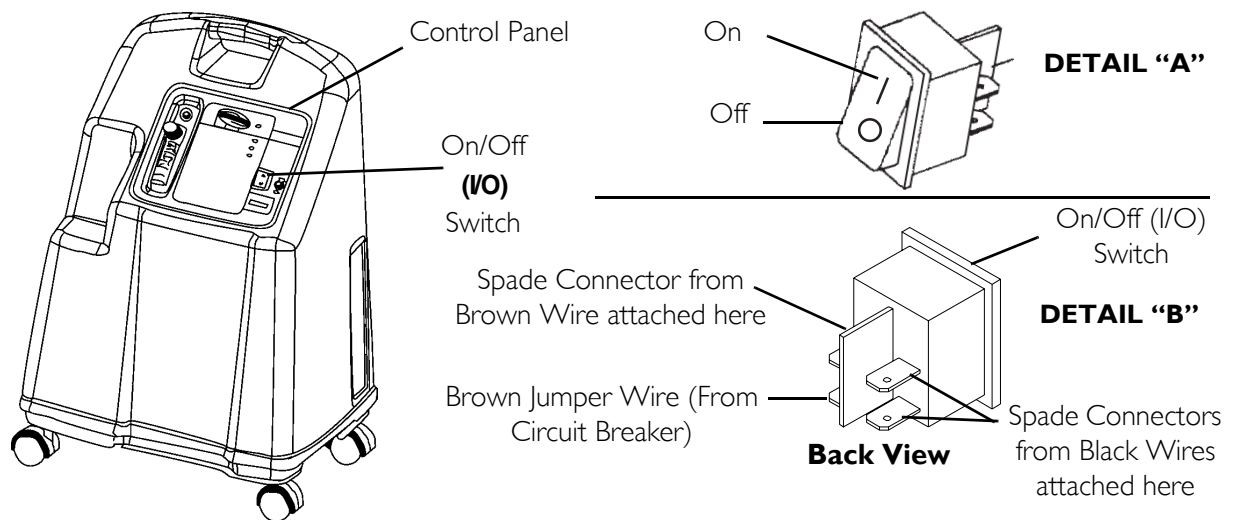

**FIGURE 17.1** Replacing On/Off Switch



---

# SECTION 18—FLOWMETER

## Replacing Flowmeter

---

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---


*NOTE: For this procedure, refer to FIGURE 18.1 on page 74.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Remove control panel. Refer to Removing Control Panel on page 64.

*NOTE: Note the location of the PVC tubing for accurate replacement after new flowmeter is installed.*

4. Remove palnuts that secure flowmeter to the control panel.
5. Remove flowmeter from front of control panel.
6. Install new flowmeter reversing STEPS 4-5.
7. Reinstall control panel. Refer to Removing Control Panel on page 64.
8. Run unit and check for leaks where ¼ inch PVC tubing engages flowmeter fittings. Refer to Leak Test on page 93.
9. Reinstall cabinet. Refer to Removing Cabinet on page 26.

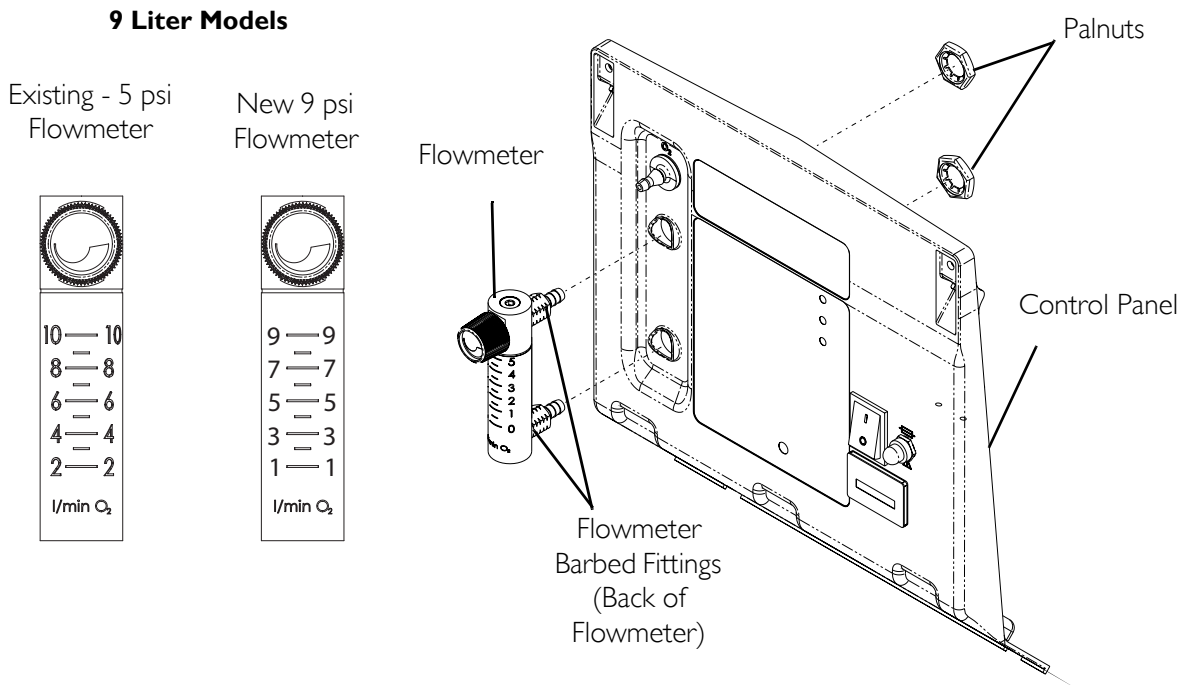

**FIGURE 18.1** Replacing Flowmeter

---

# SECTION 19—HOUR METER

## Replacing Hour Meter

---

**⚠ DANGER**


To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

*NOTE: For this procedure, refer to FIGURE 19.1 on page 76.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Remove control panel. Refer to Removing Control Panel on page 64.
4. Remove and code the two hour meter connectors and wires connected to back of hour meter.
5. Spread retaining clips on hour meter housing that secure hour meter to control panel.
6. Remove hour meter by pushing meter through front of control panel.
7. Install new hour meter reversing STEPS 4-6.
8. Reinstall control panel. Refer to Removing Control Panel on page 64.
9. Reinstall cabinet. Refer to Removing Cabinet on page 26.

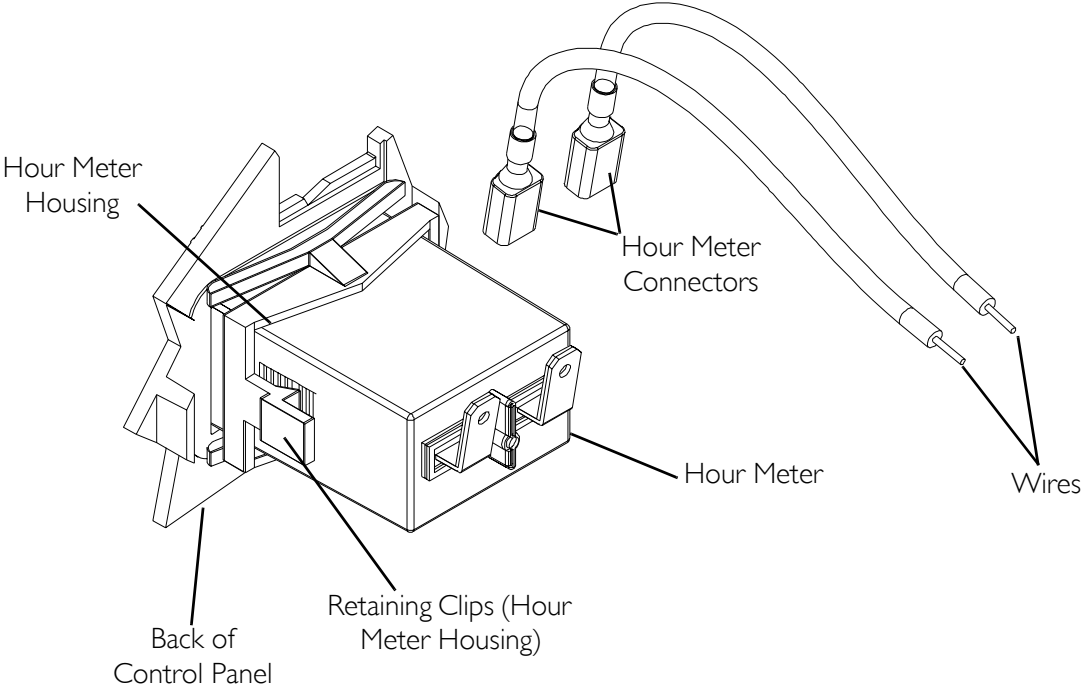

**FIGURE 19.1** Replacing Hour Meter

---

# SECTION 20— 4-WAY VALVE

---

## **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

## Replacing 4-Way Valve

*NOTE: For this procedure, refer to FIGURE 20.1 and FIGURE 20.2 on page 78.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Remove control panel. Refer to Removing Control Panel on page 64.
4. Remove the four mounting screws that secure the 4-way valve to the manifold assembly.
5. Position new 4-way valve onto manifold assembly and loosely install mounting screws through 4-way valve and into manifold assembly.

*NOTE: Ensure the manifold gasket is in place on 4-way valve before installing.*

---

## **CAUTION**

**Torque sequence and specification MUST be adhered to or possible damage to the 4-way valve could result.**


---

6. Tighten mounting screws in the sequence described below.
  7. Pre-torque the mounting screws to 1.1 N-M  $\pm$  0.2 N-M (10 in-lbs  $\pm$  2 in-lbs) using the torque sequence.
  8. The mounting screws can now be torqued to 2.5 N-M  $\pm$  0.2 N-M (22 in-lbs  $\pm$  2 in-lbs) in the same sequence.
- 

## **CAUTION**


**This is a maintenance free valve. Opening the valve will void any and all warranties applicable to the valve.**

---

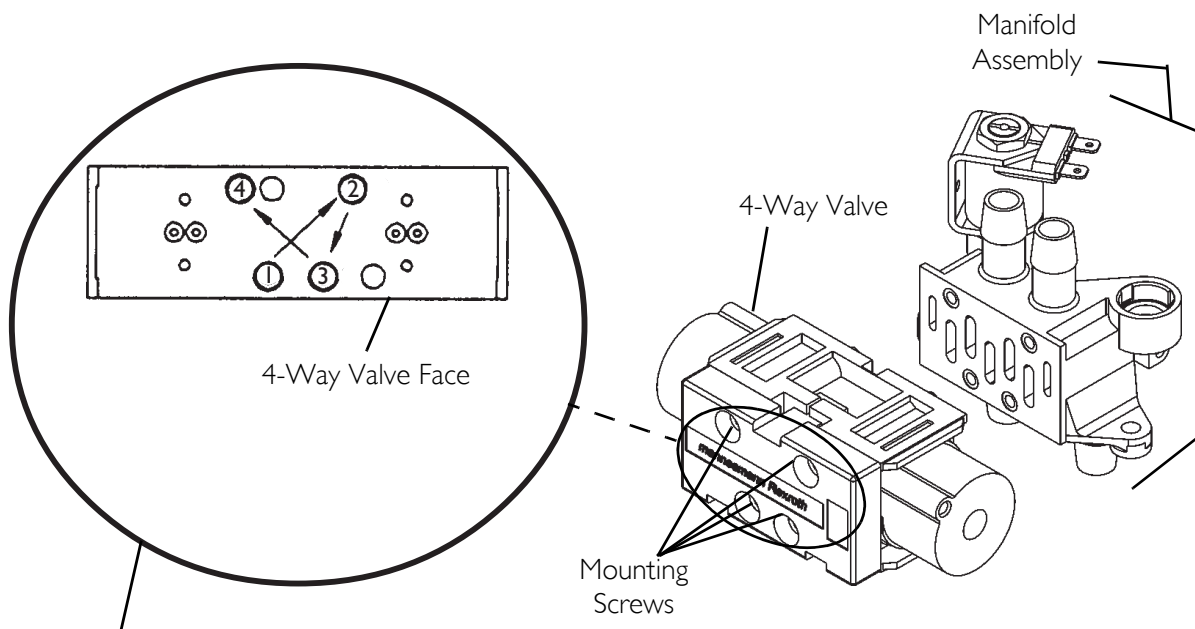

*NOTE: The Numbers indicate the mounting screw torque sequence: pre-torque to 1.1 N-M  $\pm$  0.2 N-M (10 in-lbs  $\pm$  2 in-lbs), then torque to 2.5 N-M  $\pm$  0.2 N-M (22 in-lbs  $\pm$  2 in-lbs).*

**FIGURE 20.1** Replacing 4-Way Valve - 5 Liter Models Manufactured Before 8/1/05

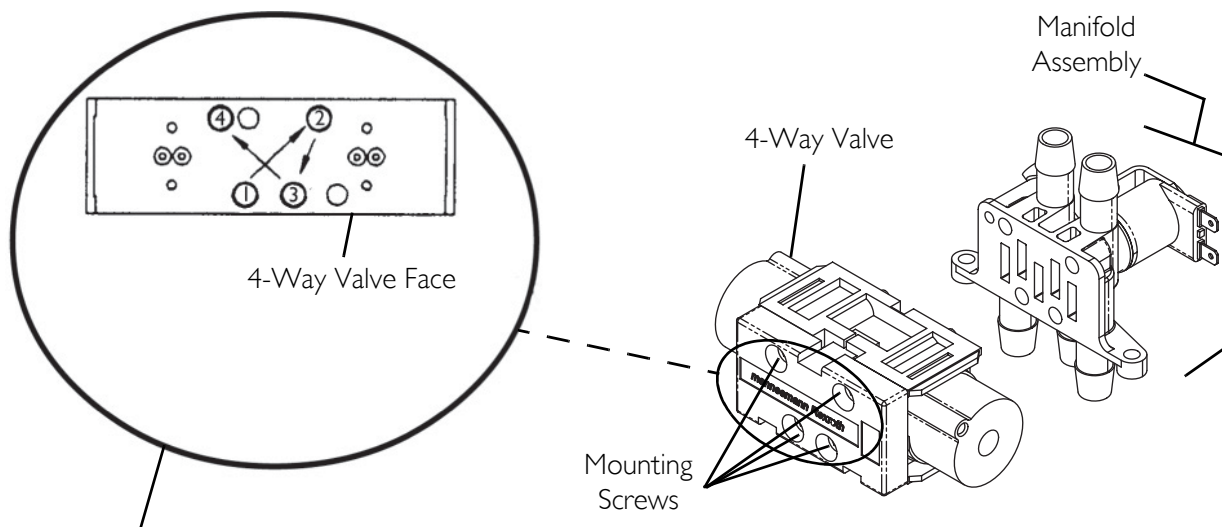

*NOTE: The Numbers indicate the mounting screw torque sequence: pre-torque to 1.1 N-M  $\pm$  0.2 N-M (10 in-lbs  $\pm$  2 in-lbs), then torque to 2.5 N-M  $\pm$  0.2 N-M (22 in-lbs  $\pm$  2 in-lbs).*

**FIGURE 20.2** Replacing 4-Way Valve - 5 Liter Models Manufactured After 8/1/05 and 9 Liter Models

## Replacing Manifold Assembly

### 5 Liter Models Manufactured before 7/24/03

*NOTE: For this procedure, refer to FIGURE 20.3 on page 81.*

*NOTE: This procedure applies to Platinum AW units manufactured before 7/24/03. Units manufactured before 7/24/03 are equipped with an exhaust canister with exhaust hose as shown in FIGURE 20.3.*

Tools Required:

- Diagonal Cutters
- Flathead Screwdriver
- ¼-inch ratchet, ¼-inch socket, and ¼-inch extension
- Pliers
- Tie-wraps

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Remove control panel. Refer to Removing Control Panel on page 64.
4. Cut and remove the tie-wraps connecting the sieve bed hoses to top of manifold assembly (Detail "A").

*NOTE: When removing the sieve bed hose from the manifold assembly use a flat head screwdriver to pry hose up while simultaneously pulling the hose upward.*


5. Remove sieve bed hoses from manifold assembly (Detail "A").
6. Remove and label the two spade connectors from the valve pilot on top of the manifold assembly (Detail "A").
7. Remove the mounting screw(s) that secures exhaust canister assembly to rear of the sound box. Refer to Replacing Exhaust Canister/Exhaust Muffler (Platinum 5 Manufactured before 7/24/03) on page 31.
8. Remove exhaust muffler from exhaust canister by turning counter-clockwise.
9. Loosen the adjustable clamp (not shown) that secures valve manifold input hose to heat exchanger (Details "A" and "C").

*NOTE: When removing the valve manifold input hose from the heat exchanger, use a flat head screwdriver to pry hose outward while simultaneously pulling the hose outward.*

10. Remove valve manifold input hose from end of heat exchanger (Details "A" and "C").
11. Remove the two mounting screws, washers (1 per side) and bushings (2 per side) securing the 4-way valve/manifold assembly to top of sound box (Detail "B").
12. The 4-way valve/manifold assembly can be removed from the concentrator by lifting the assembly straight up through the cutout in the sound box.
13. Remove 4-way valve from manifold assembly. Refer to Replacing 4-Way Valve on page 77 (Detail "A").

14. Remove exhaust canister with hoses from manifold.
15. Loosen the adjustable clamp on valve manifold input hose to bottom center port of manifold assembly.
16. Remove valve manifold input hose from bottom center port of manifold assembly. Use screwdriver to pry at connection of tubing at barbed fitting while pulling tubing away from manifold assembly.
17. Reverse STEPS 13-16 to re-assemble.
18. Slide 4-way valve/manifold assembly complete through the cutout in top of sound box then install the two mounting screws, washer and four bushings into the manifold to secure the 4-way valve assembly to the top of sound box (Detail "B").
19. Reattach components by reversing STEPS 4-10.
20. Reinstall control panel. Refer to Removing Control Panel on page 64.
21. Plug unit in and turn power On (I).
22. Run unit and check for leaks at hose connections. Refer to Leak Test on page 93.
23. Check for proper system operation. Refer to Typical Product Parameters on page 7.
24. Re-install cabinet. Refer to Removing Cabinet on page 26.



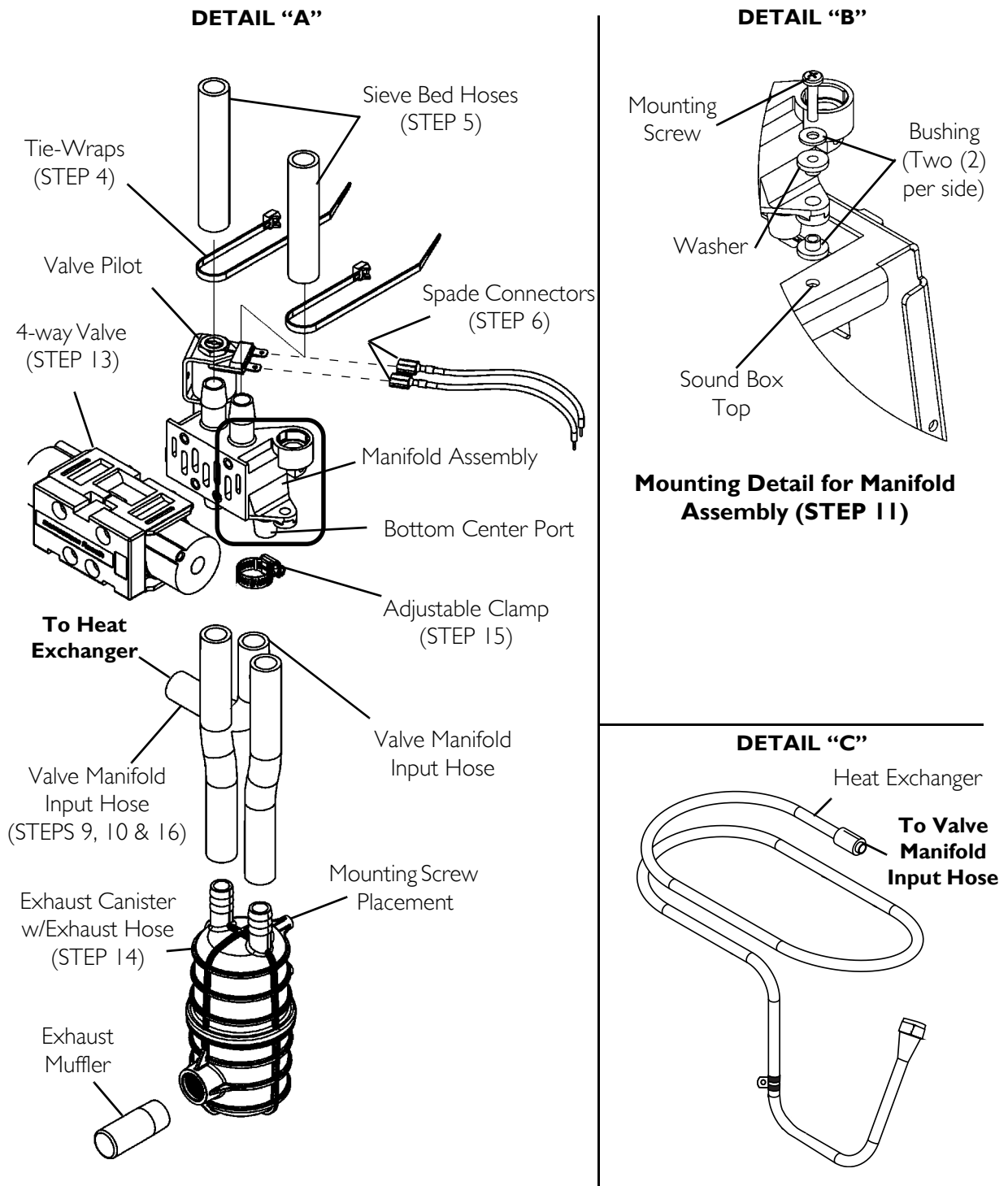

**FIGURE 20.3** Replacing Manifold Assembly- 5 Liter Models Manufactured before 7/24/03

## 5 Liter Models Manufactured after 7/24/03 and 9 Liter Models

*NOTE: For this procedure, refer to FIGURE 20.4 on page 83.*

Tools Required:

- Diagonal Cutters
- Flathead Screwdriver
- ¼-inch ratchet, ¼-inch socket, and ¼-inch extension
- Pliers
- Tie-wraps

---


### **⚠ DANGER**

**To prevent electrical shock, ALWAYS disconnect concentrator from electrical outlet before servicing.**

---

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Remove control panel. Refer to Removing Control Panel on page 64.
4. Remove the tie-wraps connecting the sieve bed hoses to top of manifold assembly.
5. Remove sieve bed hoses from manifold assembly.
6. Remove and code the two spade connectors from the valve pilot on top of the manifold assembly.
7. Loosen the adjustable clamp that secures valve manifold input hose to heat exchanger.
8. Remove hose from end of heat exchanger.
9. Remove the two mounting screws, washers (1 per side) and bushings (2 per side) securing the 4-way valve/manifold assembly to top of sound box.
10. The 4-way valve/manifold assembly can be removed from the concentrator by lifting the assembly straight up through the cutout in the sound box.
11. Remove 4-way valve from manifold assembly. Refer to Replacing 4-Way Valve on page 77.
12. Remove Y tube and throttle muffler assembly from manifold.
13. Loosen the adjustable clamp on valve manifold input hose to bottom center port of manifold assembly.
14. Remove valve manifold input hose from bottom center port of manifold assembly.
15. Slide 4-way valve/manifold assembly complete through the cutout in top of sound box then install the two mounting screws, washer and bushings into the manifold to secure the 4-way valve assembly to the top of sound box.
16. Reverse STEPS 4-10.
17. Reinstall control panel. Refer to Removing Control Panel on page 64.
18. Plug unit in and turn power On (I).

19. Run unit and check for leaks at hose connections. Refer to Leak Test on page 93.
20. Check for proper system operation. Refer to Typical Product Parameters on page 7.
21. Re-install cabinet. Refer to Removing Cabinet on page 26.

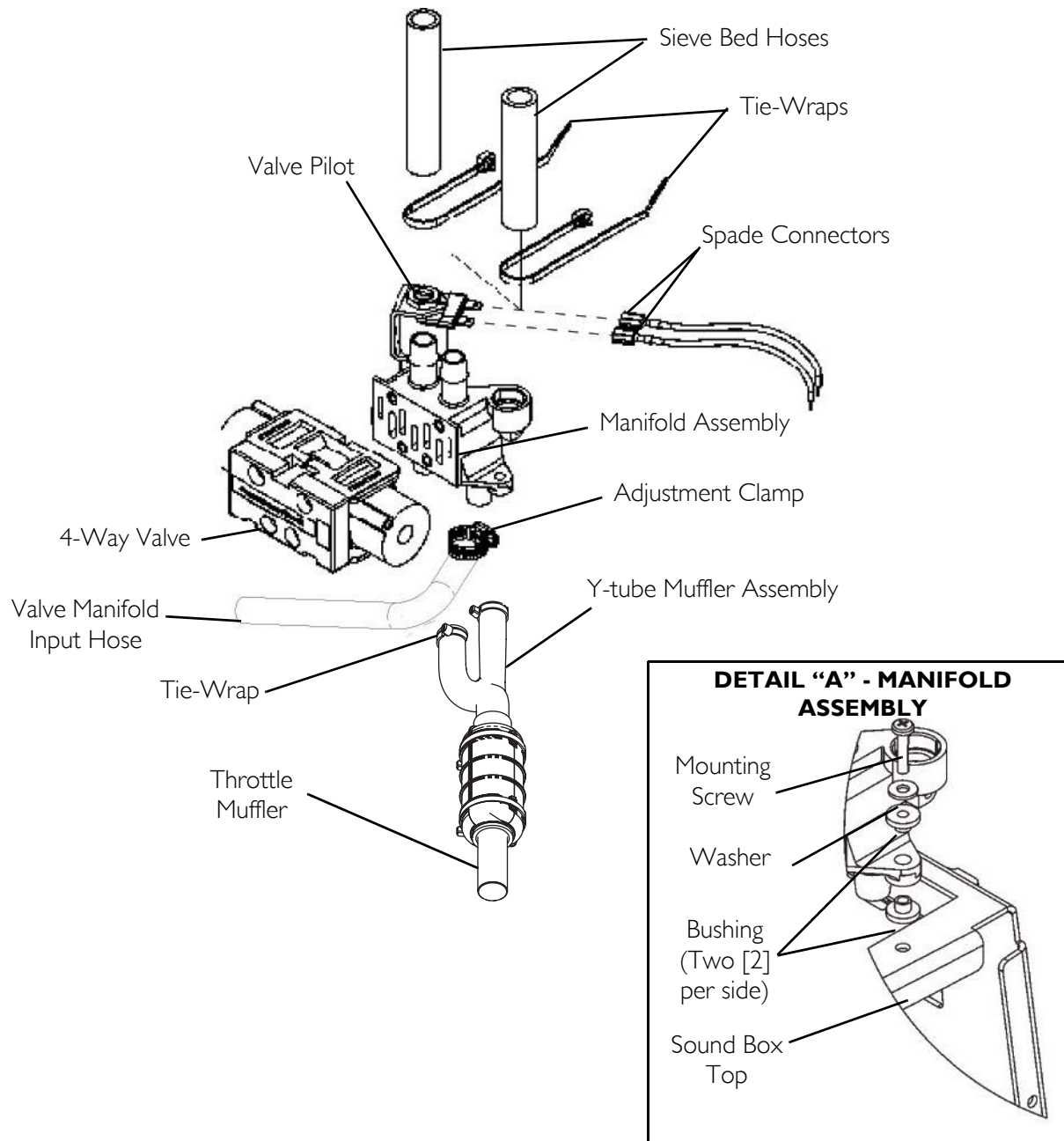

**FIGURE 20.4** Replacing Manifold Assembly - 5 Liter Models Manufactured after 7/24/03 and 9 Liter Models

## Replacing Pilot Valve Poppets and O-Rings

---

### **⚠ DANGER**

**To prevent electrical shock, ALWAYS disconnect concentrator from electrical outlet before servicing.**

---

*NOTE: For this procedure, refer to FIGURE 20.5 and FIGURE 20.6 on page 86.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove cabinet. Refer to Cabinet on page 26.
3. Remove control panel. Refer to Removing Control Panel on page 64.
4. Locate pilot valve on 4-way valve manifold assembly.

*NOTE: Pilot valve can be accessed while the 4-way valve and sieve bed hoses are intact on the concentrator.*

*NOTE: DO NOT remove spade connectors from pilot valve coils. Simply lay coils with spade connectors aside while performing this installation.*

5. Hold pilot valve stem with the flat blade screwdriver and turn the 9/16-inch locknut counterclockwise one complete turn.
  6. Unscrew the pilot valve stem assembly from the manifold while leaving the coil with yoke and locknut intact on stem.
  7. Remove pilot valve poppet from inside the pilot valve stem.
- 

### **CAUTION**

**DO NOT remove the coil yoke from the coil. The washer between the bottom of the yoke and the bottom of the coil MUST not be removed. Otherwise, components may be damaged in reassembly.**

---

8. Set aside the coil with yoke and pilot valve stem assembly with the wires still intact.
- 

### **⚠ WARNING**

**DO NOT use sharp tools to remove O-ring from plastic manifold opening. Damage to the plastic manifold and/or plastic manifold airflow passage may occur. Damage to airflow may result in improper oxygen delivery and harm to the user. Refer to Detail "A" in FIGURE 20.6.**

---

9. Remove poppet and O-ring from manifold opening (Detail "A").
  10. Discard old poppet and O-ring.
  11. Install new O-ring into the opening in the plastic manifold housing (Detail "A").
-

---

**CAUTION**

**DO NOT** overtighten pilot valve stem when installing it into plastic manifold. **Damage to the rubber O ring and/or plastic manifold may occur.**

---

12. Install new slotted poppet into the bottom of the pilot valve stem with tapered end facing to the bottom.
13. Install pilot valve stem and new slotted poppet with coil (with yoke and locknut intact) into manifold opening.
14. Use a flat blade screwdriver, and tighten pilot valve stem clockwise until snug. Torque to  $2.8 \text{ N-M} \pm 0.5 \text{ N-M}$  (25 inch-lbs.  $\pm 5$  in-lbs).
15. Install new label onto pilot valve coil yoke.

*NOTE: DO NOT remove spade connectors from pilot valve coils. Simply lay coils with spade connectors aside while performing this installation.*

16. Position pilot valve coil with yoke at an approximate  $60^\circ$  angle to the right (as viewed from the front of valve) to ensure spade connectors are not pulled or tensioned after reinstallation.
  17. Use the flat blade screwdriver and hold the pilot valve-stem in place.
  18. Tighten locknut clockwise with the 9/16-inch wrench until snug to secure the stem and coil with yoke. **DO NOT** overtighten. Torque to  $2.3 \text{ N-M} \pm 0. \text{ N-M}$  (20 inch-lbs  $\pm 3$  in-lbs).
- 

**CAUTION**

**DO NOT** overtighten locknut when installing it onto pilot valve stem. **Damage to the plastic manifold may occur.**

---



19. Reinstall control panel. Refer to [Removing Control Panel](#) on page 65.
- 

**⚠ WARNING**

**Ensure that hoses behind control panel DO NOT contact cooling fan after control panel reinstallation.**



---

20. Plug power cord in and turn concentrator ON (I) to ensure proper operation.
21. Reinstall cabinet. Refer to [Removing Cabinet](#) on page 26.
22. If any difficulties are encountered in this installation, contact Invacare Technical Service at (800) 832-4707.

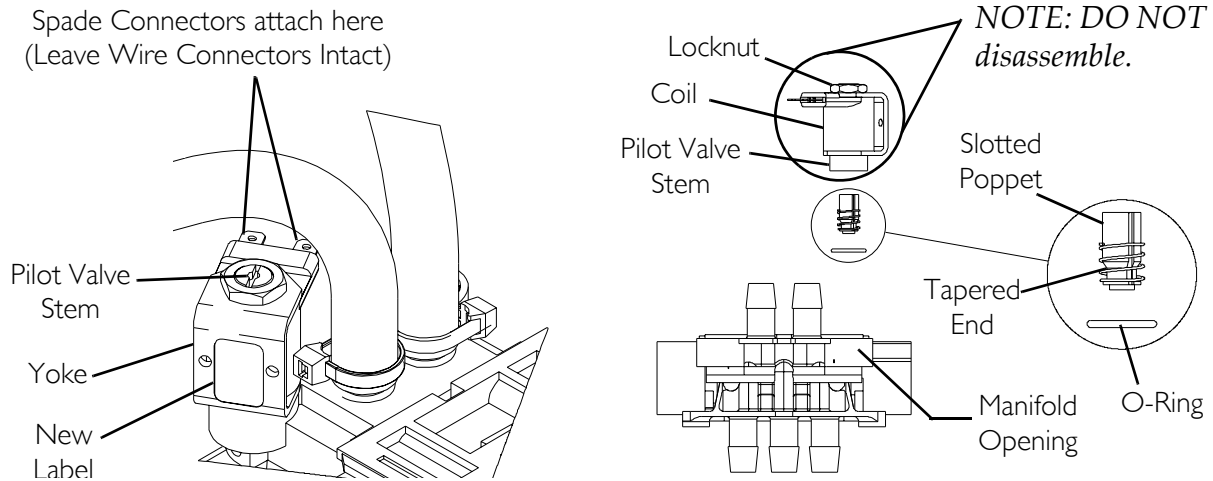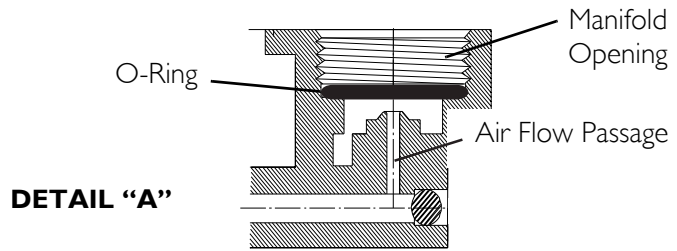

**FIGURE 20.5** Replacing Pilot Valve Poppets and O-Rings - 5 Liter Manufactured Before 8/1/05

**DETAIL "A"**

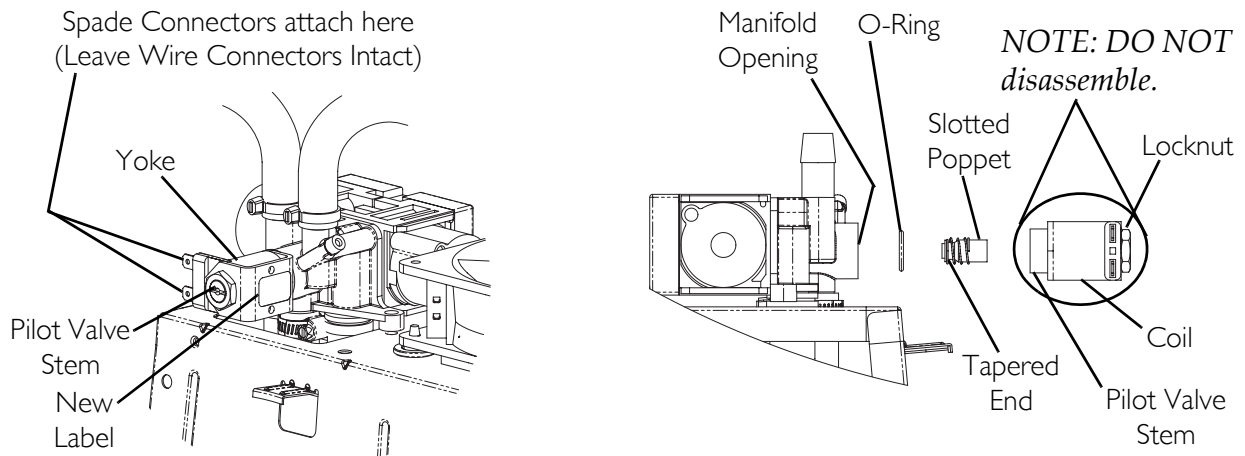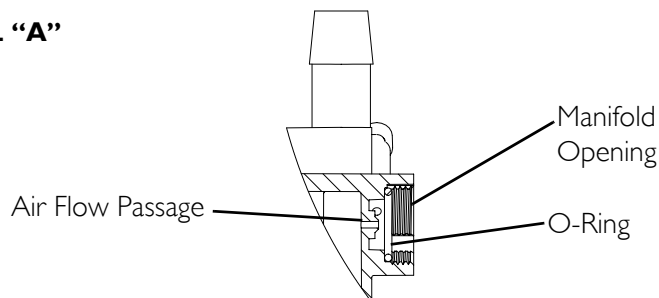

**FIGURE 20.6** Replacing Pilot Valve Poppets and O-Rings - 5 Liter Manufactured After 8/1/05 and 9 Liter models

# SECTION 21—PURITY INDICATORS

## SensO<sub>2</sub> Alarm Indicators

---

### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

---

## Operation

The Platinum with SensO<sub>2</sub> Concentrator is equipped with an oxygen purity indicator. This feature monitors the purity level of the oxygen generated by the concentrator. If purity falls below alarm thresholds, indicator lights on the control panel will illuminate.

*NOTE: Concentrator may be used during the initial start warm-up time (approximately 30 minutes) while waiting for the O<sub>2</sub> purity to reach maximum.*

When the unit is turned on, the GREEN light will come on (SYSTEM OK/O<sub>2</sub> greater than 85%). After five minutes, the oxygen sensor will be operating normally and will control the indicator lights depending on oxygen concentration values. The explanation of the indicator light functions are as follows:

## Indicator Light Explanation (SensO<sub>2</sub> Models)

*NOTE: For this procedure, refer to FIGURE 21.1 on page 88.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*


*NOTE: Indicator lights are visible only when lit.*

GREEN light (O<sub>2</sub>) - Normal Operation.

YELLOW light (⚠) - Call supplier IMMEDIATELY. You may continue to use the concentrator unless instructed otherwise by your supplier. Be certain that backup oxygen is nearby.

RED light (⚡) - Total unit shutdown. Switch IMMEDIATELY to backup oxygen supply. Call supplier IMMEDIATELY.

GREEN light - with YELLOW light flashing - Call supplier IMMEDIATELY. Oxygen sensor malfunctioning; you may continue to use the concentrator.

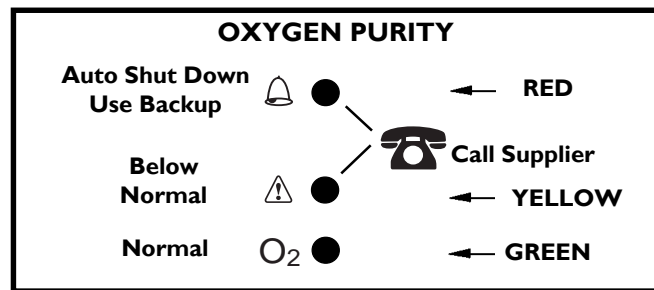


**FIGURE 21.1** Indicator Light Explanation (SensO2 Models)


## Indicator Light Explanation (LX Models)

*NOTE: For this procedure, refer to FIGURE 21.2.*

*NOTE: This procedure applies to Platinum 5 models only.*

RED light () - Total Unit Shut-Down. Switch IMMEDIATELY to a back-up oxygen supply. Call Supplier IMMEDIATELY.

GREEN light f () - ON/OFF. System okay.

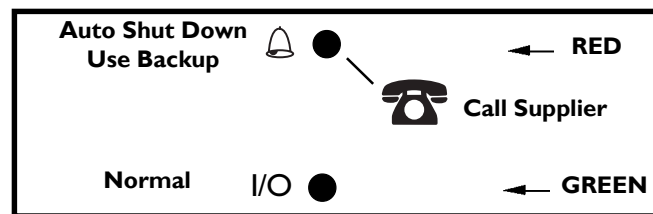

**FIGURE 21.2** Indicator Light Explanation (LX Models)



# SECTION 22—TIMING

## **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

Use extreme care when making adjustments to the timing switch. **DO NOT** allow screwdriver or your hands to contact the P.C. Board when unit is plugged in and/or on. Turn unit off and unplug before adjusting timing switch. Electrical shock is possible.

To accommodate the varying tolerances when replacing components, an adjustable timer is used to control the shift time of the pressure equalization (P.E.) valve. The SensO<sub>2</sub> board allows for automatic (microprocessor controlled) adjustments of the P.E. shift times. Non-SensO<sub>2</sub> boards allow for manual adjustment.

### **Autotuning**

*NOTE: For this procedure, refer to FIGURE 22.1 on page 90.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Turn power off (O) and unplug the unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Plug the unit in and turn the power on (I).
4. Wait for the main valve shift to indicate one of the following:
  - If the board is new, the front panel RED indicator light will slowly blink, indicating autotune has initiated.
  - If the board is used and has performed autotuning before, press and release the small push button switch near the pressure sensor.
  - The front panel RED indicator light will slowly blink, indicating the unit is autotuning and automatically adjusting the P.E. valve timing.
5. Set the output flow to 5.0 L/min. during autotuning.

*NOTE: The GREEN (onboard) diagnostic light will flash the current value of the P.E. valve timing number.*


6. Place the cover back on the unit but **DO NOT** install the cover screws. Let the unit run until autotuning is completed.

*NOTE: When autotuning is complete, the RED front panel indicator light will change from a slow blinking to a flashing of the final P.E. valve timing number.*

7. Turn the unit off and then back on. The timing value is stored in memory for future use.

*NOTE: If the unit is turned off before an autotune cycle is completed, the P.E. valve timing will not be stored or updated. If the board is new, autotuning MUST be successfully completed before the unit can be placed into service.*

8. After a successful autotune, run the unit for 30 minutes and verify that the oxygen concentration output of the unit is within specification.
9. If the oxygen output concentration is not within specification, rerun the autotune again.
10. Reinstall the cover screws.

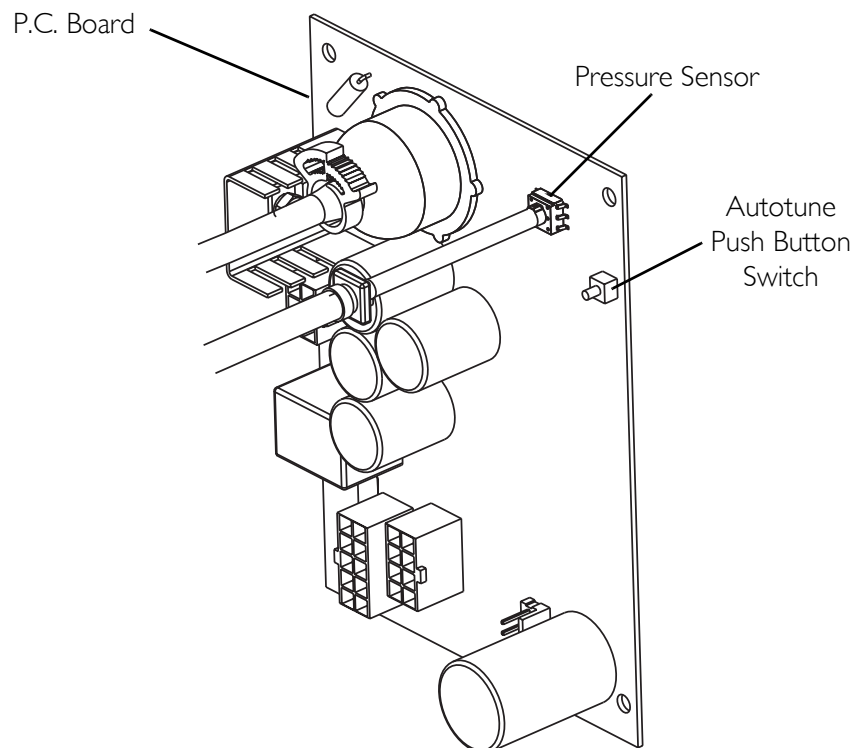

**FIGURE 22.1** Autotuning

## Manual Tuning


*NOTE: For this procedure, refer to FIGURE 22.2 on page 92.*

*NOTE: This procedure applies to Platinum 5 models only.*

1. Turn power off (O) and unplug the unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Plug unit in but DO NOT turn on (I) until instructed to below.
4. Locate the timing switch on the upper right corner of the P.C. board near the pressure sensor.
5. Insert a small common or flathead screwdriver into the adjustment screw of the timing switch.
6. Set the timing switch initially at the number “7” position.
7. Turn unit on (I) and wait 15 minutes for unit to stabilize.
8. If after 15 minutes of run time the concentration is below specification, change setting to a number one position lower than the previous setting.
9. Let unit run ten minutes. If concentration is lower still, change the switch to number two positions higher.

*NOTE: You will find that the concentrator will perform better at either the higher or lower setting. The normal adjustment range is between timing positions 3 and A.*

10. Continue to change setting one step at a time (up or down) until the concentration is the highest value achievable.
11. Once concentration has reached specification, retiming is complete. Refer to Typical Product Parameters on page 7.
12. Reinstall cabinet. Refer to Removing Cabinet on page 26.

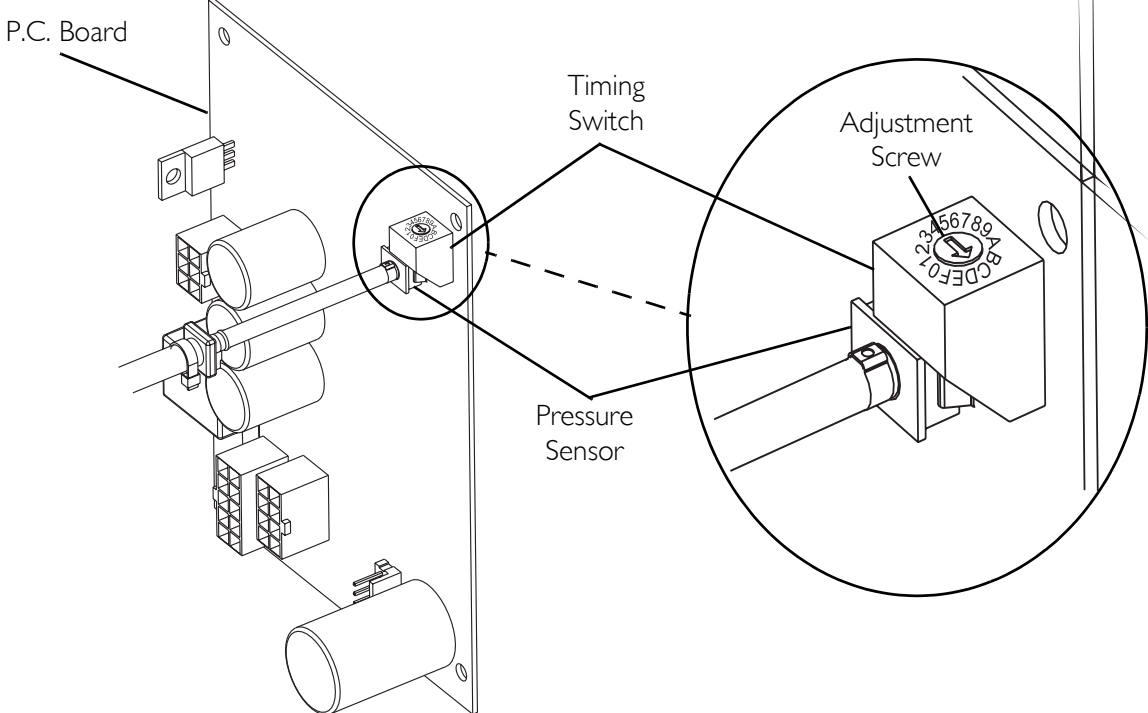

**FIGURE 22.2** Manual Tuning

# SECTION 23—LEAK TEST

## **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

## Leak Test

*NOTE: For this procedure, refer to and FIGURE 23.1 on page 94 and FIGURE 23.2 on page 95.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. Unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Plug in unit.
4. Turn the concentrator on (I).
5. Let concentrator run for 20 minutes.

## **⚠ DANGER**


**Apply small amount of leak test solution to fittings only. Avoid all other components. DO NOT leak test 4-way valve body. Carefully leak test for pressure leaks around sieve bed hoses at 4-way valve. DO NOT allow leak test solution to enter into 4-way valve and system.**

6. With leak-test solution or equivalent soapy water, leak test the following:
  - A. The two sieve bed hoses at the top of 4-way valve and hose to bottom center port on 4-way valve (FIGURE 23.1).


*NOTE: Refer to Preventive Maintenance on page 27 for maintenance of heat exchanger and filters.*

- B. Hose connections to top and bottom fittings of sieve beds.
  - C. P.E. valve hose connections on top of sieve beds (FIGURE 23.2).
  - D. Hose fittings at product tank cap and check valves (FIGURE 23.2).
  - E. The front and rear fittings on heat exchanger (FIGURE 23.2).
7. Replace any tubing that appears cracked, worn, etc.
8. Re-install cabinet. Refer to Removing Cabinet on page 26.
9. Refer to Typical Product Parameters on page 7 for concentration levels. If low concentrations are observed, refer to Troubleshooting on page 18.

**For 5 Liter Models with Canister Muffler (manufactured before 7/24/03)**

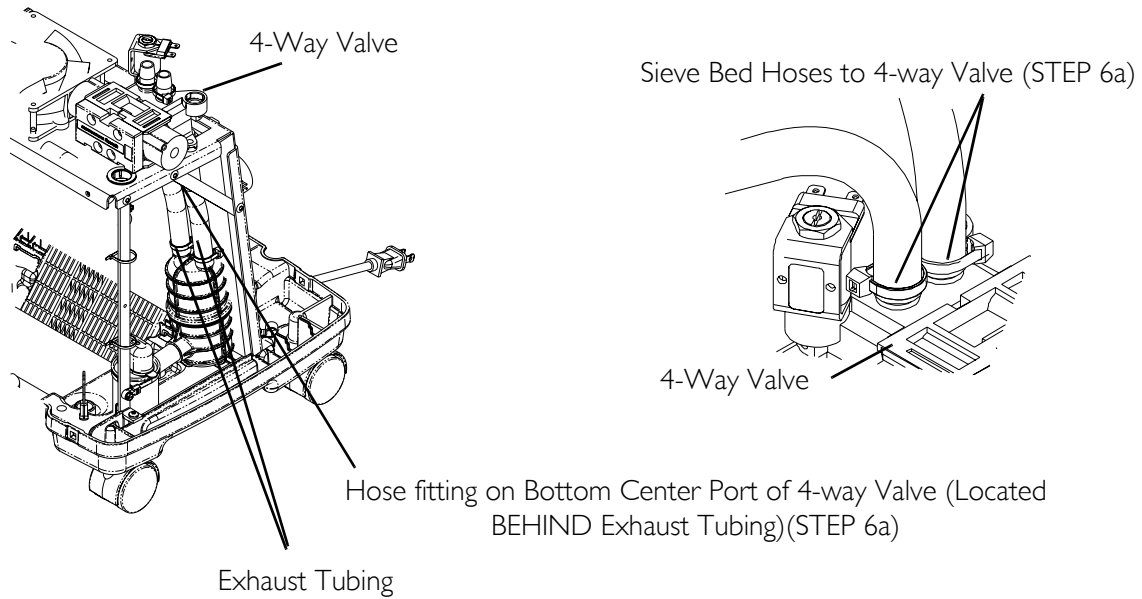


**For 5 Liter Models with Throttle Muffler (manufactured after 7/24/03)  
and 9 Liter Models**

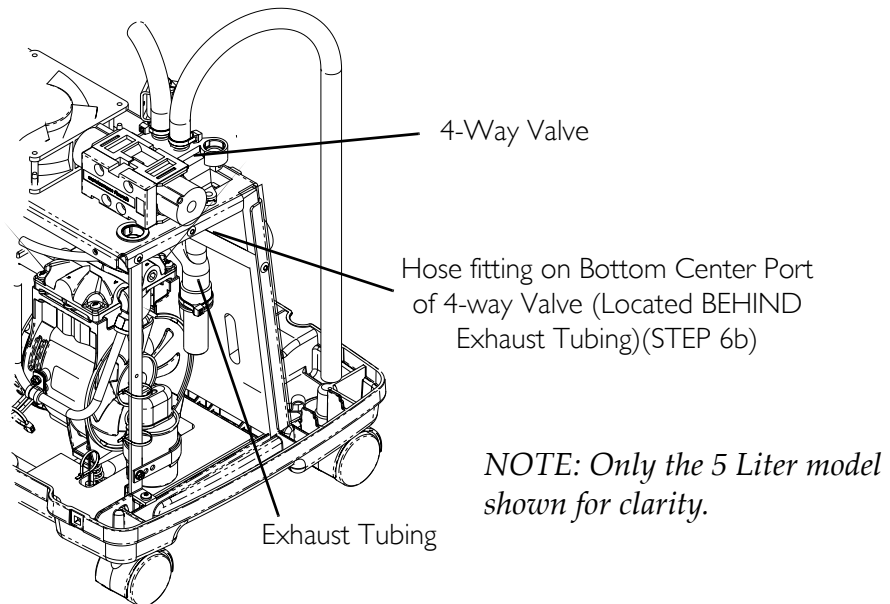


**FIGURE 23.1** Leak Test

**Sieve Bed and Product Tank Hoses**

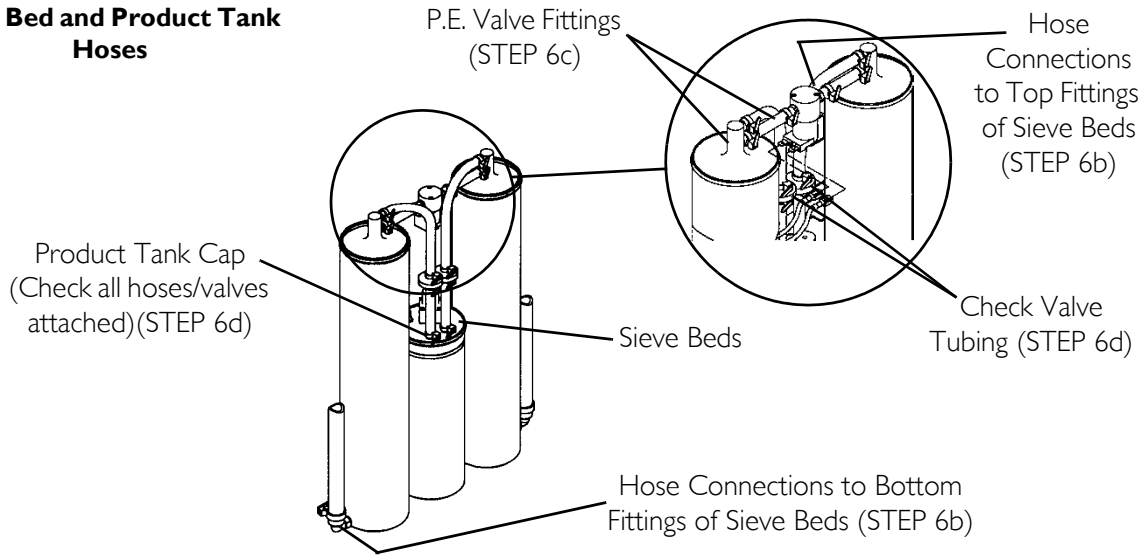


**5 Liter Heat Exchanger Hoses**


**5 Liter Heat Exchanger Hose manufactured before 9-9-05**

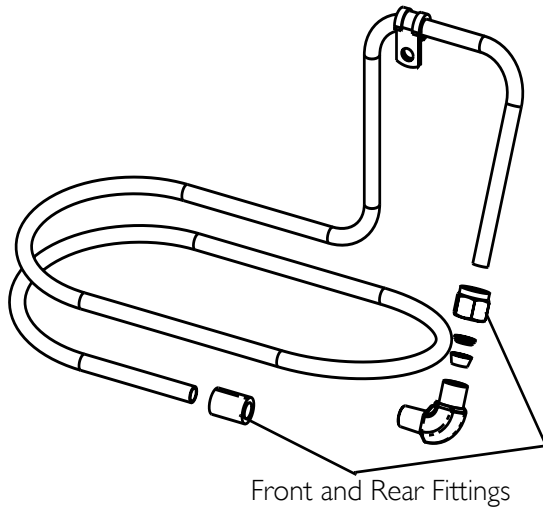

**5 Liter Heat Exchanger Hose manufactured after 9-9-05**

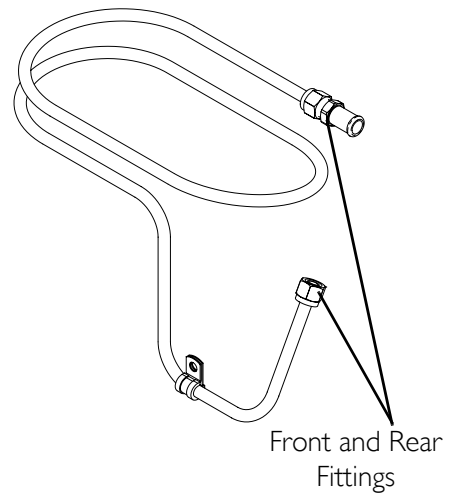

**9 Liter Heat Exchanger Hose**

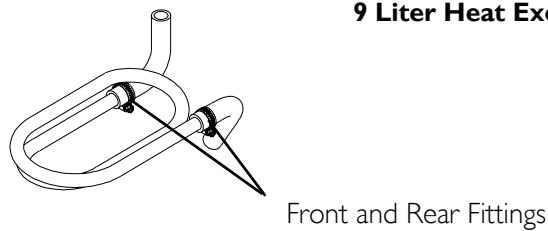

**FIGURE 23.2** Leak Test

# SECTION 24—ALARM TEST

## Alarm and Sensor Tests

---

### **DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

Check alarms periodically for proper function.

---

### **Power Loss Alarm**

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

With the unit running, remove the line cord from power source. The power loss alarm mode should sound the audible alarm IMMEDIATELY or within 30 seconds.

*NOTE: With the unit unplugged and inoperable for a short period of time, the battery free circuit will be drained. If power failure alarm does not sound with unit unplugged and power switch on, the battery free circuit is drained. It will recharge when unit is plugged in and switched on.*

### **Low Pressure Test**

*NOTE: For this procedure, refer to FIGURE 24.1 on page 97.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. There are two separate failure modes for Low Pressure:
  - Low product tank pressure. (Pressure in tank drops below a preset value, typically 48 kPa (7 p.s.i.).
  - Failure to reach the set point pressure within a prescribed time limit, or Timeout Failure.

#### **Test One**

1. With unit running, set flow at maximum rating.
2. When the main valve switches, pull the stem on the compressor relief valve out as far as it will go and hold it.
3. The low pressure alarm should activate within 30 seconds. Refer to SensO2 Indicators on page 99 for shutdown mode.

#### **Test Two**

1. With unit running, set flow at maximum rating.
2. Remove the pressure sensor tubing (tie-wrap) from the top of the product tank (not shown).



3. The low pressure alarm should activate within 30 seconds. Refer to SensO2 Indicators on page 99 for shutdown mode.
4. Replace tubing and tie wrap.


### Time-Out Test

*NOTE: For this procedure, refer to FIGURE 24.1.*

*NOTE: This procedure applies to Platinum 5 and Platinum 9 models.*

1. With unit off (O), disconnect the compressor connector from the main wiring harness.
2. Plug unit in and turn power on (I).
3. The Time-Out Failure alarm should activate within 40 seconds. Refer to SensO2 Indicators on page 99 for shutdown mode.
4. Replace tubing and tie wrap.

*NOTE: If any alarm fails to perform to specification, contact Invacare Technical Service.*

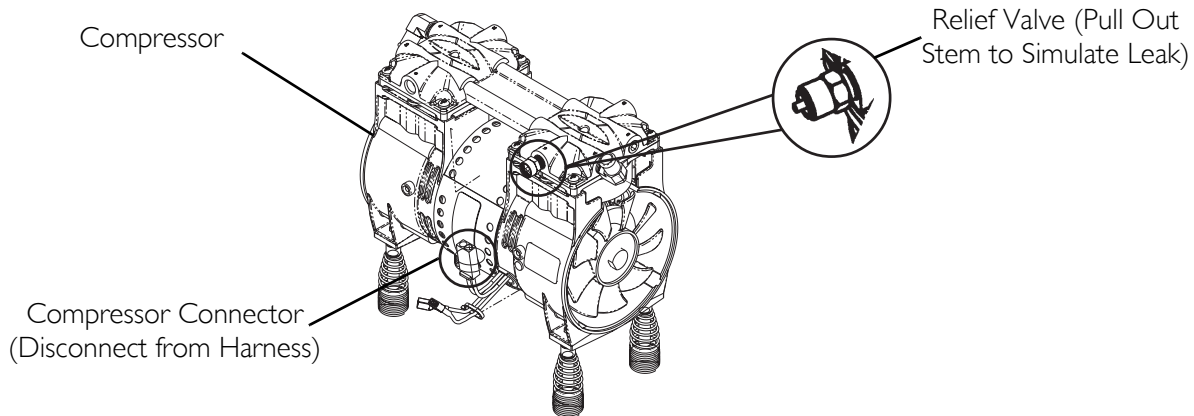



**FIGURE 24.1** Low Pressure Test - Time-Out Test

### High Pressure Test

*NOTE: For this procedure, refer to FIGURE 24.2 on page 98.*

- High Pressure - Occurs when pressure in product tank rises beyond a preset value, typically 158 kPa  $\pm$  7 kPa (23 p.s.i.  $\pm$  1 p.s.i.).
  - Main Valve Coil - Alarm sequence occurs when the main valve coil, connection or P.C. board circuitry has failed.
1. With the unit running and flow set at 5 L/min for 5 liter models and 9 L/min for 9 liter models, remove one orange wire from the main valve coil.
  2. The main valve coil alarm should activate within 40 seconds. Refer to SensO2 Indicators on page 99 for shutdown mode (FIGURE 24.3).
  3. Replace tubing and tie wrap.

*NOTE: If any alarm fails to perform to specification, contact Invacare Technical Service.*

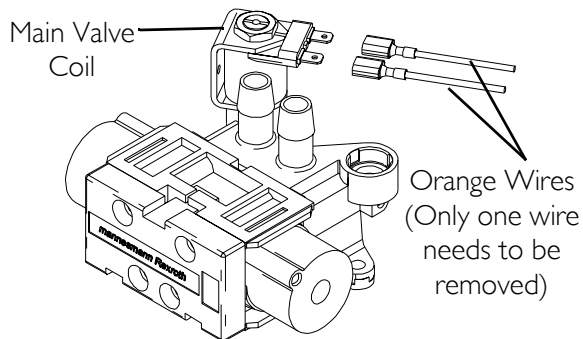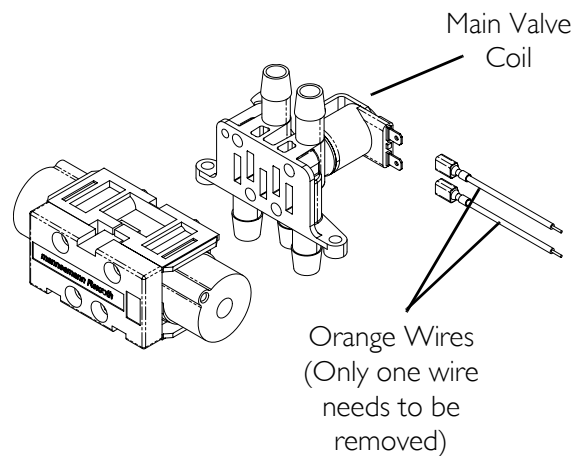**5 Liter models manufactured Before 8/1/05****5 Liter models manufactured After 8/1/05 and 9 Liter models****FIGURE 24.2** High Pressure Test**Oxygen Sensor**

*NOTE: For this procedure, refer to FIGURE 24.3 on page 99.*

Alarm sequence occurs when concentration levels fall below a preset value, typically  $73\% \pm 3\%$  or  $85\% \pm 2\%$ . This failure indicates a catastrophic failure of any component mechanical or electrical.

**Oxygen Sensor Test**

1. Turn unit on (I). Set output flow at 5 L/min. for 5 Liter models and 9 L/min for 9 Liter models.
2. Monitor the O<sub>2</sub> level.
3. With the O<sub>2</sub> level greater than 85% after five minutes, the GREEN panel indicator light will remain illuminated.
4. Slowly adjust flow beyond maximum rated flow until you can achieve a concentration level greater than 75% but less than 84%. Within 30 minutes the YELLOW panel indicator light will illuminate and the unit will continue to run.
5. Clamp off the 1/8-inch oxygen sensor tubing between the oxygen sensor and the product tank regulator.

**⚠ WARNING**

**Use care not to cut oxygen sensor tubing when clamping the oxygen sensor tubing.**

6. Within 30 minutes, the Oxygen Sensor Alarm should activate and the RED panel light indicator will illuminate. The compressor will shut down with the alarm.

SENSO<sub>2</sub> INDICATORS

LABEL SYMBOL	STATUS	INDICATOR LIGHTS (LED)
O <sub>2</sub>	SYSTEM OKAY O <sub>2</sub> over 85% (± 2%)	GREEN Indicator Light
⚠	O <sub>2</sub> Between 73% (± 3%) to 85% (±2%)	YELLOW Indicator light A. YELLOW Solid B. YELLOW Flashing Sensor Failure Call a qualified technician.
🔔	SYSTEM FAILURE O <sub>2</sub> Below 73% (±3%)	Continuous Audible Alarm Sieve- Gard™ Compressor shutdown. Call qualified technician.

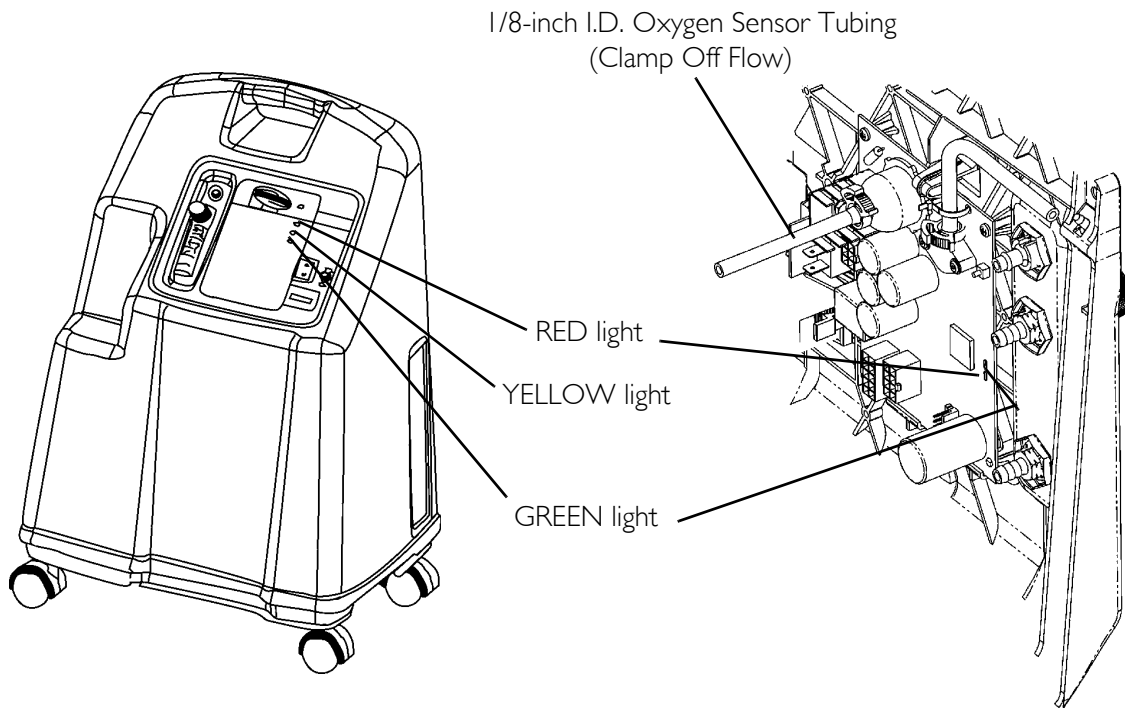


FIGURE 24.3 Oxygen Sensor

**P.E. Valve Coil Test**


NOTE: For this procedure, refer to FIGURE 24.4.

NOTE: P.E. Valve Coil - Alarm sequence occurs when the P.E. Valve Coil, connection or P.C. Board Circuitry has failed.

1. With the unit running and flow set at 5 L/min. for 5 Liter models and 9 L/min for 9 Liter models, remove one YELLOW wire from the P.E. valve coil. The P.E. valve coil alarm should activate within 10 seconds.
2. Reinstall cabinet. Refer to [Removing Cabinet](#) on page 26.


NOTE: If any alarm fails to perform to specification, contact Invacare Technical Service.

NOTE: If problems arise during test, contact Invacare Technical Service for further assistance.

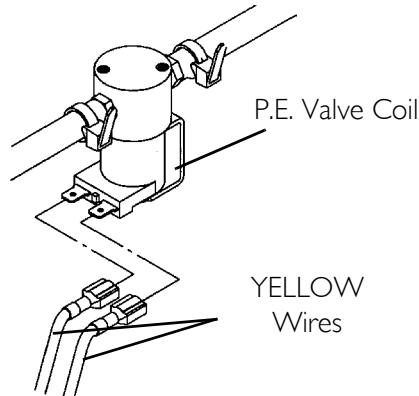

**FIGURE 24.4** P.E. Valve Coil Test

3. Refer to Preventive Maintenance Record Invacare Platinum Oxygen Concentrator on page 36 to record date and number of hours when preventive maintenance was performed on the concentrator, or any repairs made.

 <b>OXYGEN CONCENTRATOR</b> Refer To Service Manual <b>DIAGNOSTIC ALARM SYSTEM</b>			
...ADJUSTMENTS BY UNAUTHORIZED PERSONNEL VOIDS WARRANTY...			
ALARM	LED FLASHES		PROBABLE CAUSE
	RED	GREEN	
BEEP AT START UP	0	0	NO PROBLEM, SYSTEM OK
SHORT BEEP WITH LONG PAUSE	0	0	MAIN POWER LOSS
CONTINUOUS	1	1	LOW PRESSURE; MAJOR LEAK
CONTINUOUS	1	2	HIGH PRESSURE; NO SWITCHING
CONTINUOUS	1	3	TIME OUT FAILURE; COMPRESSOR
CONTINUOUS	1	5	PILOT VALVE CIRCUIT
CONTINUOUS	2	1	73% SHUTDOWN; LOW O <sub>2</sub>
CONTINUOUS	2	3	PE VALVE COIL
CONTINUOUS	2	4	EEPROM FAILURE
CONTINUOUS	3	1	OXYGEN SENSOR FAILURE
NOTE: CYCLE POWER ON AND OFF FIVE (5) TIMES TO CLEAR INTERNAL FAULT CODE.			
PREVENTIVE MAINTENANCE SCHEDULE	Cabinet Filter(s) . . . . .Clean Weekly, Replace as Needed Heat Exchanger. . . . .Clean as Needed Compressor Inlet Filter. .Check Annually or During Scheduled PM Outlet HEPA Filter. . . .Check Annually or During Scheduled PM Compressor. . . . .Rebuild as Needed Exhaust Muffler. . . . .Replace as Needed		
1115180			

# SECTION 25— SIEVE BED PRESSURE CHECK

## Models without HomeFill Connection On Back Of Unit

### Installing the Sieve Bed Pressure Tap-In Kit and Checking Bed Pressure


#### **⚠ DANGER**

To prevent electrical shock, **ALWAYS** disconnect concentrator from electrical outlet before servicing.

#### Disassembly

*NOTE: For this procedure, refer to FIGURE 25.1.*

1. Turn power off (O) and unplug unit.
2. Remove cabinet. Refer to Removing Cabinet on page 26.
3. Turn unit so the back faces you.
4. Loosen and lower the adjustable clamp on the product tank.
5. Pull product tank up and back to access the check valve PVC tubing.
6. Locate the check valve and PVC tubing to your right.
7. Cut the PVC tubing below the check valve in half.

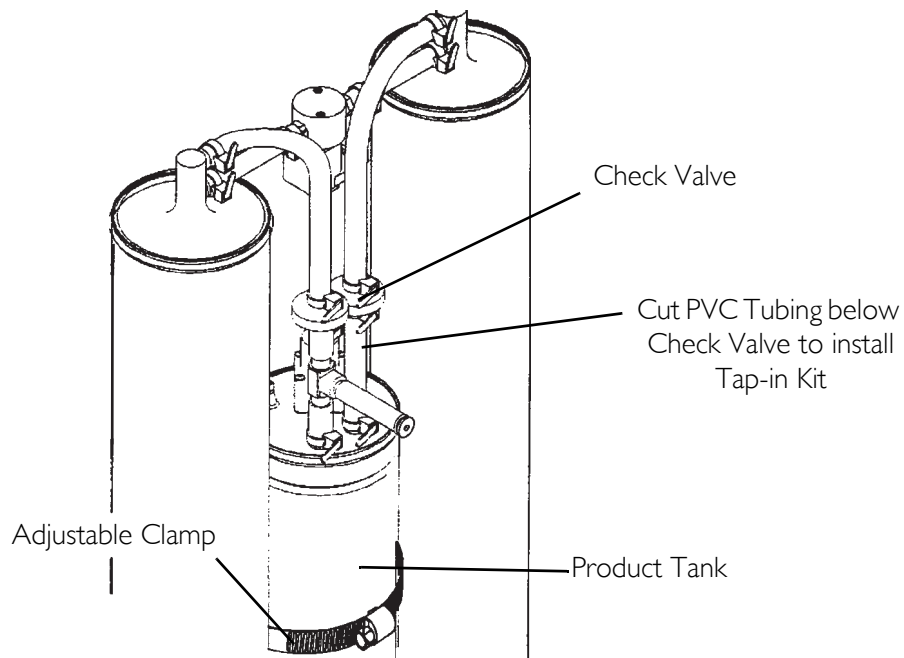


**FIGURE 25.1** Disassembly

**Installation**

*NOTE: For this procedure, refer to FIGURE 25.2.*

1. Insert the T fitting into the ends of the PVC tubing.
2. Secure 6 mm ( $\frac{1}{4}$ -inch) clamps to each end of the PVC tubing “T” fitting connection.
3. Attach the 6 mm x 5 cm ( $\frac{1}{4}$  x 2-inch) silicone tubing to the new “T” fitting and secure with a 6 mm ( $\frac{1}{4}$ -inch clamp).
4. Insert plug into the open end of the 6mm x 5 cm ( $\frac{1}{4}$  x 2-inch) silicone tubing.
5. Secure with a 6 mm ( $\frac{1}{4}$ -inch) clamp.

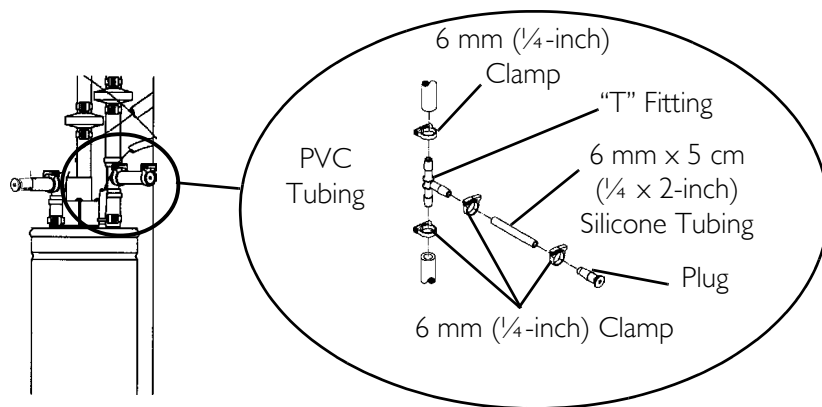

**FIGURE 25.2** Installation

**Pressure Testing**

*NOTE: For this procedure, refer to FIGURE 25.3 on page 103.*


1. Remove plug and connect pressure gauge to the 6 mm x 5 cm ( $\frac{1}{4}$  x 2-inch) silicone tubing and secure with a 6 mm ( $\frac{1}{4}$ -inch) clamp.
2. Check each connection to make sure it is tight.
3. Turn the concentrator On (I).

*NOTE: Adjust the concentrator to maximum Rated Liter flow (5 L/min. for 5 Liter and 9 L/min. for 9 Liter).*

*NOTE: Wait approximately five minutes to allow system pressures to achieve proper equilibrium before testing.*

4. The pressure in the concentrator should rise within 20 seconds to 145 kPa  $\pm$  2 kPa (21 p.s.i.g.  $\pm$  0.3 p.s.i.g.) for 5 Liter and 172.36  $\pm$  2 kPa (25 p.s.i.g.  $\pm$  0.3 p.s.i.g.) for 9 Liter and shift the 4-way valve to the opposite sieve bed.
5. After the 4-way valve shift, the pressure in the concentrator will drop to 96 kPa  $\pm$  14 kPa (14 p.s.i.g.  $\pm$  2 p.s.i.g.) before rising within 20 seconds to 145 kPa  $\pm$  2 kPa (21 p.s.i.g.  $\pm$  0.3 p.s.i.g.) for 5 Liter and 172.36  $\pm$  2 kPa (25 p.s.i.g.  $\pm$  0.3 p.s.i.g.) for 9 Liter and shift the 4-way valve to the opposite sieve bed.
6. If pressure reading is 145 kPa  $\pm$  2 kPa (21 p.s.i.g.  $\pm$  0.3) for 5 Liter and 172.36  $\pm$  2 kPa (25 p.s.i.g.  $\pm$  0.3 p.s.i.g.) for 9 Liter when both valve shifts are made, proceed to STEP 8.

7. If pressure in the concentrator does not fall into the specifications listed above, refer to [Troubleshooting](#) on page 18.
8. Turn the concentrator Off (O).
9. Unclamp the pressure gauge and remove it from the T fitting.

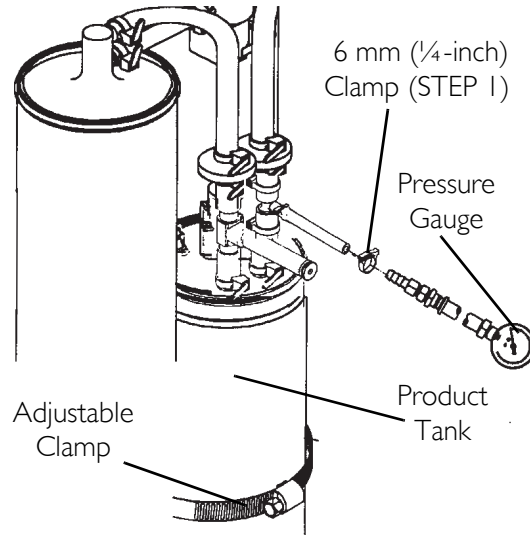


**FIGURE 25.3** Pressure Testing

### After Testing

*NOTE: For this procedure, refer to FIGURE 25.4.*

1. Insert 6 mm (1/4-inch) plug and 6 mm x 5 cm (1/4 x 2-inch) silicone tubing onto the T fitting and secure with 6 mm (1/4-inch) clamp.
2. Reinstall the cabinet. [Refer to Removing Cabinet](#) on page 26.

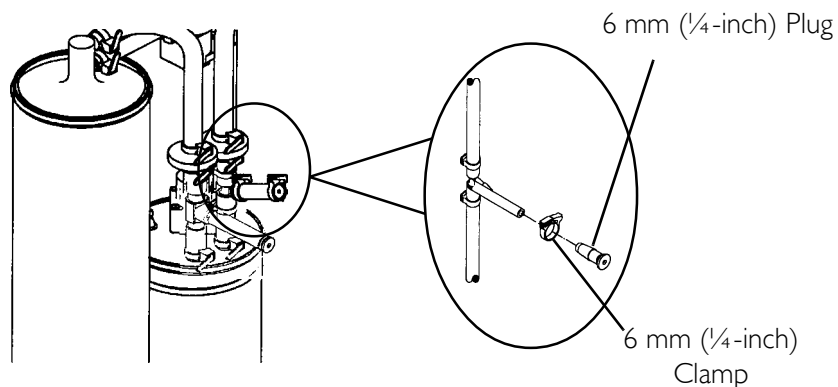

**FIGURE 25.4** After Testing

## Models With HomeFill Connection On Back Of Unit.

### Pressure Testing

*NOTE: For this procedure, refer to FIGURE 25.5 on page 105.*


1. Connect pressure gauge to the ¼ x 2-inch silicone tubing with connector (p/n 1131392).
2. Remove plug in HomeFill port in back of concentrator.
3. Connect ¼ x 2-inch silicone tubing with connector to HomeFill port.
4. Check each connection to make sure it is tight.
5. Turn the concentrator On (I).

*NOTE: Adjust the concentrator to maximum rated Liter flow to 5 L/min. for 5 Liter and 9 L/min. for 9 Liter.*

*NOTE: Wait approximately five minutes to allow system pressures to achieve proper equilibrium before testing.*

6. The pressure in the concentrator should rise within 20 seconds to 21 p.s.i.g.  $\pm 0.3$  and for 5 Liter and 25 p.s.i.g. for 9 Liter shift the 4-way valve to the opposite sieve bed.
7. After the 4-way valve shift, the pressure in the concentrator will drop to 12 p.s.i.g.  $\pm 2$  before rising within 20 seconds to 21 p.s.i.g.  $\pm 0.3$  for 5 Liter and 25 p.s.i.g.  $\pm 0.3$  p.s.i.g. for 9 Liter and shift the 4-way valve to the opposite sieve bed.
8. If pressure reading is 21 p.s.i.g.  $\pm 0.3$  for 5 Liter and 25 p.s.i.g. for 9 Liter when both valve shifts are made, proceed to STEP 10.
9. If pressure in the concentrator does not fall into the specifications listed above, refer to [Troubleshooting](#) on page 16.
10. Turn the concentrator off (O).
11. Disconnect pressure gauge from the HomeFill port and reinstall plug.



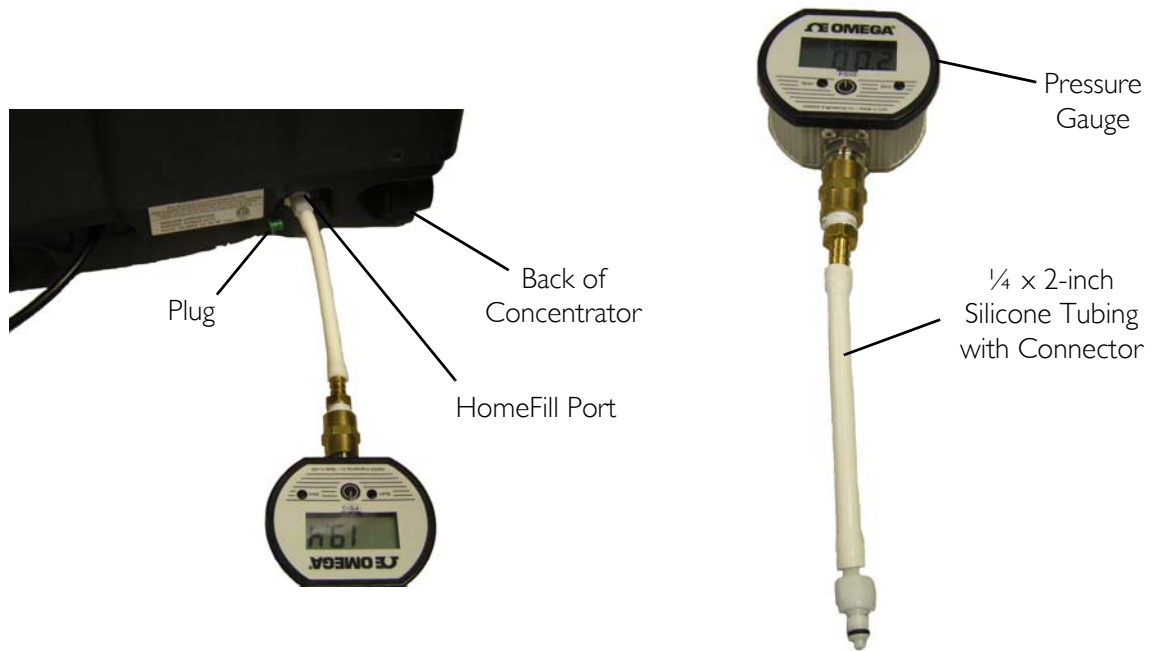

**FIGURE 25.5** Pressure Testing

# NOTES

## LIMITED WARRANTY

**NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser who purchases this product when new and unused from Invacare Corporation or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person. This warranty gives you specific legal rights and you may also have other legal rights which may vary from state to state.

Invacare Corporation warrants its Platinum Oxygen Concentrators when purchased new and unused to be free from defects in materials and workmanship for a period of five (5) years from date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty.

**DO NOT OPEN OR ATTEMPT TO SERVICE THE 4-WAY VALVE BODY.** This will void any and all warranty applicable to the valve body. If within such warranty periods any such product shall be proven to Invacare Corporation's satisfaction to be defective, such product shall be repaired or replaced, at Invacare Corporation's option. This warranty only applies to the labor for repairs performed by the Invacare Service Department or Invacare Authorized Service Centers. It does not apply to the labor performed by the purchaser or user. This warranty does not include normal wear and tear or shipping charges. Invacare Corporation's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement. Routine maintenance items, such as filters, are excluded from this warranty.

For warranty service, please contact Invacare Corporation's Service Department at the toll free number on the back cover. Upon receiving notice of an alleged defect in a product, Invacare Corporation will issue a serialized return authorization. It shall then be the responsibility of the purchaser to return the entire unit or remove, at purchaser's cost, the defective component part(s) identified, pack the component part(s) in a manner to avoid shipping damage and to ship the component part(s) to either Invacare Corporation's plant or service center as specified by Invacare Corporation in advance. Defective component part(s) **MUST** be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, SOOT OR SMOKE-FILLED ENVIRONMENTS, OR OTHER THAN NORMAL APPLICATION, USE OR SERVICE, OR TO PRODUCTS MODIFIED WITHOUT INVACARE CORPORATION'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE CORPORATION OR TO PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE CORPORATION'S CONTROL. THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT APPLY TO YOU.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**



**Yes, you can.**

**Invacare Corporation**

[www.invacare.com](http://www.invacare.com)

**USA**

One Invacare Way  
Elyria, Ohio USA  
44036-2125  
800-333-6900  
Technical Services  
800-832-4707

EU Representative  
Invacare EC - Hong A/S  
Factory  
Ostergade 3  
DK - 4270 Hong  
Denmark  
Tel: 011-45-58852722  
Fax: 011-45-58854386

Manufacturer:  
Invacare Medical Equipment (Suzhou)  
Co., Ltd.  
Unit 2B, Export Processing Zone  
No. 200 Suhong Road SIP  
(Suzhou Industrial Park)  
Suzhou, Jiangsu, PRC  
215021  
Tel: 86-512-62586192  
Fax: 86-512-62586072



All rights reserved. Trademarks are identified by <sup>TM</sup> and <sup>®</sup>.  
All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.  
Teflon is a registered trademark of DuPont Corp.  
Torx is a registered trademark of Textron, Inc.  
© 2007 Invacare Corporation

Part No 1118396  
Rev D - 8/07

